



# Chat Fusion 10 · 1<sup>st</sup> Generation

## First Steps

### 1 Charge the Chat Fusion™ Battery

You received a standard battery charger with your device. We suggest charging the battery each night.

Plug the charger into an electrical outlet—a surge protector is recommended. Plug the charger cable into the charging port on the device.



Standard charger



Charging port

You also received a charge pad with a power cable.



Device on Charge Pad

1. Plug the charge pad's power cable into an electrical outlet—a surge protector is recommended.
2. Plug the other end of the power cable into the power jack on the top edge of the charger. The charger's left LED will glow amber.
3. Position the Chat Fusion device's stand against the back of the device.
4. Center the device on the charger with the back of the device against the charger.
5. Charging should begin automatically, and the charger's right LED should glow blue. If the LED does not glow blue, adjust the position of the device in the charger until the LED glows blue.



*Using either charger:* When the battery charge is 10% or less, the Chat Fusion's red charging LED flashes. During charging, the red charging LED is illuminated. When the battery is fully charged, the blue charging LED is illuminated.

### 2 Call for Training

Call a speech and language pathologist or a Saltillo consultant for detailed training and guidance with your device. For a list of consultants in your area, go to <http://saltillo.com/reps>.


### 3 Identify Device Features

For detailed information on using these features, refer to the *Chat Fusion User's Guide*.

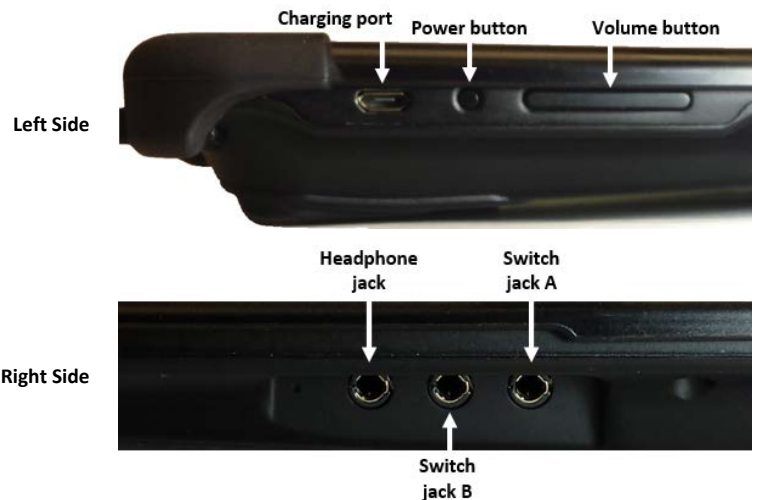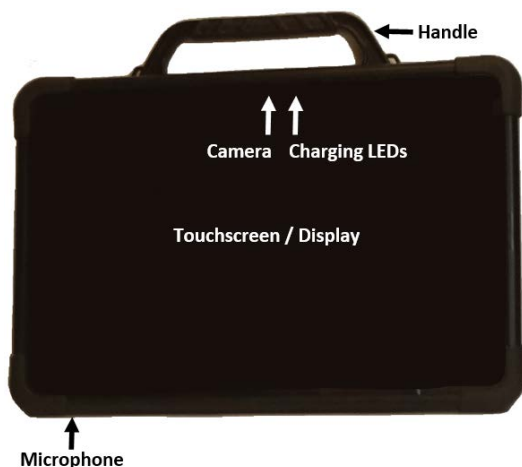

## 4 Choose a Vocabulary File


If you want to use a different vocabulary file than the one that is currently active, you can select an alternative. Your speech and language specialist may assist in choosing the best vocabulary.



1. Tap on the **Menu** button .
2. Tap on **Library**.
3. Scroll up and down the list to view the available vocabulary files.
4. Select the file you want.



## 5 Choose a Voice


If you want to use a different voice than the one that is currently active, you can choose from a variety of voices.

1. Tap on the **Menu** button .
2. Tap on **Settings**.
3. Select Speech Output.

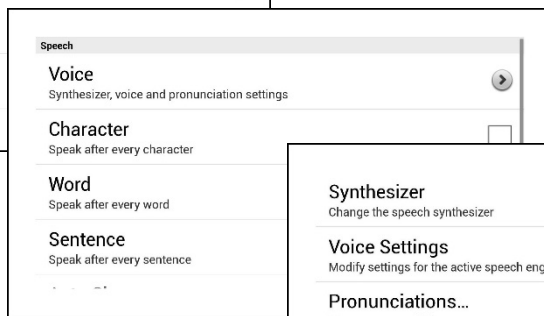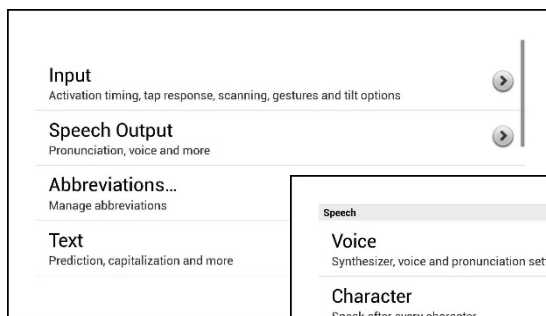


4. Tap on **Voice**.\*

5. Tap on **Voice Settings**.

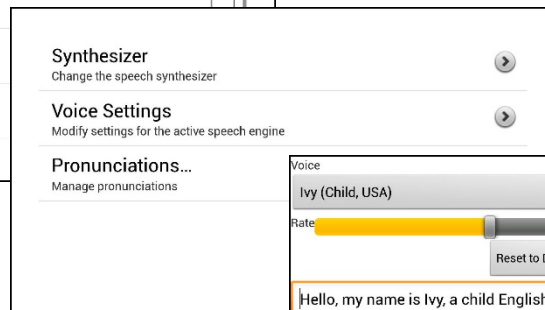

6. Tap on the arrow on the top far right to open the drop-down menu.
7. Tap on the voice you want.

*\* If you purchased Acapela Synthesizer:*  
After you tap on **Voice** (step 4), tap on **Synthesizer** and select **Acapela**. Then proceed to step 5.

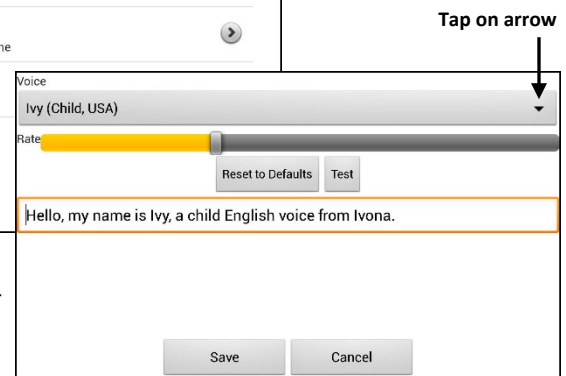

8. Tap on **Save**.
9. Tap on the **Back** button until you return to the vocabulary page.

## 6 Now, Start Exploring Your Device! Refer to the *Chat Fusion User's Guide* for Assistance.