TouchChat™ Express-D
TouchChat™ Express-mini-D

English/Spanish User's Guide

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 Saltillo Corporation
2143 Township Road 112
Millersburg, OH 44654
www.saltillo.com

TouchChat™ Express-D
TouchChat™ Express-mini-D

User's Guide

 Saltillo Corporation
2143 Township Road 112
Millersburg, OH 44654
www.saltillo.com

10144-D-en Version 1.5

Acknowledgements

Copyright/Trademark Information

© 2015-2016 Saltillo Corporation. All rights reserved.

TouchChat is a trademark of Saltillo Corporation.

Guided Access, iTunes, and Mac are registered trademarks of Apple Inc.

iOS is a trademark or registered trademark of Cisco in the U.S. and other countries and is used under license.

The Bluetooth word mark and logos are registered trademarks owned by Bluetooth SIG, Inc.

Symbol Stix is copyright of News2You.

WordPower is a trademark of Inman Innovations, Inc.

10144-D-en Version 1.5 · 07/26/2016

Electronic Interference Information

This device complies with Part 15 of the FCC Rules. Operation is subject to two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Warning! Changes or modifications to this product which are not authorized by Saltillo Corporation could exceed FCC limits and negate your authority to use this product.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. You can determine whether this product is causing interference in your radio or television by turning this product off. If the interference stops, it was probably caused by this product or one of its accessories. You can attempt to correct the interference by using one or more of the following measures:

- Turn the television or radio antenna until the interference stops.
- Move this product to one side or the other of the television or radio.
- Move this product farther away from the television or radio.
- Plug this product into an outlet that is on a different circuit from the television or radio; that is, this product should be controlled by different circuit breakers/fuses from the television or radio.
- If necessary, contact a Saltillo Corporation service technician for assistance.

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 **Warnings**

This device is not intended to be an emergency call device or sole communication aid.

When operating this device in a medical environment, do not use with any product that is not medically approved. Follow all rules for appropriate wireless device use.

Do not use this device close to sources of RF radiation or you may encounter interference. Move away, if possible, from the source of the interference.

Any mounts used should be fitted by a qualified person. Failure to install the mounting system according to the manufacturer's instructions may result in an injury to the user.

Analysis of positioning by a qualified person is required to prevent repetitive stress injuries to the user.

Cords and straps are potential entanglement/strangulation hazards. Please consider this prior to placing these items with device users.

Small parts could present a choking or other hazard

Do not use the device if the screen is cracked or broken.

Saltillo Corporation assumes no responsibility for any loss or claims by third parties which may arise through the use of this product.

Saltillo Corporation assumes no responsibility for any damage or loss caused by the deletion of data as a result of malfunction repairs or battery replacement. Be sure to back up all important data on other media (computer) to protect against its loss.

Implantable Medical Devices

A minimum separation of six (6) inches should be maintained between a handheld wireless mobile device and an implantable medical device, such as a pacemaker or implantable cardioverter defibrillator, to avoid potential interference with the device.

Persons who have such devices:

- Should ALWAYS keep the mobile device more than six (6) inches from their implantable medical device when the mobile device is turned ON;
- Should not carry the mobile device in a breast pocket;
- Should turn the mobile device OFF immediately if there is any reason to suspect that interference is taking place
- Should read and follow the directions from the manufacturer of your implantable medical device. If you have any questions about using your wireless mobile device with an implantable medical device, consult your health care provider.

Contents

Introduction.....	7
Identifying TouchChat Express Features	7
Charging the Device and Amplifier.....	8
Turning the Amplifier On and Off.....	9
Adjusting the Volume and Muting Sounds.....	10
Using the Touch Screen.....	10
Identifying Status Icons.....	11
Syncing with iTunes	12
Connecting to a Computer for the First Time	13
Backing up and Restoring Information	14
Backing up the Device	14
Restoring the Device	14
Updating the Device Software	14
Using TouchChat.....	15
Finding the TouchChat Version	15
Working with TouchChat Menus.....	15
Working with Vocabulary Files	16
English Vocabulary Files.....	17
Spanish Vocabulary Files.....	20
Downloading or Purchasing Additional Vocabulary Files	20
Loading a Vocabulary File	21
Copying a Vocabulary File	21
Creating a New Blank Vocabulary File	22
Deleting a Vocabulary File	22
Using Chat Editor with TouchChat	23
Downloading and Installing Chat Editor	23
Configuring Chat Editor.....	24
Enabling Speech in Chat Editor	24
Opening Chat Editor	24
Using Chat Editor to Create Instructional Materials.....	25
Turning Edit Mode On	26
Working with Buttons	26
Modifying Text.....	26
Inserting Special Characters.....	26
Adding a Symbol from the SymbolStix Library.....	27
Adding a Picture from the Image Library.....	27
Using the Camera to Take a Picture for Your Button.....	28
Creating a Camera Button	28
Resizing a Button	29
Changing a Button Style.....	29
Changing a Button Action	29

Removing or Changing the Order of an Action	29
Creating a Button to Link to a Page	30
Storing Text on a Button	31
Rearranging Buttons using Drag and Drop	31
Creating a Volume Control Button	31
Creating a Battery Status Button	32
Copying and Pasting a Button Style	32
Copying and Pasting an Entire Button	33
Hiding a Button	33
Hiding or Showing Multiple Buttons at Once	34
Changing Button Timing	34
Working with Pages.....	35
Renaming a Page	35
Duplicating a Page	35
Loading Another Page.....	35
Creating a New Page.....	36
Choosing a New Keyboard.....	36
Copying a Page from a Different Vocabulary File	37
Adding a Gesture to a Page	38
Creating a Template Page.....	38
Using a Template to Create a Page.....	38
Editing an Abbreviation Expansion.....	39
Editing Pronunciations	39
Adding Acapela Vocal Smileys.....	39
Adding a Sound.....	39
Adding an Exclamation	39
Finding a Vocal Smiley	40
Example: Add a Smiley Face and Laugh Sound to a Button.....	40
Creating a Visual Scene	41
Selecting a Background Image.....	41
Setting up the Page.....	41
Adding Buttons	42
Enlarging Text in the Speech Display Bar	43
Changing Text Size or Number of Lines in the SDB.....	43
Expanding the SDB by Changing Screen Orientation.....	43
Creating a Tap Action.....	43
Setting up Switch Scanning	44
Configuring Switches	45
Setting up Touchscreen Scanning.....	45
Selecting the Scan Pattern.....	45
Configuring Speech Display Bar (SDB) Scans	45
Configuring Empty Cells Scans.....	45
Setting Scan Timing.....	46



Configuring Re-scans	46
Turning on Auto Restart	46
Setting Auditory Scan.....	46
Choosing a Voice.....	46
Blocking and Restoring Access to the Editing Menus.....	47
Changing TouchChat Settings.....	48
Navigation.....	48
Page Size	48
Editing.....	48
Speech	48
Speech Display Bar.....	49
Expanded Speech Area	49
Speak.....	49
Tilt	50
Data Logging	50
Buttons	50
Updating the TouchChat Application Software.....	51
Backing up a Vocabulary File.....	51
Analyzing Language Development	52
Using the Realize Language Website.....	52
Turning on Data Logging from Your Device.....	53
Creating a Privacy Password.....	53
Manually Uploading Data for Analysis	54
Clearing Usage Data	54
Changing or Removing Your Privacy Password	55
Transferring and Copying Vocabulary Files using Chat Editor and iTunes.....	56
Transferring a Vocabulary File from Chat Editor to the Device using iTunes.....	56
Copying a Vocabulary File from the Device to Chat Editor using iTunes.....	57
Changing Device or Screen Orientation.....	58
Changing Device Orientation.....	58
Changing Screen Orientation	59
Attaching a Strap	60
Attaching or Removing a Stand.....	61
Removing the Handle	61
Replacing the Rubber Boot	62
Cleaning and Disinfecting.....	63
Routine Cleaning and Disinfecting	63
Disinfecting TouchChat Express and Accessories for Multiple Users.....	64
Troubleshooting	65
Warranty	66

Introduction

TouchChat™ Express features a reusable, protective case. The rubber boot and plastic shell provide protection while the amplifier delivers high quality audio output. The handle provides optimal portability.

Your TouchChat Express device has been set up for communication only. Your funding source requires that your device be locked when shipped. Guided Access® is turned on, other restrictions are in place, and you will not have access to apps. If you wish to enable additional features, please contact Saltillo.

Identifying TouchChat Express Features

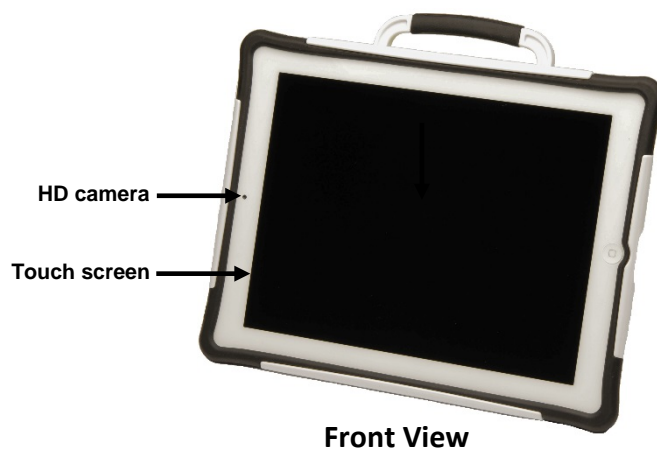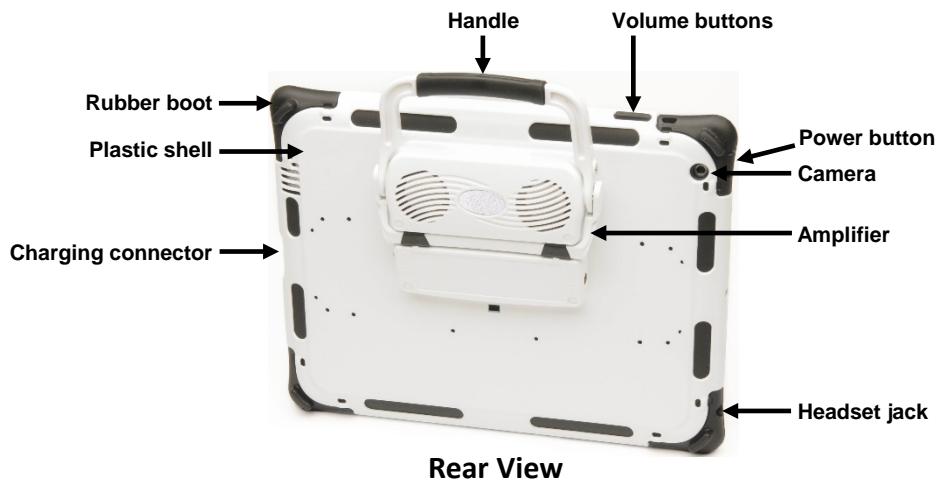Charging the Device and Amplifier

You received a battery charger with your TouchChat Express.

Connecting the Charger

One end of the charger plugs into a wall outlet—a surge protector is recommended.

The other end has two connectors, allowing you to charge the device and amplifier at the same time.

- One connector plugs into the amplifier.
- The other connector is a receptacle for the USB cable that came with your device.

Using the Backup Charger

A backup charger came with your device. Use that charger only if necessary in an emergency. The backup charger can only be used with the device. *Do not use it to charge the amplifier.*

To charge the device and amplifier:

We suggest charging both the device and the amplifier each night. The red light next to the amplifier's charger connection will be illuminated during charging and will remain illuminated when the amplifier is fully charged.

1. Tap the device's power switch to turn the display light off.
2. Plug the charger's amplifier connector into the amplifier.
3. Plug one end of the USB cable that came with your device into the device and the other end into the charger's receptacle.
4. Plug the other end of the charger into a wall outlet or surge protector.

When the device and amplifier are fully charged:

1. Unplug the connectors from the device and amplifier.
2. Unplug the charger from the wall outlet or surge protector (*optional*).
3. Tap the device's power switch to turn the display light on.

Checking Battery Status

To check the battery level or charging status at any time, create a button with the Battery Status action or find and tap a battery status button in your vocabulary.

Conserving Battery Power

To extend battery life, you can dim the screen. Choose **Settings > Display & Brightness** and drag the slider.

Disposing of the Device or the Amplifier Battery

⚠ Device Disposal: Please dispose of your device in accordance with your local, state, and/or federal electronic recycling laws.

⚠ Battery Disposal: If the battery in your amplifier needs to be replaced, dispose of the old battery in accordance with your local, state, and/or country regulations.

Turning the Amplifier On and Off

The device and amplifier communicate using a Bluetooth® connection. Typically, the amplifier should be left on. During air travel and in some medical facilities the Bluetooth option is required to be turned off. When the amplifier is off, speech is still present but comes from the device's internal speaker.

Connecting Bluetooth to the TouchChat Software


1. Open the TouchChat app.
2. Open your vocabulary file and choose **Menu > Settings**.
3. Scroll down to the ChatWrap Connect section.
4. Choose **Select ChatWrap** and choose your ChatWrap number. You will find this on the TouchChat Express serial number label.

Turning the Amplifier Off

Push and hold the button on the edge of the amplifier until both the red and blue lights come on. When the red light goes out, let go of the button.

Turning the Amplifier On

Push the button on the edge of the amplifier for a second. The blue light will flash to indicate the amplifier is on.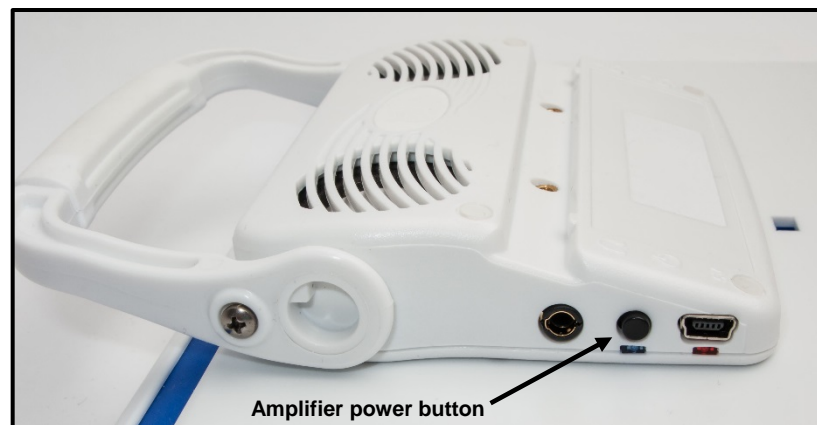

Adjusting the Volume and Muting Sounds

 Excessive volume or sound pressure level when using earphones or headphones can cause hearing loss over extended periods of time.

The volume buttons are located on the edge of the device. Pressing one button increases the volume, and pressing the other lowers the volume.

The volume buttons fit into a slot in the TouchChat Express plastic shell. The rubber boot for some models covers the two volume buttons with a single button. Press one side of the button to increase the volume and press the other side of the button to decrease the volume. By pressing a side, you are pressing the device button underneath.

To mute sound, press and hold the Volume Down button.

Using the Touch Screen

Your device comes with a capacitive touch screen. It responds best to a light touch from the pad of your finger.

Important! Using excessive force or a metallic object when pressing on the touch screen may damage the tempered glass surface.

Stylus

A stylus was provided in your TouchChat Express package. If you prefer to get your own stylus, it must be compatible with a capacitive touch screen.

Note: The stylus provided has an opening on the end for attaching to you or the device. Find a chain, string, or lanyard that matches your personal style.

 The stylus is a potential hazard for choking or for poking the eye.

Identifying Status Icons

The status bar is located at the top of the screen. These icons provide the time, battery status, and information about connectivity and operations currently running. The icons show status only. The settings cannot be changed.

Status Icon	Description
 Activity	A third-party app is using a network or generating some other activity.
 Airplane Mode	Airplane Mode is turned on. In this mode, only non-wireless features are available.
 Alarm	An alarm is set.
 Battery	Shows the battery level or charging status.
 Bluetooth	<i>Blue or White icon:</i> Bluetooth is on and paired with a device such as a headset or keyboard. <i>Gray icon:</i> Bluetooth is on and paired with a device that is out of range or turned off. <i>No icon:</i> Bluetooth is not paired with a device.
 Bluetooth battery	Shows the battery level of a supported paired Bluetooth device.
 Do Not Disturb	Do Not Disturb is turned on.
 Location Services	A location-based app is gathering and using data indicating the device's location.
 Lock	The device is locked. The display and touch capability are turned off.
 Orientation Lock	Screen orientation is locked.
 Personal Hotspot	The device is sharing its Internet connection with other iOS devices.
 Syncing	The device is currently syncing.
 VPN	The device is connected to a network using a VPN (virtual private network).
 Wi-Fi	The device has an active Wi-Fi connection. More bars indicate a stronger connection.

Syncing with iTunes

Syncing with iTunes® copies information from your computer to the device and from the device to your computer. You can sync by connecting the device to your computer.

Syncing with iTunes on your Computer

Important! Before starting this procedure, see “Connecting to a Computer for the First Time” on page 13.

Connect the device to your computer, select it in iTunes, and then set options in the different panes. If the device does not appear in iTunes, make sure you are using the latest version of iTunes, verify that the cable is correctly connected, and then try restarting your computer.


In the Summary pane, you can set iTunes to sync the device automatically when it is attached to your computer. To temporarily prevent syncing when you attach the device, hold down Command and Option (Mac®) or Shift and Control (PC) until you see the device appear in the iTunes window.

Connecting to a Computer for the First Time


When you connect your TouchChat Express-D to a computer for the first time to perform an update or backup, a message prompts you to “trust” the computer. You must tap **Trust** to proceed. On a TouchChat Express-D, this Trust message is hidden behind the application screen. Use the following procedure to access the message.

Important! You will need to repeat this procedure the first time you connect your device to a different computer.

1. On the computer, launch iTunes.
2. Connect your USB cable to the device and the computer.
3. On the TouchChat Express device, simultaneously hold down the *Home key* and the *device power button* until the device screen turns black. This will take approximately 15-30 seconds.
4. Release the Home key and power button. The device will restart. When the device has restarted, the Trust message will appear.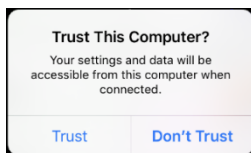

5. Tap **Trust**.
6. On the computer, a message similar to the following will appear: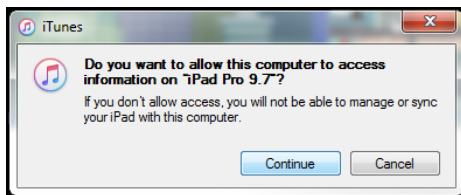

7. Click **Continue**. A progress bar will be displayed.
8. When the progress bar disappears, unplug the USB cable from the device and immediately plug it back in. This will refresh the device.
9. You are now ready to proceed with your update or backup.

Backing up and Restoring Information

The backup function does not back up TouchChat vocabulary files. For information on backing up vocabulary files, see “Backing up a Vocabulary File” on page 51.

Backing up the Device

Important! Before starting this procedure, see “Connecting to a Computer for the First Time” on page 13.

1. Open iTunes on the computer.
2. Connect your device to the computer using a USB cable.
3. In iTunes, the device will appear on the left side.
4. Right-click on the device and choose **Back Up**.
5. iTunes will start to back up the device.

Restoring the Device

Important! Before starting this procedure, see “Connecting to a Computer for the First Time” on page 13.

1. Open iTunes on the computer.
2. Connect your device to the computer using a USB cable.
3. In iTunes, the device will appear on the left side.
4. Right-click on the device and choose **Restore from Backup**.
5. iTunes will then prompt you to select the backup. Choose the one that is the most recent sync, to ensure having the current versions.
6. iTunes will start restoring the device from the backup.

Updating the Device Software

When an iOS software update is available, you can perform the update by using iTunes.

Important! Before starting the update, see “Connecting to a Computer for the First Time” on page 13.

With the latest version of iTunes installed on your computer,

1. Connect the device to the computer.
2. Open iTunes and select your device.
3. Click **Summary** and then click **Check for Update**.
4. Click **Download and Update**.
5. Choose the TouchChat image from the Home screen.
6. Press the Home button quickly three times.
7. Choose **Start** in the upper right corner to start Guided Access.

Using TouchChat


TouchChat is intended for individuals who have difficulty using their natural voice. TouchChat is designed for individuals with Autism, Down Syndrome, ALS, apraxia, stroke, or other conditions that affect a person's ability to use natural speech.

Finding the TouchChat Version


1. Tap **Vocab**.
2. Tap **Choose New Vocab**.
3. In portrait mode, the version will show at the bottom of the screen.

Working with TouchChat Menus


To access the TouchChat menus, tap **Menu**. Menu options depend on where you are in the application and whether Edit Mode is turned on. For example: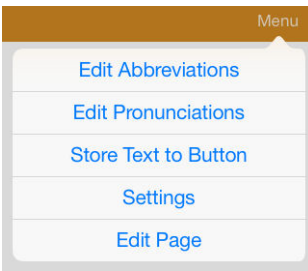

Menu options with Edit Mode turned off (gold menu bar)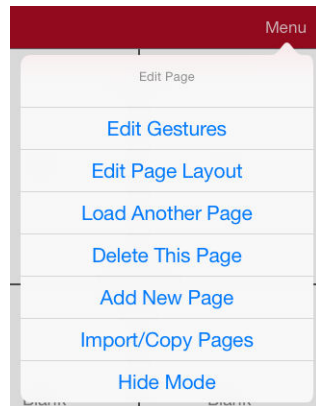

Menu options with Edit Mode turned on (red menu bar)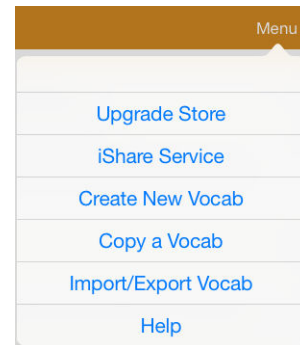

Menu options after choosing Vocab > Choose New Vocab > Menu (gold menu bar)

Working with Vocabulary Files

TouchChat provides a variety of pages that have different button layouts. Each set of pages, or vocabulary set, is called a vocabulary file. TouchChat-provided vocabulary files have locks in front of them indicating they cannot be deleted. TouchChat provides the following vocabulary files:

Language	Available Vocabulary Files
English	4-Basic, Communication Journey: Aphasia, MultiChat 15, MyCore, myQuickChat (4, 8, and 12 locations), Spelling, VocabPC, WordPower20 Simply, WordPower24, WordPower24 with Phrases, WordPower30, WordPower42, WordPower42Basic, WordPower48, WordPower60, WordPower60-Basic, WordPower80, and WordPower108
Spanish	MultiChat 15 Spanish, MultiChat 15 Spanish Bilingual, Sin sintaxis 4 x 4 English & Spanish, Sintaxis 4 x 5 English & Spanish, Teclado, VocabPC Spanish, and WordPower48 Español

English Vocabulary Files

Note: When the device language is set to English, the English vocabulary files will be listed individually when you tap Choose New Vocab and the Spanish vocabulary files will be located in a folder labeled “español”. When the language is set to Spanish, the Spanish vocabulary files will be listed individually when you tap Choose New Vocab and the English files will be located in a folder labeled “English”.

4-Basic

4-Basic offers just that, a basic vocabulary option with 4 buttons per page.

Communication Journey: Aphasia

“Communication Journey: Aphasia” is a vocabulary file containing features and vocabulary designed to support people with aphasia. It was developed by Lois Turner (SLP), Anne MacCallum (SLP) and Sarah Gauthier (SLP Assistant) in consultation with a group of speech-language pathologists who work exclusively with people with aphasia and brain injuries. All the participants work at CAYA (Communication Assistance for Youth and Adults) and the GF Strong Rehabilitation Centre in Vancouver, BC, Canada. The framework of the file was derived from the Life Participation Approach to Aphasia from the Aphasia Institute in Toronto, Canada. The pages were created to facilitate participation in daily activities and to promote social links.

Aphasia, a disorder caused by damage to the parts of the brain where language is stored, may affect the ability to comprehend language, to express oneself, to read, or to spell and write. “Communication Journey: Aphasia” can easily be adapted for use by people with different types of aphasia and a range of cognitive levels and degrees of severity.

Symbols, photographs, visual scenes, videos, or words can be used on the message, topic, and script pages. Included are pages that facilitate supported communication techniques, self-advocacy, directing care, and repair of communication breakdowns. Types of information typically included in communication books—such as pain and emotion scales, maps, and templates for personal information and life stories—are also incorporated into the vocabulary file. Specialized features include variable “rate of speech” and “pause between words”. Grid size and vocabulary are easily customized to reflect and support the life experiences of the person with aphasia.

Although this vocabulary file contains starting-point vocabulary commonly required by adults with aphasia, it *must* be customized. Each person has a unique history, and the file must reflect that person’s life, experiences, and abilities.

MultiChat 15

MultiChat 15 features a 15-button layout and is available in three versions: one for school-aged individuals with emerging language skills, one for adolescents, and one for adults. There are multiple forms of communication available in this program: sentences, phrases, individual words, recordings for story-telling, and visual scenes. Also included are interactive play and reading pages, social pages and a News-2-You starter page for the weekly newspaper’s vocabulary. There are symbols on every button with the exception of several core words.

myCore

myCore is a combination of core and phrase-based vocabulary geared toward individuals who have literacy skills.

myQuickChat

myQuickChat is an introductory communication system geared toward child and adult AAC users with complex communication needs. myQuickChat is available for both kids and adults in 4, 8, and 12 locations, each with an identical 16-location Support Master Home Page. myQuickChat was created to offer a high frequency, phrase-based communication system in an easy to use yet very engaging format. The progressive system offers a variety of topics for everyday needs and conversation and provides AAC users immediate and more successful communication exchanges within a variety of settings.

Spelling

Spelling is a QWERTY keyboard page set with four word prediction buttons and a few pre-stored phrases.

VocabPC

VocabPC was designed by Gail Van Tatenhove, PA, MS, CCC-SLP, for adults and adolescents with developmental disabilities. Vocabulary in VocabPC is arranged as carrier phrases, interactive sentences, activity vocabulary, and naming words. VocabPC uses a 12 location page layout. Both portrait and landscape versions of VocabPC are included.

WordPower20 Simply

WordPower20 Simply incorporates carrier phrases with core words for quick and easy language generation. It can be used as a phrase-based system or as a word-based system. For example, you could use the carrier phrases “I want...”, “I like...”, and “I need...” to generate sentences. Or you could use the individual pronouns “I”, “it”, and “you”, or sentences starters such as “can” and “do.” When the individual pronouns are selected, a page of frequently used verbs and helping verbs also becomes available. You can quickly finish a sentence using a phrase such as “to eat...”, “to play...”, “to watch...”, etc., or you can select “to” and have a more expanded set of verbs available.

WordPower24

WordPower24 uses high-frequency core words to facilitate quick and easy sentence generation. It also includes nouns and adjectives that are organized in logical categories. All grammatical word classes have been included. As a sentence is being built, you often find that the next word you want to say can be spoken with one or two button presses. “Grammar” keys are used to provide morphological endings to verbs, nouns, and adjectives. A spelling/word prediction page is used to spell words that are not included as separate buttons on the pages. Being a word-based vocabulary, this page set can be used by a wide age group. Nouns, adjectives, and verbs can be added and customized to meet the particular needs of the individual.

WordPower24 with Phrases

WordPower24 with Phrases is designed for individuals who are unable to use a word-based vocabulary design, but are able to generate novel thoughts using carrier phrases and semantic categories.

WordPower30

WordPower30 is very similar to the 24-location, but in the extra column on the left, you now have a clear display, delete word, period, and plural. This configuration can be set up for scanning or to be used with a keyguard.

WordPower42

WordPower42 is a word-based vocabulary that allows for easy and intuitive communication. WordPower42 is a generative language system that will be familiar to those using WordPower on other systems, and it is easy to learn for new users.

WordPower42 Basic

WordPower42 Basic is a simplified version of WordPower42. This word-based system remains rich in core vocabulary, and it may be appropriate for beginning users or those who need a more basic vocabulary design. It is designed to make communication fast and easy.

WordPower48

WordPower48 is very similar to the 42-location, but in the extra column on the left, you now have a clear display, delete word, period, and plural. This configuration can be set up for scanning or to be used with a keyguard.

WordPower60

WordPower60 consists of a large number of high-frequency words available on the main page, resulting in fast communication with reduced keystrokes. Word completion and logical next words are used, and the “grammar” function provides morphological endings to verbs, nouns, and adjectives. Spelling with word prediction along with the category-based pages contained in all other versions of the vocabulary are also available. This vocabulary is configured to work with a keyguard.

WordPower60-Basic

WordPower60-Basic was designed with early language learners in mind, providing ample picture support and novel structures to facilitate expressive language development. Core vocabulary is embedded within category and activity pages, allowing easy generation of language from the main core page as well as within specific topic-based pages. Core words remain consistent in terms of placement and access, providing a natural bridge between core and fringe words. Early literacy is enhanced by providing pages designed to teach language and core through children’s books.

WordPower80

WordPower80 is for someone who has good vision, motor skills, and literacy skills. WordPower80 vocabulary pages consist of core vocabulary, spelling, and word prediction.

WordPower108

WordPower108 consists of a large number of high-frequency words on the main page, resulting in fast communication and quick access to core words. Word completion and logical next words are used, and the “grammar” function provides morphological endings to verbs, nouns, and adjectives.

Spanish Vocabulary Files

Note: When the device language is set to Spanish, the Spanish vocabulary files will be listed individually when you tap Choose New Vocab and the English vocabulary files will be located in a folder labeled “English”. When the language is set to English, the English vocabulary files will be listed individually when you tap Choose New Vocab and the Spanish vocabulary files will be located in a folder named “español”.

MultiChat 15 Spanish

MultiChat 15 Spanish is the student version developed for devices running TouchChat. This Spanish version is very similar to the English MultiChat 15 Student file. However, due to the differences in the two languages, some changes have been made to accurately represent each language.

MultiChat 15 Spanish Bilingual

This vocabulary file is exactly the same as the MultiChat 15 Spanish file except that the labels on the buttons show both English and Spanish words or phrases. Please note that when the device speaks the words or phrases, they are only spoken in Spanish.

Sin sintaxis 4 x 4 English & Spanish

Sin sintaxis 4 x 4 English & Spanish consists of categories of words with a symbol on each button. This file is intended for children who do not have the ability to construct sentences yet.

Sintaxis 4 x 5 English & Spanish

Sintaxis 4 x 5 English & Spanish is intended for children who are beginning to create simple sentences. Each page contains 20 buttons, and each button contains a word or phrase with a picture. Words are organized into categories with some core words for creating beginning sentences.

Teclado

Teclado is a Spanish keyboard.

VocabPC Spanish

VocabPC Spanish is a direct translation of the English version. VocabPC was designed by Gail Van Tatenhove, PA, MS, CCC-SLP, for adults and adolescents with developmental disabilities. Vocabulary in VocabPC is arranged as carrier phrases, interactive sentences, activity vocabulary, and naming words.

WordPower48 Español

Designed to be intuitive and easy to use, WordPower 48 Español meets the communication needs of a wide group of individuals in Spanish speaking cultures. The vocabulary includes easy to access basic communication for beginning communicators, as well as sophisticated language and grammar for individuals with advanced cognitive and linguistic abilities.


Downloading or Purchasing Additional Vocabulary Files

The vocabulary file Primary is no longer included on the device by default. Primary is available for downloading from <http://touchchatapp.com/support/articles> or http://touchchatapp.com/pagesets#pageset_primary.

Note: To learn how to get started with WordPower and how to use WordPower vocabulary files, you can download training videos from <http://touchchatapp.com/support/touchchat>.

Loading a Vocabulary File

1. Tap **Vocab**. Choose New Vocab will be displayed.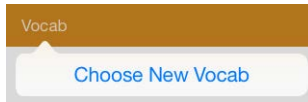

2. Tap **Choose New Vocab**. A page will display a list of the available vocabulary files. Scroll down to see them all.
3. Select the vocabulary file you want to load.
4. Once loaded, explore the pages and button organization by choosing the buttons. Buttons with arrows will navigate to new pages.

Note: After choosing an appropriate vocabulary file, you must make a copy of the file before you can customize it. See the next topic, “Copying a Vocabulary File”.


Copying a Vocabulary File

To customize an original vocabulary file, you must first make a copy of the original file. Once you make a copy, you can edit the copy. This keeps the original files in place in case they are needed at a later time.

1. Tap **Vocab** and tap **Choose New Vocab**.
2. Tap **Menu** and tap **Copy a Vocab**.
3. Tap the vocabulary file to copy.
4. Give the file a new title and description.
5. Tap **Save**.
6. Tap the new file name to open the file.


Creating a New Blank Vocabulary File

1. Tap **Vocab** and tap **Choose New Vocab**.
2. Tap **Menu** and tap **Create New Vocab**.
3. Give the new vocabulary file a title and description.
4. Tap **Save**.
5. Scroll to the new file you created and tap it to open it.
6. Tap **Menu** and tap **Edit Page**. A page of blank buttons opens.
7. For each blank button, select **Create New Button** or **Copy Existing Button**.
8. When you finish, tap **Done**. You have created the first page of your new vocabulary.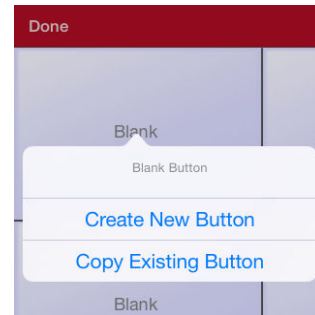


Deleting a Vocabulary File

1. From the home page of the vocabulary file, tap **Vocab**.
2. Tap **Choose New Vocab**.
3. Tap **Edit**.
4. The delete icon  will appear to the left of the padlock images. You will only be able to delete custom vocabulary files. The original vocab files are displayed with a locked padlock and cannot be deleted.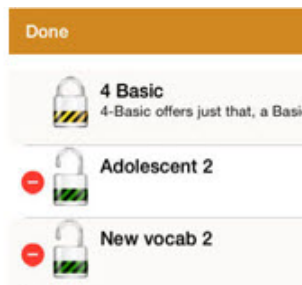

5. When you tap the delete icon , “Delete” will appear on the far right side.

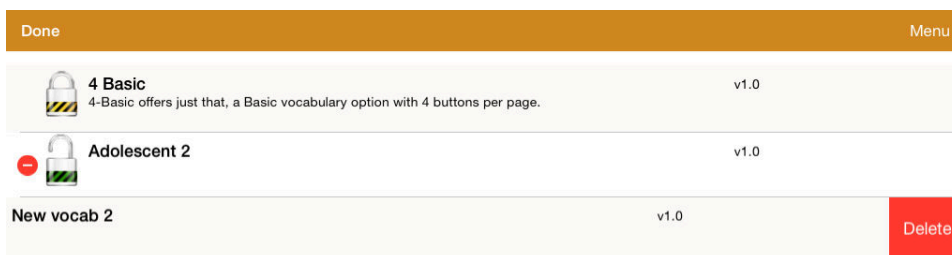6. Tap **Delete** and the vocabulary file will be deleted from TouchChat. Tap **Done**.

Using Chat Editor with TouchChat

Chat Editor is a supplemental program that runs on a Windows® desktop or laptop computer. It allows you to customize vocabulary files. Although customizing is also possible on the device itself, the Editor allows the family and/or professionals to customize the vocabulary when the device is not present.

Note: Having the custom vocabulary on a computer is always recommended as a backup to the device.

Downloading and Installing Chat Editor

Important! With software release 2.0, we replaced TouchChat Windows Editor with Chat Editor. If you have used TouchChat Windows Editor in the past, it will no longer work with 2.0 files or newer.

The installation process installs Chat Editor on your computer and creates desktop shortcuts to the “Chat Editor” application and the “Chat Editor Import” folder.

Hint: To see a video that explains how to download and install Chat Editor, go to

<http://touchchatapp.com/support/videos/touchchat-chat-editor-installation>

To download and install Chat Editor:

1. Go to <http://touchchatapp.com/support/software-updates>
2. Under “Chat Editor for Windows”, select the Download version link.
3. Save the downloaded Chat Editor_x.x.x_Setup.exe file to your desktop.
4. Double-click the Chat Editor_x.x.x_Setup.exe file to start the installation process.
5. Select the language you want to use for the installation and click **OK**.
6. Click **Next**, accept the license agreement, and click **Next**.
7. Click **Install**. When the installation process is complete, click **Finish**. You should see desktop shortcuts to the “Chat Editor” application and the “Chat Editor Import” folder.

Configuring Chat Editor

1. Double-click the “Chat Editor” desktop shortcut to open Chat Editor.
2. Select **TouchChat** from the Choose Product window and click **OK**.
3. Select your language from the Choose Language window and click **OK**.
4. Select one or more symbol sets from the Choose Symbol Sets window.
Important! This symbol set must match what is on the device you are supporting.
5. Click **OK**.
6. Type a descriptive name for the configuration in the Enter Name window and click **OK**.
7. Chat Editor will open.

Note: You may need to set up multiple configurations if you support more than one client. For example: Tommy with only SymbolStix™ images and Sarah with both SymbolStix and PCS™ symbols.

Enabling Speech in Chat Editor

Important! Before starting this procedure, see “Connecting to a Computer for the First Time” on page 13.

Note: When you install Chat Editor on a computer that had Chat Editor installed previously, speech will be enabled.

When you install Chat Editor on a computer that did not have Chat Editor installed previously, the following message will be displayed:

Speech is currently disabled.
Please connect a TouchChat device to this computer to enable speech.

Choose **OK** to close the message.

To enable speech in Chat Editor, simply connect a USB cable between your computer and your device. Test the Editor to verify that speech is now working and then disconnect the cable. Speech will now be enabled whenever you use the Editor; you will not need to re-connect the cable.

Opening Chat Editor

To open the Editor for normal use after you have set up a configuration, choose the Chat Editor shortcut from your computer’s desktop. The Editor will open on your computer.

Using Chat Editor to Create Instructional Materials

Capture sequences of buttons from screens in Chat Editor to paste into instructional materials you create such as Word documents, PowerPoint presentations, flash cards, etc.

1. In Chat Editor, choose **Capture** in the top section of the Editor window. The Button Capture window will open.
2. With **Capture Selections** selected, start selecting the buttons you want to appear in your document. Each button you select will be captured in the Button Capture window.

3. When you finish capturing a sequence of buttons, choose **Copy to Clipboard**. The sequence will be cleared from the Button Capture window.
4. In your document, insert the cursor and select **Paste**. The sequence will be pasted into the document.
5. Capture the next sequence.
6. When you finish capturing sequences, select **Exit** to close the Button Capture window.

Notes on Capturing Button Sequences

If you need to try several sequences to find the one you want, turn off Capture Selections. Once you decide what you want to capture, turn Capture Selections back on.

If you make a mistake, select **Clear** and start capturing the sequence again. Keep in mind that Clear removes **all** captured content from the Button Capture window.

Turning Edit Mode On

You can only edit a copy of a vocabulary file, not the original. To make a copy, see “Copying a Vocabulary File” on page 21.

1. Go to the page where the button will be created.
2. Tap **Menu**.
3. Tap **Edit Page**.
4. The page is now in Edit Mode.

Working with Buttons

What a button speaks, how it appears, what icon is shown, and what action the button will perform can be changed on any button. All button modifications are performed in Edit Mode.

1. Tap the button you want to modify.
2. Tap **Edit This Button**. A window will appear that will allow you to modify the button.

Modifying Text

Within the Edit window:

1. Tap in the Button Label area to change the label that will be displayed on the button.
2. Tap in the Button Message area to change the message that will be spoken.

Inserting Special Characters

Sometimes you may need to insert a special character in a word that you type, such as a letter with an accent; for example, **ó**.


On the keyboard, press and hold the applicable character key. A small menu of special characters available for that character will open above the key. Select the special character you want.

For example, to insert the special character **ó**, you would hold down the “**o**” key on the keyboard and select **ó** from the small menu.

Adding a Symbol from the SymbolStix Library


1. While in the Edit Button window, tap **Select Image**.
2. Tap the magnifying glass icon at the top right.
3. Type the name of the icon.
4. Tap **Search** on the keyboard.
5. Select the image you want.

Note: While editing a symbol, you can also choose to browse and find your icon by choosing **Select Image**, then tap a category.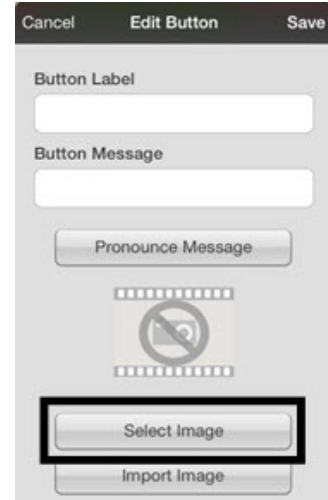

Adding a Picture from the Image Library

1. Tap **Menu** and tap **Edit Page**.
2. Tap the button to which you want to add the picture.
3. Tap **Edit This Button**.
4. Tap **Import Image**. A small menu opens above Import Image.
5. Tap **Image Library**.
6. Tap **Camera Roll** or another available library.
7. Find the picture want and tap it.
8. Tap **Use** and give the button an appropriate label.
9. Tap **Save** and tap **Done**.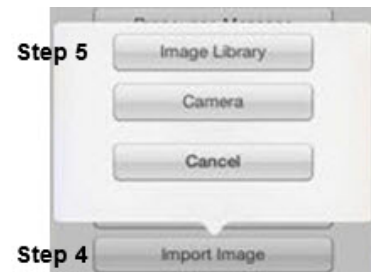

Using the Camera to Take a Picture for Your Button


1. While in the Edit Button window, tap **Import Image**.
2. Tap **Camera**.
3. Take your picture by pressing the camera icon.
4. Tap **Use**.
5. Tap **Save** and **Done**.

Creating a Camera Button

You can create a button that allows you to take a picture that will be imported to the button automatically every time you tap the button. Each time you press the button, a new image from the camera will replace the existing image on the button. If the old image on the button was imported and not used elsewhere, it will be deleted. An image will appear on the button until you press the button again.


To set up the button action:

1. Go to the page with the button location you want.
2. Tap **Menu > Edit Page**.
3. Press and hold the button you want to use.
4. Tap **Edit This Button**.
5. Add a label and message.
6. Scroll down to “Button Actions” and tap **Add** to open the list of actions.
7. Scroll down the list and tap **Take a Photo**.
8. The Take a Photo action will be added to the button’s actions.
9. Tap **Save** and then tap **Done**.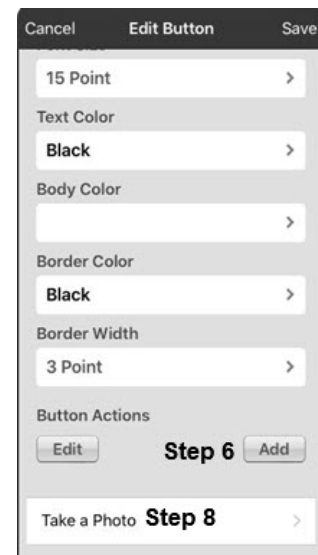

To take a photo for your button:

1. Tap the new button to activate the camera.
2. Take a picture of the object or person.
3. Tap **Use Photo**.
4. The photo will appear as the image on the button.

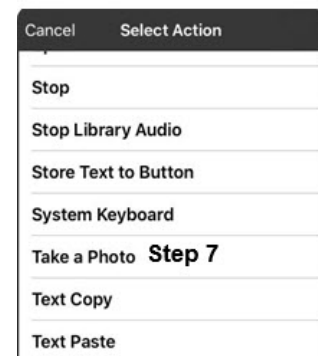To delete photos:

Important! Before starting this procedure, see “Connecting to a Computer for the First Time” on page 13.

To delete photos, you must connect your device to your computer and remove the photos using Explorer from the computer.

Resizing a Button

1. Tap **Menu** and tap **Edit Page**.
2. Tap the button you want to edit and tap **Change Button Size**.
3. Select the appropriate number of rows and columns to cover the desired area of the message.
4. Tap **Save**.

Changing a Button Style


1. While still in the Edit Button window, scroll down to the area you want to edit.
2. Choose to edit the font, size, color, and border and make those changes.
3. Tap **Save**.

Changing a Button Action

1. While still in the Edit Button window, scroll down to the Button Actions section.
2. Tap **Add**.
3. Tap the appropriate action (scroll to see the full list of actions). You can tap an existing action to change it to a different action.

Removing or Changing the Order of an Action

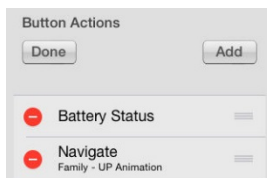1. While in the Button Actions area, tap **Edit**. The delete icon  will appear in front of each action linked to the button.

2. To remove an action, tap the delete icon  and then tap **Delete**. To rearrange the order of the actions, touch and hold the gray bars  to the right of an action and drag the action into the correct position.
3. Tap **Done**.
4. Tap **Save** and **Done**.

Creating a Button to Link to a Page

When a button action is set to Navigate or Visit you can determine what page that button will take you to. Navigate will take you to a New Page and leave you there requiring an additional button to be hit to return to your previous page or to the home page. Visit will take you there for one selection and immediately return to the previous page.

The following linking button actions are available.

Action	Description
Navigate	Links to a new page and requires another button press to move away from this page
Visit	Links to a page for a single hit and then automatically goes back to the previous page
Navigate Back	Returns to the previous page
Navigate to Home	Links to the Home page of the vocabulary file


To create a linking button:

1. Turn Edit Mode on.
2. Tap the button to be linked to the next page.
3. Tap **Edit This Button**.
4. Enter the label and message. (Entering a symbol is optional.)
5. Under the Button Actions, tap **Add**.
6. Select a navigation action for the button from the drop-down list (Navigate, Visit, Navigate Back, Navigate to Home). Navigate and Visit offer a listing of pages to link to. If you choose Navigate Back or Navigate to Home, skip to step 8.
7. Choose the page to navigate to or visit from the listing. If the page does not already exist, choose + and create a new page to link the button to.
8. Tap an animation (how the page will appear as it navigates), or tap **No Animation**.
9. Tap **Save**.

Note: There may have been a previous action on this button. To remove that action, tap **Edit**, tap the delete icon , tap **Delete**, and tap **Done**.

Storing Text on a Button

1. Go to the page that includes the button on which you want to store text.
2. Tap **Menu** and then tap **Edit Page**.
3. Tap the button on which you want to store text.
4. Tap **Edit This Button**.
5. Name the button, and optionally create a message and modify the style.
6. Scroll down to **Button Actions** and tap **Add**.
7. Scroll down and tap **Store Text to Button**. The action will be displayed in the Button Actions section.
8. Tap **Save** and then tap **Done**.

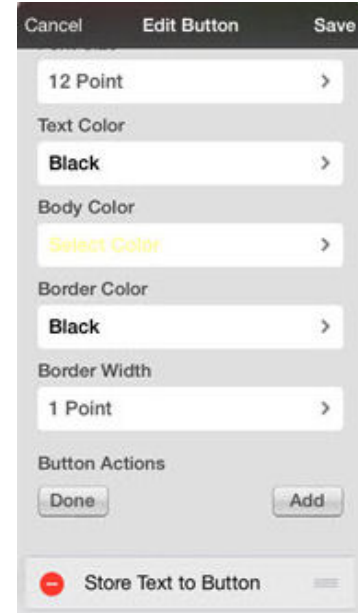Note: Once a sentence or phrase has been constructed, it can now be saved on a button by navigating to the location of the Store Text button that you created. Tap the button, tap **Continue**, tap the button in which you want to store the text, and then tap **Okay**. The text will appear in the button you selected, and each time you tap that button it will speak that saved text.

Rearranging Buttons using Drag and Drop

1. While editing a page, touch a button and drag it to another location.
2. Upon release, the button will move to the new location. If there is already a button in the new location, it will move to the old position.

Creating a Volume Control Button

Excessive volume or sound pressure level when using earphones or headphones can cause hearing loss over extended periods of time.

You can add volume controls to buttons within vocabulary pages.

1. In TouchChat, tap **Menu > Edit Page**.
2. Go to the page on which you want to create the button.
3. Press and hold on the button you want to use. A list of button options appears.
4. Choose **Edit This Button**.
5. Add a button label and icon.
6. Under Button Actions, choose **Add**. A list of actions appears.
7. Choose either **Volume Up** or **Volume Down**.
8. Choose **Save**.

Creating a Battery Status Button

You can create a button that displays the battery status of both the device and the amplifier in the Speech Display Bar (SDB).

1. In TouchChat, tap **Menu > Edit Page**.
2. Go to the page on which you want to create the button.
3. Press and hold on the button you want to use. A list of button options appears.
4. Choose **Edit This Button**.
5. Add a button label and icon.
6. Under Button Actions, choose **Add**. A list of actions appears.
7. Choose **Battery Status**.
8. Choose **Save**.

Note: To view the status of the amplifier battery, you must have ChatWrap Connect installed. If ChatWrap Connect is not installed, refer to page 2 of *Installing and Setting up TouchChat*, which was included in your TouchChat Express package.

Copying and Pasting a Button Style

“Button Style” refers to the background color, the font size, and other style options that are associated with a button. There are times when it might be faster to copy and paste a button’s style from an existing button rather than editing each of these items on a new button.

1. Tap **Menu** and tap **Edit Page**.
2. Tap the button whose style you want to copy.
3. Tap **Copy Button and Style**.
4. Tap the button you wish to paste the style to. You are presented with two options:
 - Select **Paste Style Only** to change only the style of the button.
 - Select **Paste Button and Style** to paste the contents of the button (label, image, etc.) and the style to the new location.
5. Select **Paste Style Only**.
6. Tap **Done**.


Copying and Pasting an Entire Button

1. Tap **Menu** and tap **Edit Page**.
2. Tap the button you would like to copy.
3. Tap **Copy Button and Style** to copy the button.
4. Tap **Done**.
5. Go to the page where you wish to add the copied button.
6. Tap **Menu** and tap **Edit Page**.
7. Tap the location you wish to copy to.
8. Tap **Paste Button and Style**. The button you copied will appear in that location.
9. Tap **Done**.

Hiding a Button

There are times that presenting all buttons at once might be intimidating, or in order to complete a task a button might need to be hidden.

1. Tap **Menu** and tap **Edit Page**.
2. Tap the button you want to hide and tap **Edit This Button**.
3. Turn **Visible** off.
4. Tap **Save** and **Done**.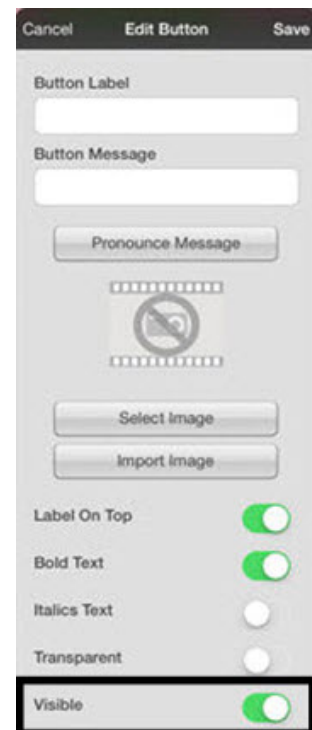

Hiding or Showing Multiple Buttons at Once

1. Tap **Menu** and tap **Edit Page**.
2. Tap **Menu** and tap **Hide Mode**.
3. Tap the buttons you want to hide or show.
4. Tap **Menu** and tap **Normal Mode**.

Changing Button Timing

Tap **Menu** and tap **Settings**. Scroll down to the “Buttons” section.

Activate on Release


This option offers to have buttons activated when released rather than upon touch. This allows the device user to slide his or her finger across the screen before finally releasing on the button that he or she wants to select. Turn the toggle switch to the **On** position.

Dwell Time

The dwell time setting controls the delay before a button press is recognized. When the user touches a button, a slight dimming and a white square box identifies the chosen button. As long as the user continues to keep his or her finger inside the button area the dwell timer counts down. Once the time is reached, the button press is accepted. To set a dwell time, tap **Dwell Time** and select a value from 0.1 Seconds to 5.0 Seconds.

Release Time

Release time is intended to help prevent accidental double button activations. When the time is set, all buttons are disabled for the specified period of time after each button press event. To set a release time, tap **Release Time** and select a value from 0.1 Seconds to 5.0 Seconds.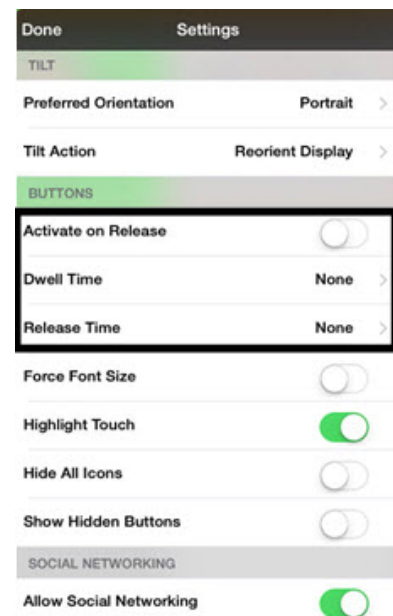

Working with Pages


TouchChat allows you to rename, duplicate, load, and create pages, choose a different keyboard, copy a page from a different vocabulary, and add gestures to pages.

Renaming a Page

1. Tap **Menu** and tap **Edit Page**.
2. Tap **Menu** and tap **Edit Page Layout**.
3. In the area under Page Name, tap, then backspace to erase the name, and then type the new name.
4. Tap **Save**.

Duplicating a Page


1. Tap **Menu** and tap **Edit Page**.
2. Tap **Menu** and tap **Import/Copy Pages**.
3. Tap **Duplicate This Page**. A duplicate of this page will now appear in the list of pages within this vocabulary set.
4. Tap **Done**.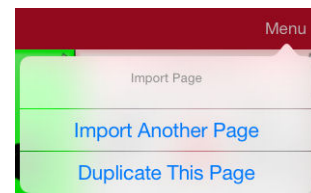

Loading Another Page


1. Tap **Menu** and tap **Edit Page**.
2. Tap **Menu** and tap **Load Another Page**.
3. Scroll up to find the page you want to load.
4. Tap **Done**.

Note: This page is now available to copy buttons or make additions or changes to your vocabulary set.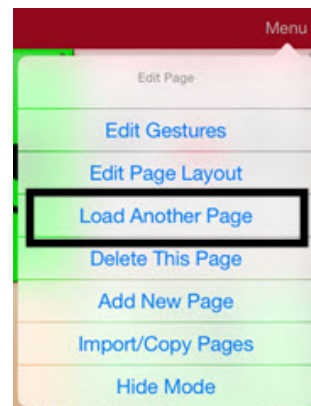

Creating a New Page

1. Tap **Menu** and tap **Edit Page**.
2. Tap **Menu** and tap **Add New Page**.
3. Give the page a name.
4. Select how many rows and columns you would like.
5. Choose a background color.
6. Select an image (*optional*).
7. Tap **Save**. A new page will appear with blank buttons.
8. Start creating buttons on this page.
9. Tap **Done** when you are finished.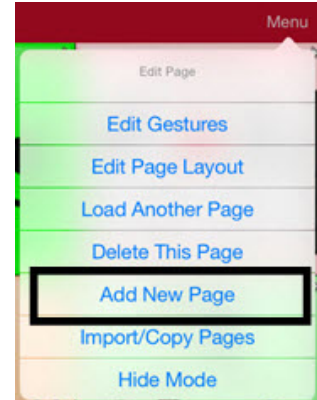


Choosing a New Keyboard

1. Tap **Menu** and tap **Edit Page**.
2. Tap **Menu**.
3. Tap **Import/Copy Pages**.
4. Tap **Import Another Page**.
5. Tap the vocabulary file that contains the keyboard you want and tap the appropriate page to import.
Note: WordPower files have alternative pages (for example, QWERTY vs. ABC) within the same file. Simply edit the linking button to navigate to the new page **or** follow steps 6, 7, and 8.
6. Tap **Import**.
7. Edit the original keyboard button and have it navigate to the new keyboard page.
8. Edit the linking button to navigate to the new page.

Copying a Page from a Different Vocabulary File

There are times that borrowing previously created pages could be a time saver. TouchChat allows you to import multiple pages at one time.

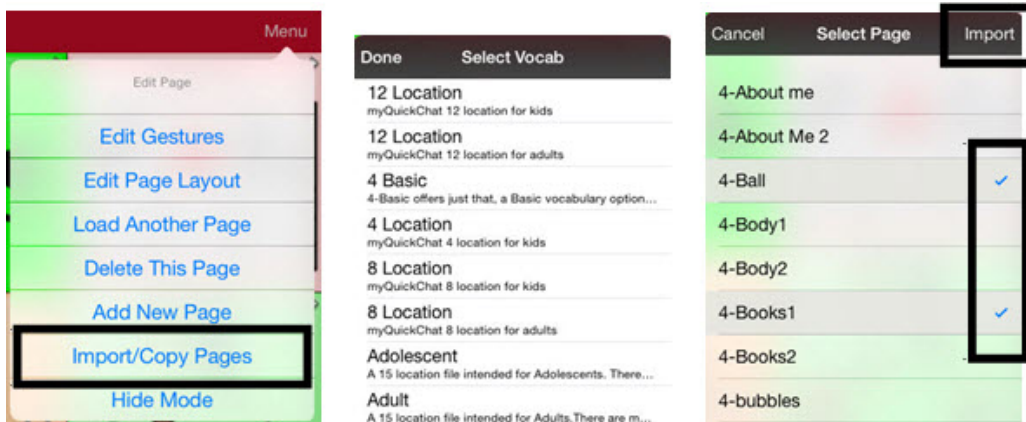1. Have your vocabulary file open.
2. Tap **Menu** and tap **Edit Page**.
3. Tap **Menu** and tap **Import/Copy Pages**.
4. Tap **Import Another Page**.
5. Tap the vocabulary file that contains the page you want and tap the appropriate pages to import. A checkmark will appear following each page you tap.


6. Tap **Import** and tap **Done**.
7. While still in Edit Mode, tap the button that you want to navigate to the imported pages.
8. Tap **Edit This Button**.
9. Give the button a label and, if needed, an icon and make any style changes you want.
10. Scroll down to Button Actions, tap **Add**, and then tap **Navigate**.
11. Tap the page you imported.
12. Tap an animation or **No Animation**.
13. Tap **Save** and tap **Done**.

Adding a Gesture to a Page

1. Tap **Menu** and tap **Edit Page**.
2. Tap **Menu** and tap **Edit Gestures**.
3. Tap **+**.
4. Add a label and message.
5. Tap the gesture type to tap the type you want. An arrow will appear following the type you selected
6. Tap a gesture action or tap **Add** to add an action.
7. To add gestures to an entire vocabulary file at one time, tap **Apply to All Pages**.

Note: To delete a gesture action, tap **Edit** and tap the action you want to delete. The delete icon  will be displayed in front of the action. Tap the delete icon  to delete the action.

8. Tap **Save** and tap **Done**.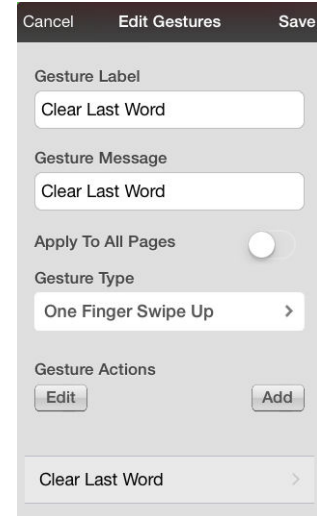

Creating a Template Page

You can create a template page in Chat Editor. This feature is handy when building vocabulary files with the same configurations of buttons to be displayed on each page. Create a page with the buttons that will be consistent on each page. Set this page as a template to be used when creating new pages.

1. Turn Edit Mode on.
2. Create a page to use as the template.
3. Click **Pages**.
4. Choose the page you would like to duplicate.
5. Choose **Save as Template**.

Using a Template to Create a Page

You can create a page from a template in Chat Editor.

1. Turn Edit Mode on.
2. Select **Pages > Templates > New Page from Template**.
3. Select a template category.
4. Select a specific template. When you select the template, a preview will be displayed.
5. Select **Choose** and type a descriptive name for the new page.
6. Select **Save**.

Editing an Abbreviation Expansion

1. Tap **Menu** and tap **Edit Abbreviations**.
2. Tap **+**.
3. In the space under Abbreviation, type the abbreviated form.
4. In the space under Expands to, type the full word, phrase, or sentence that it will expand to.
5. Tap **Save** and tap **Done**.

Note: When typing the abbreviation, type a period after the abbreviation to activate the action.

Editing Pronunciations

1. Tap **Menu** and tap **Edit Pronunciations**.
2. Tap **+**.
3. In the space under Word, type the correct spelling.
4. In the space under Pronounce As, type word phonetically or the way it should be pronounced. Tap **Pronounce Word** to determine if it is pronounced correctly. If not, change the spelling until the word is pronounced correctly.
5. Tap **Save** and tap **Done**.

Note: If the word already exists in the Pronunciation list, scroll up to find the word and then tap the word to edit to the correct pronunciation.

Adding Acapela Vocal Smileys

Acapela-Group has created the capability of using Voice Sounds and Exclamations.

Adding a Sound

Sounds are produced by the speaker's voice for laughing, breathing, sneezing, coughing, or whatever sounds our voices can produce to mimic sounds we make in our daily lives. Sounds are always between two pound signs; for example: **#LAUGH02#** in capital letters and sometimes followed by numbers if there are more than one of the same kind. The children's voices have more sounds than adult voices.

Adding an Exclamation

Exclamations include a variety of letters and symbols. Type exactly what you see.

Finding a Vocal Smiley

To find the list of Acapela Vocal Smiley commands:

1. Go to <http://www.acapela-group.com/doc/Vocal%20smileys/excla.html>
2. Click the link to the appropriate language.

Important! The language you select must match the language of the voice you are using.


3. Select the specific Vocal Smiley you want.

Example: Add a Smiley Face and Laugh Sound to a Button

A button can be set up with the command in the message. However, if you use the “speech message” action, the command appears in the Speech Display Bar (SDB), which isn't ideal. A better way may be to set up a button with an emoticon symbol in the message.

For example, you could use a smiley face symbol. Note that you may need to go to **Settings > General > Keyboards** and add the Emoji keyboard so that you can type emoticon symbols. Once the button is set up with the emoticon and speech message, go to Pronunciations and add a pronunciation for the smiley face emoticon. Enter a command such as **#LAUGH02#** for the pronunciation.

The Pronunciation Editor can be found in **Menu > Edit Pronunciations > +**. Add the same Emoticon used on the button to the Word box and add the command—for example, **#LAUGH02#**—to the Pronounce As box.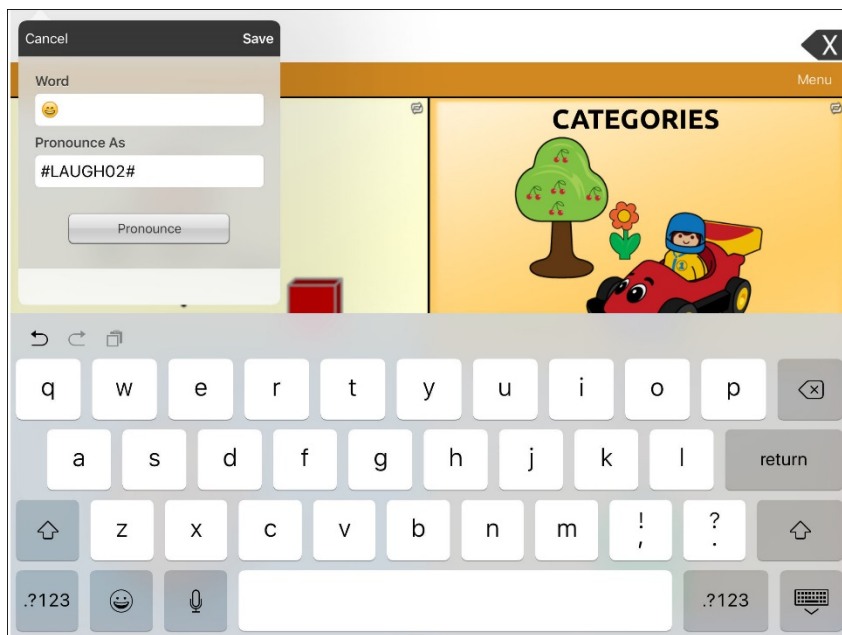

Now when you touch the button, the emoticon will appear in the SDB and the laugh sound will be heard.

Creating a Visual Scene

To create a visual scene, select a background image, set up the page, and add buttons.

Selecting a Background Image

Start by having the picture you want to use for the background image on the device. You can take a picture using your device if it has a built-in camera. Otherwise, transfer the picture to the device from a computer.

1. Tap **Menu** and tap **Edit Page**.
2. Tap the location to add a linking button (the button you wish to link to your visual scene) and tap **Edit This Button**.
3. Add a label (and optional symbol), scroll down to the Button Actions section, and replace the existing action with **Navigate**.
4. Tap the **+** in the upper right corner to create your new visual scene page.
5. Name the page.
6. Adjust the rows and columns on the page. Choosing as many rows and columns as possible will make it easier to create buttons that fit with the images on the page.
7. Scroll down to **Import Image**.
8. Tap **Import Image** and take a picture or find the image you want. A thumbnail view of the image will appear above Import Image.
9. Tap **Save**.


Setting up the Page

1. Choose the new page from the listing and select if you want animation.
2. Tap **Save** to create the page.
3. Tap **Done**.
4. Tap the button to navigate to the new page.
5. Tap **Menu** and tap **Edit Page**.

Adding Buttons

Now with the background picture in place, using the blank buttons, buttons can be created that speak or move to other pages. In our example, create a button that speaks “Horse” whenever tapping on the horse in the picture (assuming we used a picture with a horse in it).

1. Tap the upper most left corner location that covers the ear.
2. Tap **Create New Button**.
3. Type **Horse** into the button message box.
4. Set the Transparent option to the **On** position. That way the scene behind it comes through.
5. Tap **Save**.
6. Tap the newly created button and choose **Change Button Size**. From here, you can adjust the button height and width so that it fits the entire image.
7. Tap **Save**.
8. Create the remaining buttons and re-size them to go along with the scene. All the buttons can have their own size and shape.
9. When you finish, tap **Done**.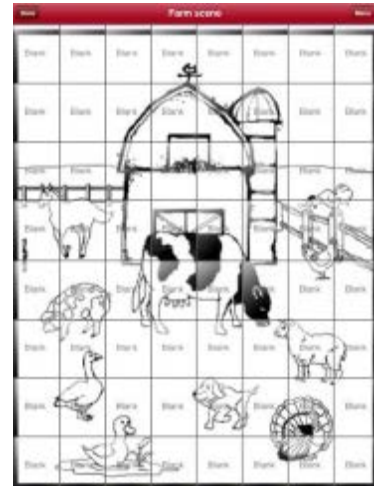

Enlarging Text in the Speech Display Bar

There are a number of ways to enlarge text in the Speech Display Bar (SDB).

Changing Text Size or Number of Lines in the SDB

1. In TouchChat, tap **Menu** and tap **Settings**.
2. Scroll to the Speech Display bar section.
3. Under Font Size, choose a larger font. Or, under Number of Lines, select the number of lines you want the SDB to display.


Expanding the SDB by Changing Screen Orientation

You can set TouchChat to expand the Speech Display Bar (SDB) when the device is rotated between portrait and landscape.

Expand and Speak: When you rotate the device, text in the Speech Display Bar (SDB) will expand to fill the screen and the device will speak it.

Expand Only: When you rotate the device, text in the Speech Display Bar (SDB) will expand to fill the screen, but the device will not speak it.

1. In TouchChat, tap **Menu** and tap **Settings**.
2. Scroll down to the Tilt section.
3. Tap **Tilt Action** and tap either **Expand and Speak** or **Expand Only**.
4. Tap **Done**.

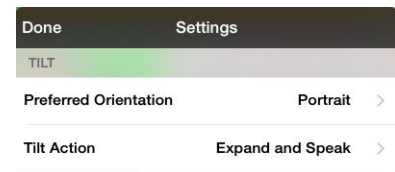Creating a Tap Action

Create a tap action to enlarge the text in the SDB.

1. In TouchChat, tap **Menu** and tap **Settings**.
2. Scroll down to the Speech Display Bar section.
3. Under Tap Action, tap **Speak and Enlarge Text**.

Setting up Switch Scanning

As an alternative to directly selecting each button on the screen, the application offers single switch and two-switch scanning. To access the options for customizing a scanning technique, choose **Menu > Settings > Scanning**.

Setting	Description
Scanning Enabled	Turns Scanning on or off. You must turn scanning on to access the other settings.
Auto Scan	You can choose to have scanning activate automatically, or you can choose to either hold the switch or step to each location. With Auto Scan off, a switch activation is required to move the scan through each button, one at a time.
Auto Restart	Allows you to restart a scan cycle automatically after an activation.
Use Screen As Switch	Allows you to use the screen as a switch
Number of Switches	Sets the number of switches being used, 1 Switch or 2 Switches
Auditory Scan	Sets audio feedback during scanning
Use Headphone For Prompt	Sets the speaker or an earphone as the audio output device
Prompt Voice	Sets the Prompt Voice
Prompt Voice Rate	Sets the Prompt Voice rate
Prompt Voice Pitch	Sets the Prompt Voice pitch
Scan Speech Display Bar	Turns on or off scanning of the Speech Display Bar
Skip Empty Cells	Turns on or off scanning of empty button areas
Scanning Type	Sets the scanning pattern to Sequential or Row/Column
Scan Speed	Sets the scanning speed
Number of Rescans	Sets the number of rescans
Number of Switches	Sets the number of switches to be used

Configuring Switches

To use an external switch, plug the switch into either switch jack A or switch jack B on the side of the device. You can choose to have scanning activate automatically, or you can choose to either hold the switch or step to each location. With Auto Scan off, a switch activation is required to move the scan through each button, one at a time.

1. To enable scanning, choose **Menu > Settings > Scanning > Scanning Enabled**.
2. Choose **Number of Switches** and select **1 Switch** or **2 Switches**.
3. To set scanning to activate without holding a switch, choose **Auto-Scan**.

Setting up Touchscreen Scanning

There are times when a user is not capable of activating specific areas on the screen. For example, maybe the user does not have the pointing skills to hit one button. Using the touchscreen to scan allows the user to tap anywhere on the screen to initiate a scan of the buttons. In this scenario, the device screen acts as a single switch.

1. Choose **Menu > Settings > Scanning > Enable Scanning**.
2. Choose **Use Screen As Switch**.
3. Configure other settings as appropriate.

Selecting the Scan Pattern

To select the scanning mode:

1. Choose **Menu > Settings > Scanning > Scanning Type**.
2. Choose Sequential or Row/Column

Using Sequential Scan Mode

A sequential scan steps through the buttons from left to right beginning in the first row. When the button you want is highlighted, activate your switch.

Using Row/Column Scan Mode

A row/column scan highlights each row of buttons beginning with the top row and move down the screen. When the row that contains the button you want is highlighted, activate your switch. The scan will then step through the buttons from left to right across the row. When the button you want is highlighted, activate your switch.

Configuring Speech Display Bar (SDB) Scans

To include the SDB in scans:

Selecting the SDB speaks the current text. Turn scanning of the Speech Display Bar on or off by choosing **Menu > Settings > Scanning > Scan Speech Display Bar**.

Configuring Empty Cells Scans

Turn scanning of empty button areas on or off by choosing **Menu > Settings > Scanning > Skip Empty Cells**. When selected, scanning will skip over an area without a button or a button that only has a speech message action but no text.

Setting Scan Timing

Scan speed is the amount of time it takes for your device to move from one button, row, or column to the next in a scan. The default scan speed is one second. At this setting, it takes one second from the time one key is highlighted until the next key is highlighted. To change the scan speed, choose **Menu > Settings > Scanning > Scan Speed**. Choose the appropriate time.

Configuring Re-scans

Select the number of times you want your device to scan the page automatically. Once you reach the set number of re-scans, the scan will stop until a switch is activated again. To set the number of re-scans, choose **Menu > Settings > Scanning > Number of Rescans**. The Number of Rescans window opens. Choose the appropriate number of rescans.

Turning on Auto Restart

This option allows you to restart a scan cycle automatically after an activation. To turn on auto restart, choose **Menu > Settings > Scanning > Auto Restart**.

Setting Auditory Scan

This option enables audio feedback during scanning. To set auditory prompts, choose **Menu > Settings > Scanning > Auditory Scan**. If you are scanning, simply activate your switch. To retrieve a message, activate the switch again after you hear the prompt.

Audio Output Device

Choose **Menu > Settings > Scanning > Use Headphone for Prompt**. You can hear prompts through the device's speaker. You can also plug stereo headphones into the headphone jack on the side of the device. This allows you, but not others, to hear the prompts. If you are using headphones, turn On User Headphone For Prompt.

Choosing a Voice

When choosing a voice to use for the auditory prompts, it can be helpful to choose a different voice than the one the system uses for speaking messages. To change the voice, choose **Menu > Settings > Scanning > Prompt Voice**. You can also adjust **Prompt Voice Rate** and **Prompt Voice Pitch** as needed.

Blocking and Restoring Access to the Editing Menus

There may be times when you need to block access to the Editing menus. To do this, the **Use Menu Passcode** setting can be turned on or off. When this setting is turned on, the Menu and Vocab items can only be accessed by entering a password.


To password-protect the Editing menus:

1. In TouchChat, tap **Menu** and tap **Settings**. The Settings menu will open.
2. Under “Editing”, turn **Use Menu Passcode** on.
3. Tap in the Passcode field to open a keyboard.
4. Enter your password and tap **Done** on the keyboard.
5. Tap **Done** on the Settings menu.

Now when the user taps either **Menu** or **Vocab**, a password prompt will be displayed.

To remove password protection from the Editing menus:

1. In TouchChat, tap **Menu** or **Vocab**. The Password prompt will be displayed.
2. Enter your password. The Settings menu will open.
3. Turn **Use Menu Passcode** off.
4. To remove the password, tap in the Passcode field to open the keyboard. Use the Keyboard backspace key  to remove the password.
5. Tap **Done** on the keyboard.
6. Tap **Done** on the Settings menu.

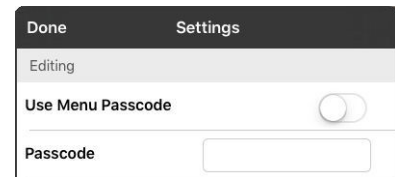Changing TouchChat Settings

TouchChat settings are located by going—within the TouchChat app—to **Menu > Settings**. From the TouchChat Settings screen, scroll to see all available settings.

Navigation

Setting	Description
Page Animation	When turned on, this setting allows you to navigate a page by scrolling up, down, right, or left.
Allow Gestures	When turned on, this setting allows you to add a gesture to a page such as swiping left or right to engage an action.
Gesture Pop Sound	When turned on, this setting provides audible feedback when a gesture is performed.
Button Click Sound	When turned on, this setting provides audible feedback when a button is selected.

Page Size

Setting	Description
Keypad Inset	When turned on, this setting adjusts the display for a keypad.

Editing

Setting	Description
Use Menu Passcode	When turned on, this setting requires a password to access the Editing menus.
Passcode	When applicable, enter the password you want to use in this field.

Speech

Setting	Description
Voice	Since there are many voice options, voices are downloadable. This will save a lot of space on the app by only downloading the voices needed. To download a voice, tap the green down arrow and select Yes . <i>Note:</i> Some voices are only available at a cost. These voices must be purchased from the Upgrade Store before they are available in the Voice listing.
Rate	Use the slide bar to modify the rate of speech.
Voice Pitch	You can make a voice sound different by modifying the pitch.
Pause Between Words	You can add a speech delay between individual words.

Speech Display Bar

Setting	Description
Hide Speech Display	When turned on, this setting hides the entire SDB.
Auto Capitalization	When turned on, this setting capitalizes the first word of a sentence automatically.
Show Icons	Choose this setting if you want to display icons with text in the SDB.
Hide Delete Button	When turned on, this setting hides the SDB's Delete button.
Font	Select one of the 7 available fonts for use for text in the SDB.
Font Size	Determines the size of the font that appears in the SDB.
Font Color	Choose one of the 12 available font colors to apply to text in the SDB.
Background Color	Choose one of the 12 available background colors to apply to the SDB.
Number of Lines	Choose how many lines of text you want to appear in the SDB.
Single Line Cursor	Choose this setting if you want a cursor to appear in the SDB.
Tap Action	Choose Speak , Speak and Enlarge Text , or No Action when the SDB is tapped.

Expanded Speech Area

Setting	Description
Font	Select one of the 7 available fonts for use for text in the expanded speech area.
Font Size	Determines the size of the font that appears in the expanded speech area.
Font Color	Choose one of the 12 available font colors to apply to text in the expanded speech area.
Background Color	Choose one of the 12 available background colors to apply to the expanded speech area.

Speak

Setting	Description
Speak Characters	When this setting is turned on, each character will be spoken as entered.
Speak Words	When this setting is turned on, each word will be spoken as entered.
Speak Sentences	When this setting is turned on, each sentence will be spoken as entered.
Auto Clear	When this setting is turned on, the SDB clears automatically after speaking.
Prevent Interruption	When turned on, this setting will prevent interruptions of speaking a sentence if you accidentally hit another button while it is speaking.

Tilt

Setting	Description
Preferred Orientation	Select either Portrait or Landscape.
Tilt Action	Select to Expand Text in the Speech Display Bar and Speak It , Expand the Text Only , Reorient the Screen , or No Action .

Data Logging

Setting	Description
Enable Logging	Turn on data logging, allowing your device to collect language usage data which you can upload to the Realize Language website for web-based analysis or save to a file to analyze manually.
Privacy Password	Set or change your data logging privacy password.
Clear Log	Clear usage data for the current person.
Realize Language Setup	Upload data collected for a person to the Realize Language website for analysis.

Buttons

Setting	Description
Activate on Release	When this setting is turned on, a button will not activate until released, rather than on touch.
Dwell Time	Controls the delay before a button press is recognized in case a user needs to slide across multiple buttons before pressing the intended button.
Release Time	All buttons are disabled for specified period after each button press event to prevent accidental double button presses.
Force Font Size	When this setting is turned off, the button label font size is automatically reduced as needed to allow the entire label to fit into the button area.
Highlight Touch	When this setting is turned on, the button that is touched is highlighted briefly—the button turns red for a moment when activated.
Hide All Icons	When turned on, this setting will hide all of the icons from the buttons and only the text will appear. Note that hiding the button icons will also force the SDB setting “Show Icons” off.
Show Hidden Buttons	When turned on, buttons that have been hidden will be displayed.

Updating the TouchChat Application Software

Important! Before starting this procedure, see “Connecting to a Computer for the First Time” on page 13.

1. Connect the device to the computer.
2. Launch iTunes on the computer.
3. Go to the Apps section of iTunes.
4. In the bottom right of the screen, locate the option “Check for Updates”.
5. Enter the Apple ID that you used to purchase TouchChat previously.
6. The update will start downloading.

You can also update TouchChat directly from the device. The App Store icon should show when updates are available to download on the device. When an update is available, the app store icon will display a number on the right hand corner of the icon.

1. Tap on the App Store icon
2. Tap on **Updates** on the bottom right of the screen.
3. The apps that require updating will show up.
4. Tap either **Update** on the application or **Update All** in the upper right hand corner.
5. Enter the Apple ID that you used to purchase TouchChat previously.
6. The update will start downloading.

Backing up a Vocabulary File

It is always a good idea to keep a copy of all custom files on a second computer for safekeeping. We suggest always using a wired connection to a PC to copy files consistently.

Important! Before starting this procedure, see “Connecting to a Computer for the First Time” on page 13.

1. Connect the device to the computer.
2. Open iTunes on the computer.
3. On the device, go to the Vocab File section and tap on **Menu** in the top right portion of the screen.
4. Tap **Import/Export Vocab**.
5. Tap **Export to iTunes**.
6. Choose the vocabulary file you want to export.
7. Name the file that you are exporting and tap **Save**.
8. In iTunes on the computer, tap on the device and go to the Apps section.
9. Scroll down until you see TouchChat in the File Sharing area.
10. Highlight the vocabulary file that you are exporting.
11. Click **Save To** on the bottom right.
12. A Windows save box will appear. Select the folder in which you want to save the file.

Analyzing Language Development

You can use the data logging capabilities of your device to collect information on a person's language development. Then you can upload that information to the Realize Language™ website.

This process allows you to monitor, measure and maximize the person's use of the device.

Using the Realize Language Website

Realize Language is a subscription-based online service that organizes and analyzes information and presents the results in easy-to-understand graphic formats that provides valuable insights into each person's language development. With this service you can

- Track the person's progress and communication development over time
- Compare different aspects of communication automatically
- Create a detailed summary of performance
- Share information with others
- Quickly create reports that anyone can understand

To learn more, go to the Realize Language website: <https://realizelanguage.com/info/>

To view or download the *Realize Language Starter's Guide*, go to <https://realizelanguage.com/info/support>

For Customer Support:


If you have a problem with the Realize Language website, email support@realizelanguage.com.

If you have a problem with your device, call Saltillo technical support at 1-800-382-8622 or email service@saltillo.com.


Turning on Data Logging from Your Device

When data logging is turned on, your device collects language usage data which you can upload to the Realize Language website for web-based analysis or save to a file to analyze manually.

1. In TouchChat, tap **Menu** and tap **Settings**. The TouchChat Settings screen will be displayed.
2. Under Data Logging, turn on **Enable Logging**.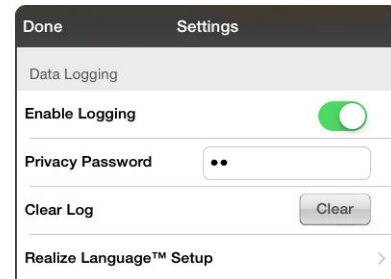

When data logging is turned on, the data logging icon appears in the Speech Display Bar.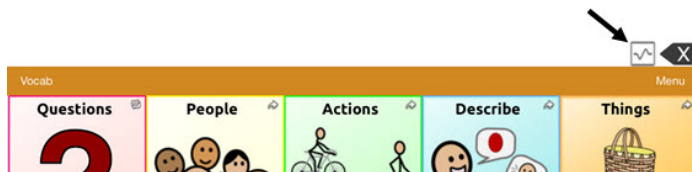


Important! If you plan to use the Realize Language website to analyze data, set up a person before you start collecting data.

Creating a Privacy Password

Data logging provides the option to create a privacy password to protect the data you collect against unauthorized access.

1. Under Data Logging, tap the password box located to the right of **Privacy Password**. A cursor will appear in the password box and keyboard will be displayed.
2. Type the privacy password you want to use and tap **Done**. The Confirm New Privacy Password prompt will be displayed.
3. Type your password again and tap **Enter**. “New Password Set” will be displayed.
4. Tap **OK** to close the message.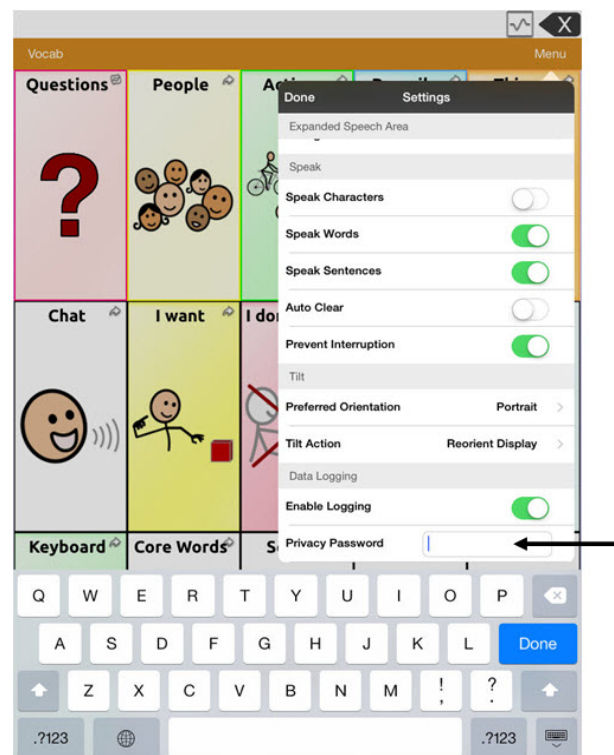

Manually Uploading Data for Analysis

Upload data collected for a person to the Realize Language website for analysis.


Important! Before starting this procedure, see “Connecting to a Computer for the First Time” on page 13.

1. Connect the device to your computer.
2. Open iTunes on the computer.
3. In iTunes on the computer, tap on the device and go to the Apps section.
4. Scroll down until you see TouchChat in the File Sharing area.
5. Highlight the **logFile.txt** file that you are uploading.
6. Click **Save To** on the bottom right. A Windows save box will open. Select the folder in which you want the file saved.
7. Log onto the Realize Language website.
8. Select **Upload Data File** and then **Select a File to Upload**.
9. Select the file and click **Upload**.

Clearing Usage Data

To clear usage data for the current person:

1. Under Data Logging, tap the **Clear** button.
2. The Privacy Password prompt and keyboard will be displayed. Type your privacy password and tap **Enter**.
3. “Do you really want to Clear Log?” will be displayed. Choose **Yes** to clear the data log.
4. “Clear Log Completed” will be displayed. Tap **Okay** to close the message.

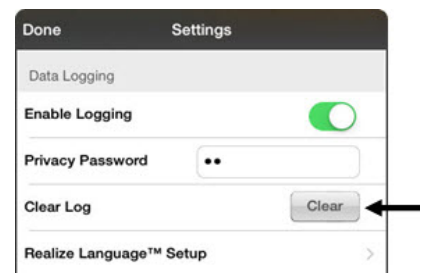
Changing or Removing Your Privacy Password

By default, each time you save data to a file, you enter a password. Entering a password helps ensure privacy—only you will be able to view the data. You can change this privacy password as often as you want, or you can eliminate the need for a privacy password.

To change your privacy password:

1. Under Data Logging, tap the password box located to the right of **Privacy Password**. A Privacy Password prompt and keyboard will be displayed.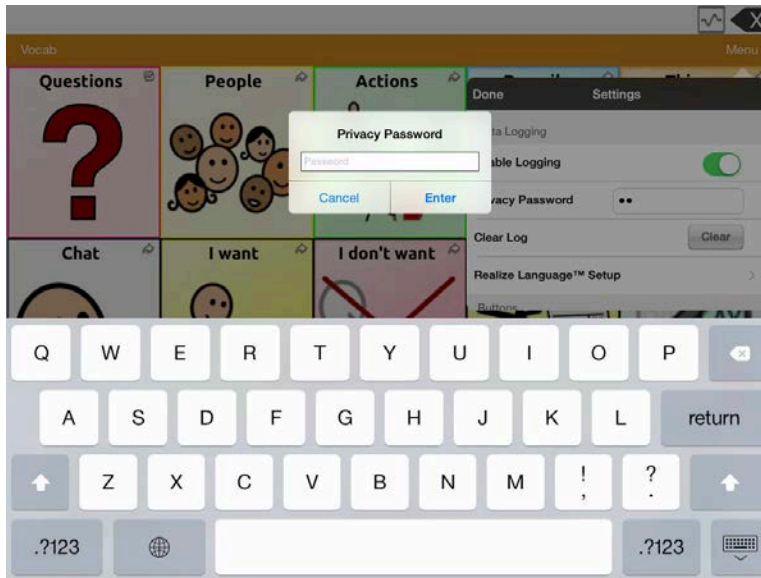

2. Type your old privacy password and tap **Enter**. A cursor will appear in the password box.
3. Use the backspace key  to clear the Privacy Password field.
4. Type your new password and tap **Done**.
5. The Confirm New Privacy Password prompt will be displayed. Type your new password again and tap **Enter**.
6. “New Password Set” will be displayed. Tap **OK** to close the message.

To remove your privacy password:

If you prefer not to enter a password each time you save data to a file, use this procedure. Keep in mind, however, that you will also lose the privacy protection a password provides.

1. Under Data Logging, tap the password box located to the right of **Privacy Password**. A Privacy Password prompt and keyboard will be displayed.
2. Type your old privacy password and tap **Enter**. A cursor will appear in the password box.
3. Use the backspace key  to clear the Privacy Password field and tap **Done**.
4. “Privacy Password Cleared” will be displayed. Tap **OK** to close the message.

Transferring and Copying Vocabulary Files using Chat Editor and iTunes

Transferring a Vocabulary File from Chat Editor to the Device using iTunes

Important! Before starting this procedure, see “Connecting to a Computer for the First Time” on page 13.

1. Open Chat Editor on your computer.
2. Choose **Library > Transfer > Vocab to File**. The Transfer box will appear.
3. Highlight the file you want to transfer and click **Transfer** in the top left.
4. Save your vocabulary file to your computer (remember where you save it).
5. Open iTunes on the computer.
6. Attach the device to the computer.
7. Tap on the device (left column) and go to the Apps section.
8. Scroll down until you see TouchChat in the File Sharing area.
9. Tap **TouchChat**. “Add File” should appear in the lower right-hand section.
10. Tap **Add File**. A Windows Save screen will appear.
11. Locate the file you want to add to your device and tap **Open**.
12. The file should appear in the TouchChat Documents section.
13. Go to the device and go to the Vocab File section.
14. Tap **Menu** in the top right and tap **Import/Export Vocab**.
15. Choose **Import from iTunes**.
16. Select the file you want to import, then tap **Save** in the top right portion of the screen. Your imported file should appear in the Vocab File list.
17. Select the file.

Copying a Vocabulary File from the Device to Chat Editor using iTunes

Important! Before starting this procedure, see “Connecting to a Computer for the First Time” on page 13.

1. Connect the device to the computer.
2. Open iTunes on the computer.
3. On the device, go to the Vocab File section and tap on *Menu* in the top right portion of the screen.
4. Tap **Import/Export Vocab**.
5. Tap **Export to iTunes**.
6. Choose the vocabulary file you want to export.
7. Name the file that you are exporting and tap **Save**.
8. In iTunes on the computer, tap on the device and go to the Apps section.
9. Scroll down until you see TouchChat in the File Sharing area.
10. Highlight the vocabulary file that you are exporting.
11. Click **Save To** on the bottom right.
12. A Windows save box will appear. Select the folder in which you want the file saved.

To place the file into Chat Editor, use the Chat Editor Import folder located on your computer’s desktop. The folder was placed there when Chat Editor was installed. If the folder is not there, you can locate the import folder depending on the version of Windows you are using at:

Windows Version	Path to Import Folder
Windows XP	My computer > C drive (local drive) > Documents and Settings > Users (*log in ID*) > Application Data > Chat Editor > Import
Windows Vista or Windows 7, 8, or 10	C: Drive > Users > *User ID* > AppData > Roaming > Chat Editor > Import

Note: The Application Data folder (for Windows XP) or the App Data Folder (for Windows Vista/7) might be hidden. To make these folders visible, go to **Control Panel** and choose **Folder Options**. Then choose the **View** tab, and under Advanced Settings > Hidden Files and Folders, select **Show Hidden Files and Folders**. Then click **OK**. The folders should now be visible. The vocabulary file should start importing automatically after a few minutes.

Changing Device or Screen Orientation

You can change the orientation of your TouchChat Express as well as the orientation of the display.

Changing Device Orientation


If you want to change the orientation of the amplifier, holes were provided in the plastic shell for arranging the amplifier in either a landscape or portrait layout. You will need a Phillips screwdriver.

First: Remove the Amplifier


Remove the shell and boot from the device, and then remove the boot from the shell. Remove the four screws from the shell that are holding the amplifier in place.

Next: Open new mounting holes in the shell

Position the shell in the orientation you want.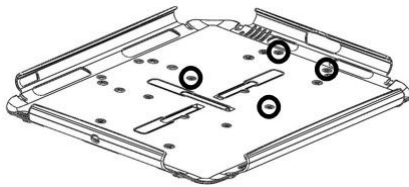

For portrait: Use the holes on one of the shorter sides

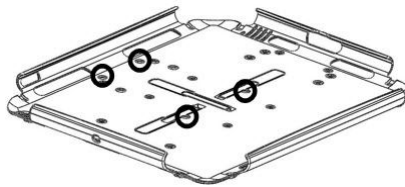For landscape: Use the holes on the longer side

Use the tip of the Phillips screwdriver to punch out the four holes circled in the appropriate illustration above.

Use extra care when punching out the holes. Do **NOT** hold the shell in your hands or against your body. Place the shell on a flat surface and hold the shell on a side as far away as possible from the holes you are going to punch. Do **NOT** place your hand under the shell.

Then: Replace the Amplifier

Fasten the amplifier to the shell in the new position. Put the boot over the device, and then insert the device/boot assembly into the shell.

Changing Screen Orientation

Important! TouchChat screen orientation settings do not affect iOS screen settings.

Changing Screen Orientation in TouchChat

1. Tap **Menu** and tap **Settings**.
2. Scroll down to the Tilt section.
3. Tap **Tilt Action** and tap one of the options:

Option	Description
Expand and Speak	When you rotate the device, text in the Speech Display Bar (SDB) will expand to fill the screen and the device will speak it.
Expand Only	When you rotate the device, text in the Speech Display Bar (SDB) will expand to fill the screen, but the device will not speak it.
Reorient Display	When you rotate the device, the screen orientation will switch between portrait and landscape.
No Action	When you rotate the device, the screen orientation will not change. Screen orientation will be locked.

4. Tap **Done**.

Locking Screen Orientation in TouchChat

1. Tap **Menu** and tap **Settings**.
2. Scroll to the Tilt section and tap **Tilt Action**.
3. Tap **No Action**.
4. Tap **Done**.

Attaching a Strap


For hands-free transport, a strap and loops for attaching the strap were provided in your TouchChat Express package.

Step 1: Decide whether you want to carry the unit in landscape or portrait orientation.


Step 2: Attach two loops to the appropriate side as shown.

1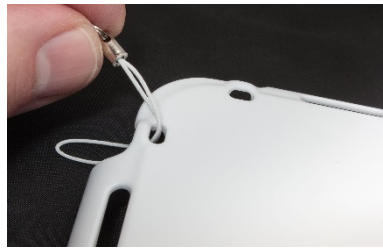

2

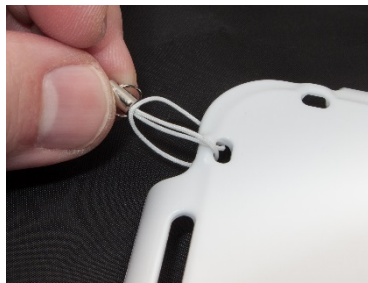3

4

Step 3: Attach the connectors on the ends of the strap to the loops.

Important! Do not attach a strap directly to the plastic casing as this may damage the device.


Attaching or Removing a Stand

Two stands—a long stand and a short stand—were provided with your TouchChat Express. The amplifier itself provides a bit of an angle for viewing the screen when sitting on a flat surface. If an additional angle is needed, attach one of the stands to the back of the amplifier.

The stands were designed to be break-away to avoid pinching. Either stand is secured to the amplifier handle with a tension fit.

To attach a stand: Spread the legs of the stand apart, position the stand over the amplifier, and insert one leg, then the other in the holes at the base of the amplifier's handle.

To remove a stand: Spread the sides of the stand and pull the legs out of the holes at the base of the handle.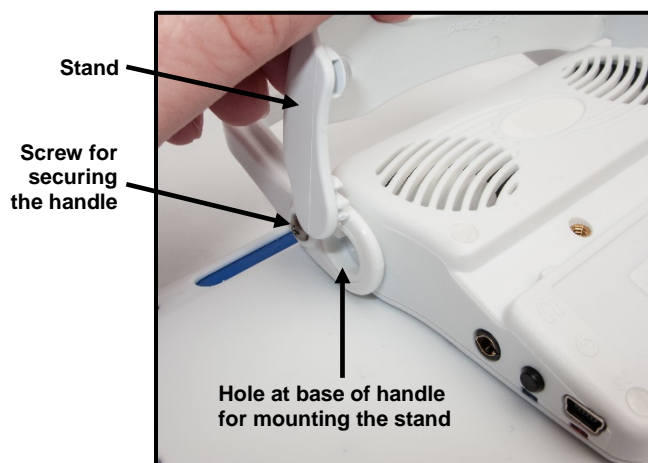

Removing the Handle

To remove the handle from the amplifier:

1. Remove the stand, if it is in place.
2. Remove the screw from each side.
3. With some tension, extend the sides out around the case.


The stand and screws from the handle are potential choking hazards.


Replacing the Rubber Boot

To replace the rubber boot, follow these instructions.

1. Remove the existing plastic shell/rubber boot assembly from the device.
2. Remove the existing boot from the shell.
3. Fit one end of the device into the replacement rubber boot. Be sure to line up the buttons and ports on the device's sides with the corresponding receptacles and openings in the boot.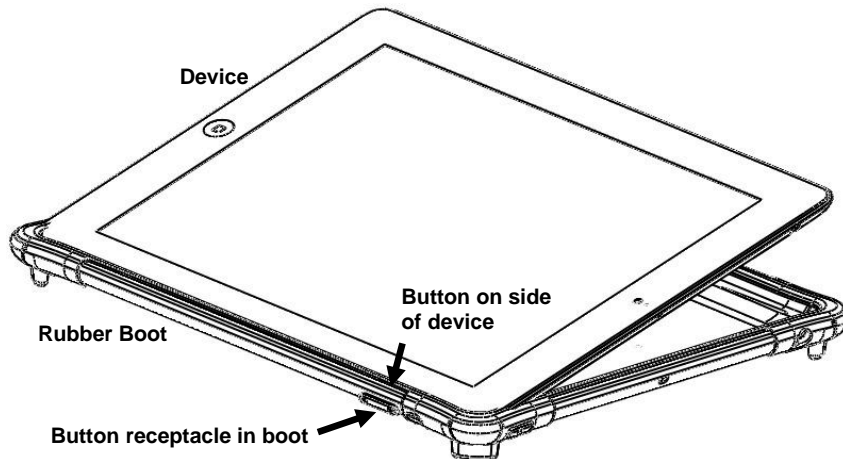

4. When you are sure that the device and boot are properly aligned, push the device down into the boot and pull the edges of the boot over the sides until the device is fully “wrapped” in the boot on all sides.
5. Fit one end of the device—wrapped in the boot—into the shell. Be sure to line up the buttons and ports on the device's sides with the corresponding openings in the shell.

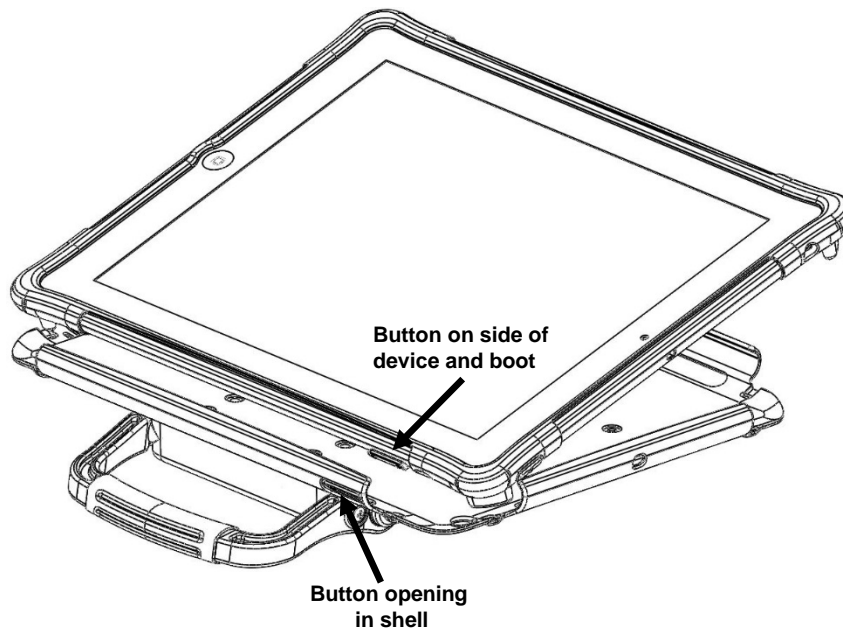6. When you are sure that the device/boot assembly is properly aligned, carefully push each side into the shell until all sides snap into place.

Cleaning and Disinfecting

If your TouchChat Express will only be used by one person, routine cleaning and disinfecting should be sufficient. If, however, more than one person will be using the unit, take additional measures to clean and disinfect the unit after it has been used by one person before allowing it to be used by another.

Routine Cleaning and Disinfecting

Important! Do not use solvents or abrasives or spray water or cleaner on the unit.

Cleaning the touch screen

The touch screen is very sensitive. Fingerprint smears, dust, grime, saliva, etc. will affect its performance. The screen also reacts to raindrops and extremes in temperature (condensation).

To dust the touch screen, use a soft, lint-free cloth. For heavier cleaning, use a slightly dampened, lint-free cloth and then dry the screen with another soft, lint-free cloth.

Cleaning the TouchChat Express components and keyguard

The unit can be disassembled and the TouchChat Express components and keyguard washed in hot, soapy water. Dry them thoroughly before reassembling the unit. The best practice for regular cleaning is to dry accessories by hand, lay them on a dry towel, and let them air dry for about 5 minutes before placing them back on the device.

Disinfecting

To disinfect everything but the display, use a solution of vinegar and water ($\frac{1}{4}$ cup vinegar to 1 cup water). Wipe the rubber boot (not the display) using a cloth dampened with this solution.

Wash the plastic shell and keyguard separately with the vinegar and water solution. Allow all equipment to air dry. **Do not use vinegar and water on the display!**

What about drooling/saliva?

Any saliva should be wiped up immediately, especially on the touch screen or around any of the connectors. Use a slightly dampened, lint-free cloth to wipe the device.

What about spills, rain, or accidental immersion?

If you spill liquid or any runny substance on your TouchChat Express; if you are caught in a downpour; or if the unit is accidentally dropped in liquid, immediately dry the components as thoroughly as possible. Try using the device. If it does not work, call the Saltillo Service Department and tell them what happened. They may ask you to send in your device for servicing.

Disinfecting TouchChat Express and Accessories for Multiple Users

Note: This section applies only when more than one person will be using the same unit. In that situation, it is vital to effectively disinfect the unit and accessories that have been used by one person before allowing them to be used by another.

First Step: Put on protective gloves

Next Step: Clean the TouchChat Express components and accessories

1. Wipe down the TouchChat Express components and accessories with disinfectant wipes (Virucidal, Bactericidal, Pseudomonacidal, Tuberculocidal, Fungicidal)—Metrex CaviWipes™. Follow the disinfectant product manufacturer's instructions for *cleaning*.
2. Take a Q-tip with cleaner applied (for example, Windex® or another commercial cleaner) and clean inside all cracks and crevices. You may want to use a toothbrush or similar brush.
3. Blow the components off with an air hose, or wipe it dry.

Important! Make sure all soil is removed from the TouchChat Express components and accessories. This is vital before proceeding to disinfecting the equipment.

Next Step: Disinfect the TouchChat Express components and accessories

1. Wipe down the TouchChat Express components and accessories with a new disinfectant wipe (Virucidal, Bactericidal, Pseudomonacidal, Tuberculocidal, Fungicidal)—Metrex CaviWipes. Follow the disinfectant product manufacturer's instructions for *disinfecting*.
2. Make sure to clean inside all cracks and corners, and wipe more than once if necessary to keep the components wet for a minimum of three (3) minutes.
3. Allow the components and accessories to dry.

Final Step: Wipe the touch screen

1. When the TouchChat Express components and accessories are dry, wipe down the viewing area (device touch screen, keyguard, etc.) with glass cleaner, so the screen does not discolor.
2. Allow the equipment to dry.

Troubleshooting

Speakers/amplifier no longer produce sound. / Sound is limited to the device's speakers only.

First try this: Make sure the amplifier has been charged. If the amplifier is charged ...

Next try this: Make sure Airplane Mode is off. Press the device's **Home** button and choose **Settings**. If Airplane Mode is on, turn it off.

Then try this: Check the blue LED on the side of the amplifier. If it is flashing, the amplifier needs to be reconnected. Choose **Settings** > **Bluetooth** and select the Bluetooth identifier from the Bluetooth "Devices" list. The Bluetooth identifier is printed on the label on the back of the amplifier.

To contact Saltillo for operational or technical support	
Phone	1-800-382-8622
Email	service@saltillo.com

Warranty

Saltillo Corporation warrants the TouchChat Express to be free from defect in material and workmanship under normal use for the device warranty period of one year from date of purchase.

Please contact the Saltillo Service department (1-800-382-8622, option 1) prior to making alterations to the TouchChat Express for mounting, transportation, etc.

All warranty service should be arranged through Saltillo Corporation prior to sending a system back. Please call the company at 1-800-382-8622, option 1, or email at service@saltillo.com to obtain a Return Authorization number prior to sending the system to Saltillo.

If replacement is necessary, the replacement TouchChat Express may be a new or re-conditioned TouchChat Express of equal value.

Note: Saltillo Corporation considers bending of the unit, submerging it into liquid of any kind, or any breaks and/or cracks in the LCD screen to be misuse. However, Saltillo will cover one screen replacement or one iPad replacement per warranty year.

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Salttillo Corporation