English/Spanish User's Guide

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TouchChat® Express 8 · 2nd Generation

TouchChat® Express 10 · 2nd Generation

Saltillo Corporation
2143 Township Road 112
Millersburg, OH 44654
www.saltillo.com

User's Guide

TouchChat® Express 8 • 2nd Generation

TouchChat® Express 10 • 2nd Generation

 Saltillo Corporation
2143 Township Road 112
Millersburg, OH 44654
www.saltillo.com

Acknowledgements

Copyright/Trademark Information

© 2015-2018 Saltillo Corporation. All rights reserved.

TouchChat and ChatWrap are trademarks of Saltillo Corporation.

Apple, the Apple logo, AirDrop, AirPlay, FaceTime, Guided Access, iBooks, iPad, iPad Air, iPod, iPod touch, iSight, iTunes, Lightning, Mac, Safari, and Siri are registered trademarks of Apple Inc.

iPad mini and Touch ID are trademarks of Apple Inc.

iCloud and iTunes Store are service marks of Apple Inc.

iOS is a trademark or registered trademark of Cisco in the U.S. and other countries and is used under license.

Microsoft and Windows are registered trademarks of Microsoft Corporation in the United States and/or other countries.

The *Bluetooth®* word mark and logos are registered trademarks owned by the Bluetooth SIG, Inc. and any use of such marks by Saltillo Corporation is under license.

Acapela is a trademark of Acapela Group.

Dropbox is a registered trademark of Dropbox, Inc.

Google Drive, Gmail, and YouTube are trademarks of Google Inc.

The Picture Communication Symbols ©1981–2017 by Tobii Dynavox. All Rights Reserved Worldwide. Used with permission. Boardmaker® is a trademark of Tobii Dynavox.

SymbolStix®, Copyright 2017, SymbolStix, LLC. All rights reserved. Used with permission.

VocalID is a trademark of VocalID, Inc.

WordPower is a trademark of Inman Innovations, Inc.

10144en Version 1.8 · 02/14/2018

Electronic Interference Information

This device complies with Part 15 of the FCC Rules. Operation is subject to two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Warning! Changes or modifications to this product which are not authorized by Saltillo Corporation could exceed FCC limits and negate your authority to use this product.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. You can determine whether this product is causing interference in your radio or television by turning this product off. If the interference stops, it was probably caused by this product or one of its accessories. You can attempt to correct the interference by using one or more of the following measures:

- Turn the television or radio antenna until the interference stops.
- Move this product to one side or the other of the television or radio.
- Move this product farther away from the television or radio.
- Plug this product into an outlet that is on a different circuit from the television or radio; that is, this product should be controlled by different circuit breakers/fuses from the television or radio.
- If necessary, contact a Saltillo Corporation service technician for assistance.

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Warnings

This device is not intended to be an emergency call device or sole communication aid.

When operating this device in a medical environment, do not use with any product that is not medically approved. Follow all rules for appropriate wireless device use.

Do not use this device close to sources of RF radiation or you may encounter interference. Move away, if possible, from the source of the interference.

Any mounts used should be fitted by a qualified person. Failure to install the mounting system according to the manufacturer's instructions may result in an injury to the user.

Analysis of positioning by a qualified person is required to prevent repetitive stress injuries to the user.

Cords and straps are potential entanglement/strangulation hazards. Please consider this prior to placing these items with device users.

Small parts could present a choking or other hazard

Do not use the device if the screen is cracked or broken.

Amplifier batteries are not replaceable by the customer.

Saltillo Corporation assumes no responsibility for any loss or claims by third parties which may arise through the use of this product.

Saltillo Corporation assumes no responsibility for any damage or loss caused by the deletion of data as a result of malfunction repairs or battery replacement. Be sure to back up all important data on other media (computer) to protect against its loss.

Implantable Medical Devices

A minimum separation of six (6) inches should be maintained between a handheld wireless mobile device and an implantable medical device, such as a pacemaker or implantable cardioverter defibrillator, to avoid potential interference with the device.

Persons who have such devices:

- Should ALWAYS keep the mobile device more than six (6) inches from their implantable medical device when the mobile device is turned ON;
- Should not carry the mobile device in a breast pocket;
- Should turn the mobile device OFF immediately if there is any reason to suspect that interference is taking place
- Should read and follow the directions from the manufacturer of your implantable medical device. If you have any questions about using your wireless mobile device with an implantable medical device, consult your health care provider.

Environmental Intended Use and Storage Conditions

Intended Use: 32° to 95°F or 0° to 35°C

Storage: -4° to 113°F or -20° to 45°C

Relative Humidity: 5% to 95% (non-condensing)

Contents

Introduction.....	7
Identifying TouchChat Express 8 Features.....	8
Identifying TouchChat Express 10 Features.....	9
Charging the Device and Amplifier.....	10
Turning the iPad On and Off or Putting It to Sleep	11
Turning the iPad On and Off.....	11
Locking the iPad and Putting It to Sleep.....	11
Turning the Amplifier On and Off.....	12
Adjusting the Volume and Muting Sounds	14
Using the Home Button	15
Using the Home Screen.....	15
Using the Touchscreen.....	16
Adjusting Screen Brightness.....	16
Identifying iPad Status Icons	17
Using Control Center.....	18
Syncing with iTunes	19
Backing up and Restoring Information.....	20
Backing up the Device	20
Restoring the Device	20
Updating the Device Software	20
Organizing Apps and Changing Wallpaper	21
Setting the Date and Time	22
Using Accessibility Features	22
Turning on Accessibility Features.....	22
Setting up an Accessibility Shortcut	22
Setting up and Using Guided Access	23
Turning Guided Access On.....	23
Configuring Guided Access Settings	23
Setting up and Starting a Guided Access Session	24
Ending a Guided Access Session.....	24
Using TouchChat.....	25
Opening TouchChat.....	25
Closing TouchChat	25
Finding the TouchChat Version	25
Working with TouchChat Menus.....	25
Working with Vocabulary Files.....	26
English Vocabulary Files	26
Spanish Vocabulary Files	28
Loading a Vocabulary File	29
Customizing a Vocabulary File	30
Copying a Vocabulary File	30
Creating a New Blank Vocabulary File	30
Deleting a Vocabulary File.....	31
Backing up a Customized Vocabulary File.....	31

Activating Edit Mode	31
Using Chat Editor with TouchChat	32
Downloading and Installing Chat Editor.....	32
Configuring Chat Editor.....	32
Enabling Speech in Chat Editor	33
Opening Chat Editor.....	33
Using Chat Editor to Create Instructional Materials	33
Working with Buttons	34
Editing a Button	34
Adding or Changing a Button Action.....	34
Removing or Changing the Order of an Action	35
Modifying Text	35
Inserting Special Characters.....	35
Adding a Symbol from the SymbolStix Library.....	36
Adding a Picture from the Image Library	36
Using the Camera to Take a Picture for Your Button.....	36
Creating a Camera Button.....	37
Resizing a Button.....	37
Changing a Button Style	37
Creating a Button to Link to a Page.....	38
Storing Text on a Button	39
Creating a Volume Control Button.....	39
Creating a Battery Status Button	39
Copying and Pasting a Button	40
Copying and Pasting the Same Button.....	41
Copying and Pasting a Button Style	41
Moving a Button	42
Hiding a Button	42
Hiding or Showing Multiple Buttons at Once.....	42
Changing Button Timing.....	43
Hiding and Showing Button Navigation Arrows.....	43
Working with Pages.....	44
Renaming a Page.....	44
Duplicating a Page.....	44
Loading Another Page.....	44
Creating a New Page	45
Creating a New Page from a Navigate Action	45
Choosing a New Keyboard	46
Copying a Page from a Different Vocabulary File	46
Adding a Gesture to a Page.....	47
Creating a Template Page	47
Using a Template to Create a Page	47
Editing an Abbreviation Expansion.....	48
Editing Pronunciations	48
Adding Acapela Vocal Smileys.....	48
Adding a Sound	48
Adding an Exclamation.....	48
Finding a Vocal Smiley.....	48
Example: Add a Smiley Face and Laugh Sound to a Button	49

Setting up Your VocaliD™ Voice	50
Creating a Visual Scene	51
Selecting a Background Image	51
Adding Buttons.....	52
Enlarging Text in the Speech Display Bar	53
Changing Text Size or Number of Lines in the SDB	53
Expanding the SDB by Changing Screen Orientation	53
Creating a Tap Action	53
Setting up Switch Scanning in TouchChat	54
Configuring Switches.....	55
Setting up Touchscreen Scanning	55
Selecting the Scan Pattern	55
Configuring Speech Display Bar (SDB) Scans.....	55
Configuring Empty Cells Scans	55
Setting Scan Timing	56
Configuring Re-scans.....	56
Turning on Auto Restart.....	56
Setting Auditory Scan.....	56
Choosing a Voice	56
Making an In-App Purchase.....	56
Playing a Song from TouchChat.....	57
Using SocialChat	57
Set up a Button to Play a YouTube™ Video	58
Blocking and Restoring Access to the Editing Menus.....	59
Working with Profiles	60
Changing TouchChat Settings.....	62
Updating the TouchChat Application Software	65
Analyzing Language Development	66
Using the Realize Language Website.....	66
Turning on Data Logging from Your Device.....	67
Creating a Privacy Password.....	67
Uploading Data for Analysis	68
Clearing Usage Data	68
Changing or Removing Your Privacy Password	69
Transferring and Copying Vocabulary Files	70
Copying Files To and From iShare	70
Transferring a Vocabulary File from Chat Editor to the Device using iShare	70
Copying a Vocabulary File from the Device to Chat Editor using iShare	71
Transferring a Vocabulary File from Chat Editor to the Device using iTunes.....	71
Copying a Vocabulary File from the Device to Chat Editor using iTunes.....	72
Sharing a Customized Vocabulary File.....	73
Sending the File.....	73
Receiving the File	73
Setting up and Using Switch Scanning.....	74
Enabling Scanning.....	74
Defining the Switches.....	74

Setting the Scanning Style	75
Auto Scanning	75
Manual Scanning.....	75
Single Switch Step Scanning.....	75
Additional Timing Options	76
Configuring Additional Scan Settings	76
Using Keyboards	78
Using On-screen Keyboards	78
Using the Apple Wireless Keyboard	79
Using an External Keyboard with TouchChat	79
Entering and Modifying Text.....	80
Typing Text	80
Working with Predictive Text	80
Correcting Misspelled Words	80
Modifying Text	81
Using Shortcuts	81
Dictating Text	81
Searching Apps	82
Changing the iPad Language and Word Prediction Dictionaries	83
Setting up Restrictions.....	83
Changing Screen Orientation	84
Attaching a Strap	85
Positioning, Removing, or Reattaching the Stand	87
Positioning the Stand	87
Removing the Stand	88
Reattaching the Stand	88
Removing the Handle	88
Attaching a Wheelchair Mounting Plate	89
Replacing the Rubber Boot	90
Cleaning and Disinfecting.....	91
Routine Cleaning and Disinfecting	91
Disinfecting TouchChat Express and Accessories for Multiple Users.....	91
Troubleshooting	92
Warranty	93
Appendix A: Advanced Grammar Features	94
Creating a Grammar Button	94
Identifying Parts of Speech.....	94
Creating a Grammar Rule Button	95
Using Your Grammar Rule Button	96
Clearing a Grammar Rule	96

Introduction

TouchChat® Express features a reusable, protective case. The rubber boot and plastic shell provide protection while the amplifier delivers high quality audio output. The handle provides optimal portability.

This user's guide offers step-by-step instructions for setting up and using your TouchChat Express device. Additional information and support are available.

Finding Additional Information on Using Your iPad:

This user's guide covers iPad® basics. For detailed instructions on using all iPad features, refer to the *iPad User Guide*, available from any of these sources.

Source	How to Find
Safari®	Open Safari, tap  , and tap the iPad User Guide bookmark. Hint: Add an icon for the <i>iPad User Guide</i> to the Home screen. Tap  and then tap Add to Home Screen .
iBooks®	Open iBooks and search for iPad user in the iBooks Store.
Internet	Go to http://help.apple.com/ipad .

Support articles can be found on the Saltillo website:

<http://saltillo.com/support/>

Training opportunities can be found on the Saltillo website:

<http://saltillo.com/webinars>

Contact Saltillo for Operational or Technical support at:

Phone: 1-800-382-8622

Email: service@saltillo.com

Trouble Tickets:


<http://saltillo.com/tt/>

For Realize Language™ website support:

support@realizelanguage.com

Identifying TouchChat Express 8 Features


Rear View

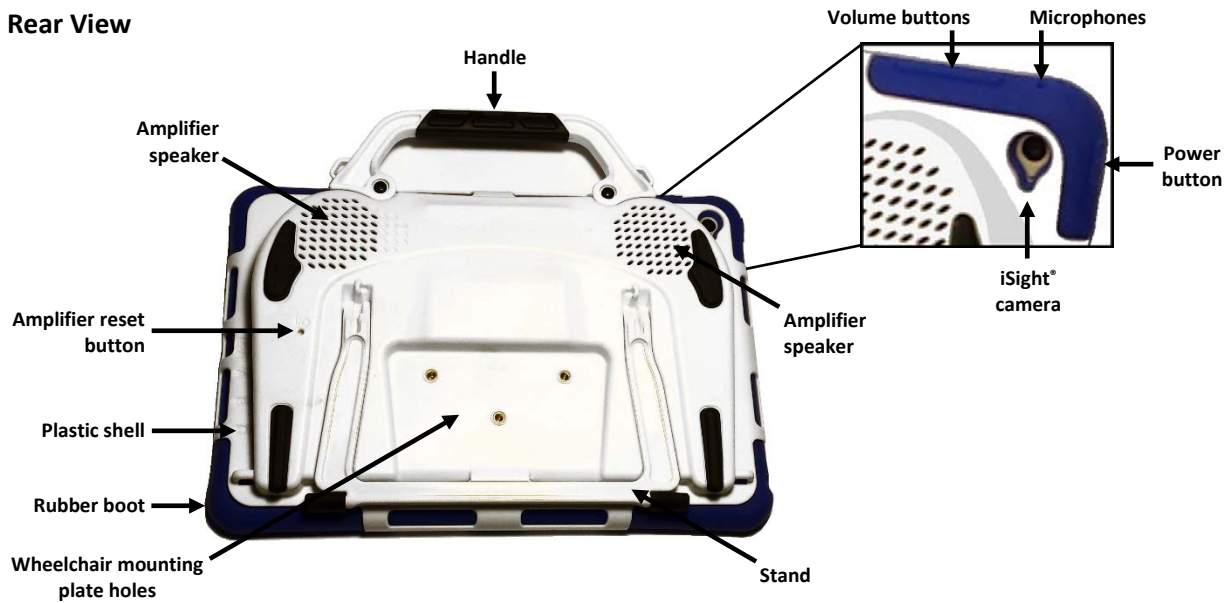Front View


Side View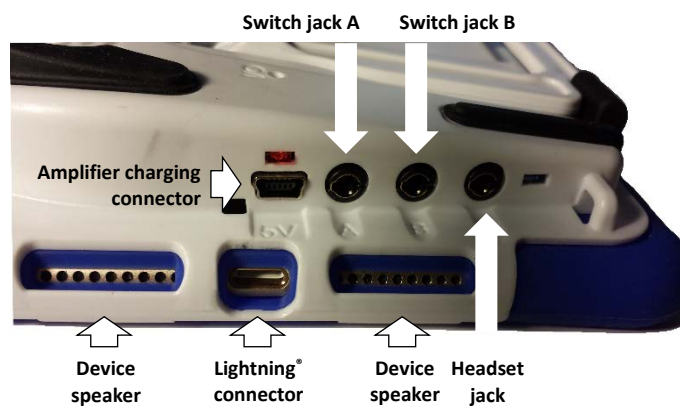

Identifying TouchChat Express 10 Features


Rear View

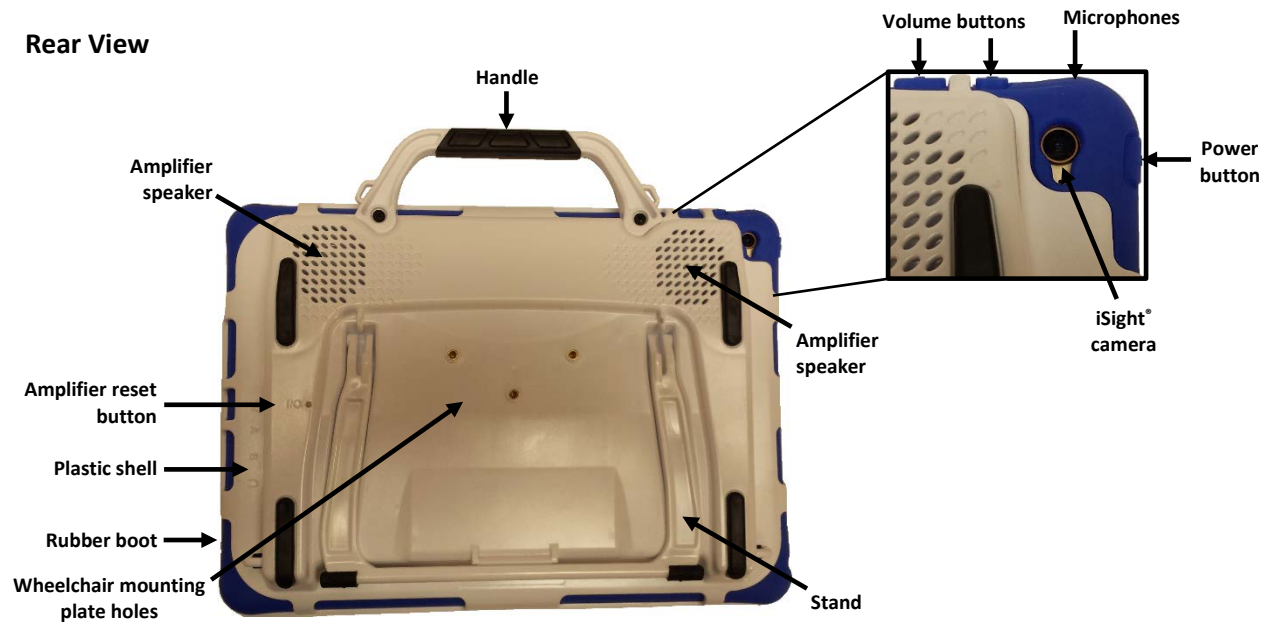Front View

Side View

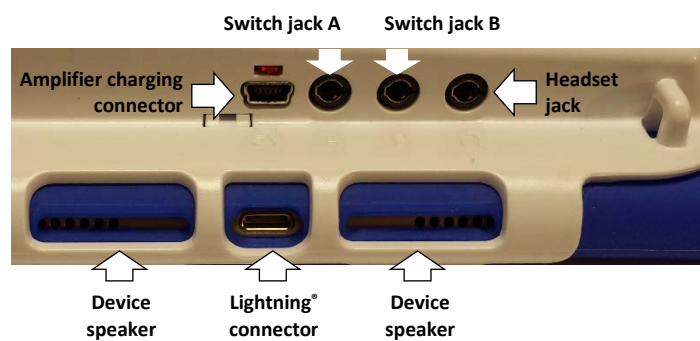
Charging the Device and Amplifier

Connecting the Charger

You received a battery charger with your TouchChat Express. One end of the charger plugs into a wall outlet—a surge protector is recommended. The other end has two connectors, allowing you to charge the device and amplifier at the same time.

- One connector plugs into the amplifier.
- The other connector is a receptacle for the USB cable that came with your device.

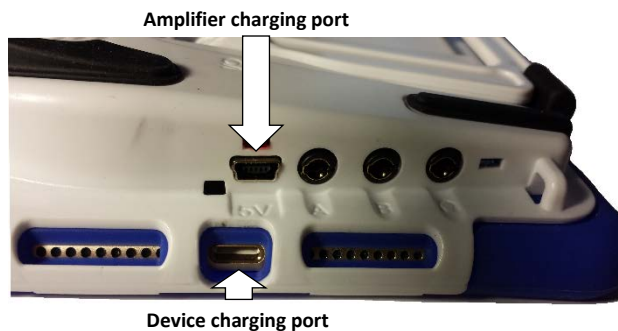Using the Backup Charger

A backup charger came with your device. Use that charger only if necessary in an emergency. The backup charger can only be used with the device. *Do not use it to charge the amplifier.*

To charge the device and amplifier:

Hint: We suggest charging both the device and the amplifier each night.

1. Tap the device's power switch to turn the display light off.
2. Plug the charger's amplifier connector into the amplifier.
3. Plug one end of the USB cable that came with your device into the device and the other end into the charger's receptacle.
4. Plug the other end of the charger into a wall outlet or surge protector.
5. The amber light next to the amplifier's charger port will flash during charging, and the blue light will begin flashing when the amplifier is fully charged.

When the device and amplifier are fully charged:

1. Unplug the connectors from the device and amplifier.
2. Unplug the charger from the wall outlet or surge protector (*optional*).
3. Tap the device's power switch to turn the display light on.

Checking Battery Status

To check the iPad's battery level or charging status at any time, press the **Home** button. The current battery status is displayed in the top right corner of the screen. *The battery status indicator only shows the battery level or charging status of the iPad, not the amplifier.*

You can also create a button with the Battery Status action or find and tap a battery status button in your vocabulary.

Amplifier batteries are not replaceable by the customer.

Disposing of the Device or the Amplifier Battery

Device Disposal: Please dispose of your device in accordance with your local, state, and/or federal electronic recycling laws.

Battery Disposal: If the battery in your amplifier needs to be replaced, dispose of the old battery in accordance with your local, state, and/or country regulations.

Turning the iPad On and Off or Putting It to Sleep

The power button allows you to turn the iPad on and off or lock the iPad and put it to sleep.

Turning the iPad On and Off

To turn the iPad on, hold down the power button until the Apple® logo appears on the screen.

To turn the iPad off, hold down the power button for a few seconds until “slide to power off” appears on the screen. Then drag the **slide to power off** text.

Locking the iPad and Putting It to Sleep

You can lock the iPad and put it to sleep when you are not using it. Locking the iPad turns off the display, and turns off touch capability, and saves battery power. The iPad is still able to receive calls, text messages, alarms, and notifications. You can also listen to music and adjust the volume.

To lock the iPad:

To lock the iPad, press the power button. If you do not touch the screen for two minutes, the iPad will lock automatically. To change the auto-lock time, choose **Settings > Display & Brightness > Auto-Lock**.

To unlock the iPad:

TouchChat Express 8: Press the **Home** button or the power button.

TouchChat Express 10: Press the **Home** button or the power button, and then drag the **slide to unlock** text that appears on the screen.

To add a passcode:

If you want to require a passcode to unlock the iPad, choose **Settings > Touch ID & Passcode**. Then use the passcode options to set the passcode.

Turning the Amplifier On and Off

The device and amplifier communicate using a Bluetooth connection. Typically, the amplifier should be left on. During air travel and in some medical facilities the Bluetooth option is required to be turned off. When the amplifier is off, speech is still present but comes from the device's internal speaker.

Turning Bluetooth Off

1. Open the device's **Settings** menu.
2. Turn **Bluetooth** off.


Turning Bluetooth On

1. Open the device's **Settings** menu.
2. Turn **Bluetooth** on.
3. Choose the amplifier from the list of options under "Devices" to the right of "Bluetooth".
4. Close the Settings menu.
5. The blue light on the edge of the amplifier should be solid blue.

Connecting Bluetooth to the TouchChat Software

1. Open the TouchChat app.
2. Open your vocabulary file and choose **Menu > Settings**.
3. Scroll down to the ChatWrap Connect section.
4. Choose **Select ChatWrap** and choose your ChatWrap® number.

Turning the Amplifier On and Off

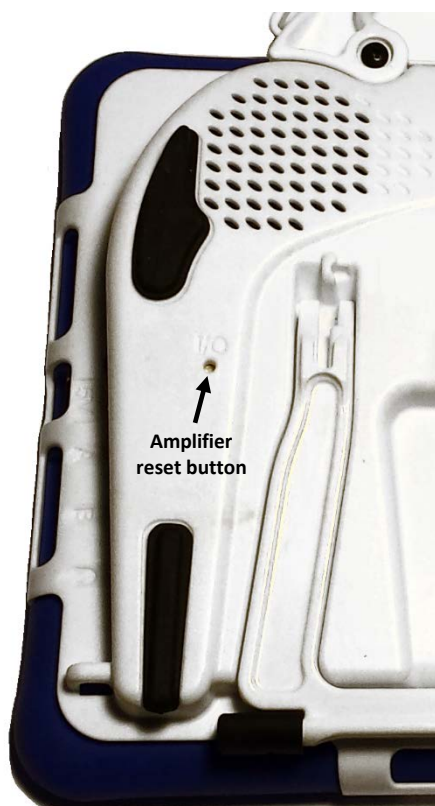To turn the amplifier off: Press and hold the amplifier reset/power button (labeled “I/O”) until both the amber and blue lights come on. When the amber light goes out, let go of the button.

Hint: The button is recessed. Use the tip of a pen when you need to press it. Do **not** use an object with a sharp point, such as a pin; it could damage the button.

To turn the amplifier on: Press the reset/power button for a second. The blue light will flash to indicate the amplifier is on.

Adjusting the Volume and Muting Sounds


Excessive volume or sound pressure level when using earphones or headphones can cause hearing loss over extended periods of time.

Using the Volume Buttons


The iPad volume buttons are located on the edge of the device. Pressing one button increases the volume and pressing the other lowers the volume.

The volume buttons fit into a slot in the TouchChat Express plastic shell. The rubber boot for some models covers the two volume buttons with a single button. Press one side of the button to increase the volume and press the other side of the button to decrease the volume. By pressing a side, you are pressing the device button underneath.

To mute sound, press and hold the Volume Down button.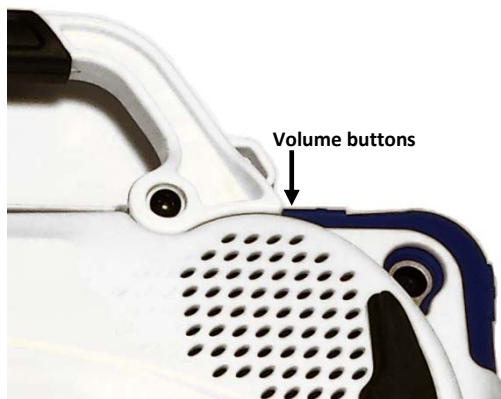

TouchChat Express 8

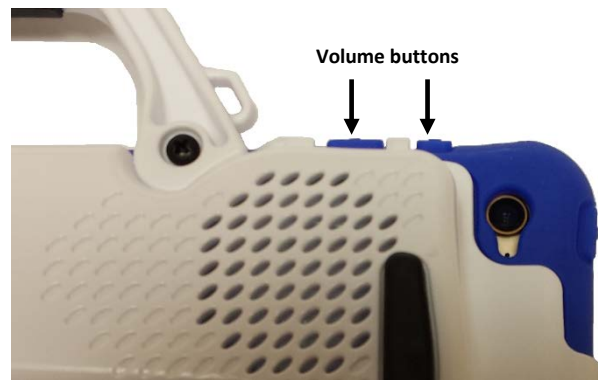TouchChat Express 10

Using Control Center

When iPad is locked or when you are using another app, swipe up from the bottom edge of the screen to open Control Center. Do Not Disturb, also available in Control Center, is an easy way to keep the iPad silent. For additional information, see [“Using Control Center”](#) on page 18.

Using the Home Button

Press the Home button to display the Home screen at any time.

The Home screen displays apps. Tap an app to open it. To see the apps that you have opened, double-press the **Home** button and swipe left or right.

To use Siri® (on iPad 3rd generation or later), press and hold the **Home** button.

You can also press the **Home** button quickly three times to turn accessibility features on or off.

To take a screen shot, press and hold the power button and **Home** button at the same time.

Note: On iPad models with Touch ID™, you can use the sensor in the Home button, instead of using your passcode or Apple ID password, to unlock iPad or make purchases in the iTunes Store, App Store, and iBooks Store. You can also use the Touch ID sensor for authentication when using Apple Pay to make a purchase from within an app.

Using the Home Screen

All the apps that came on your iPad, plus any apps you download, are available from the Home screen. To open an app, simply tap it.

To display the Home screen at any time, press the **Home** button. Swipe left or right to go to other screens.

Using the Touchscreen

Your device comes with a capacitive touchscreen. It responds best to a light touch from the pad of your finger.

Important! Using excessive force or a metallic object when pressing on the touchscreen may damage the tempered glass surface.

Stylus

A stylus was provided in your TouchChat Express package. If you prefer to get your own stylus, it must be compatible with a capacitive touchscreen.

Note: The stylus provided has an opening on the end for attaching to you or the device. Find a chain, string, or lanyard that matches your personal style.

The stylus is a potential hazard for choking or for poking the eye.

Adjusting Screen Brightness

Dim the screen to extend battery life, or use Auto-Brightness.

1. Choose **Settings > Display & Brightness**.
2. Drag the slider.
3. If Auto-Brightness is turned on, the iPad adjusts the screen brightness for current light conditions using the built-in ambient light sensor.

Note: You can also adjust screen brightness in Control Center.

Identifying iPad Status Icons


The status bar is located at the top of the iPad screen. These icons provide the time, battery status, and information about connectivity and operations currently running.

Note: You can hide the status bar when using the TouchChat app. In TouchChat, choose **Menu > Settings**. Under Page Size, turn **Status Bar** off.

Status Icon	Description
	Activity A third-party app is using a network or generating some other activity.
	Airplane Mode Airplane Mode is turned on. In this mode, only non-wireless features are available. You cannot access the Internet or use Bluetooth devices.
	Alarm An alarm is set.
	Battery Shows the battery level or charging status.
	Bluetooth <i>Blue or White icon:</i> Bluetooth is on and paired with a device such as a headset or keyboard. <i>Gray icon:</i> Bluetooth is on and paired with a device that is out of range or turned off. <i>No icon:</i> Bluetooth is not paired with a device.
	Bluetooth battery Shows the battery level of a supported paired Bluetooth device.
	Do Not Disturb Do Not Disturb is turned on. FaceTime® calls and alerts will not sound or light up the screen.
	Location Services A location-based app is gathering and using data indicating the iPad's location.
	Lock The iPad is locked. The display and touch capability are turned off.
	Orientation Lock Screen orientation is locked.
	Personal Hotspot The iPad is sharing its Internet connection with other iOS devices.
	Syncing iPad is syncing with iTunes®.
	VPN The iPad is connected to a network using a VPN (virtual private network).
	Wi-Fi The iPad has an active Wi-Fi connection. More bars indicate a stronger connection.

Using Control Center

Control Center provides quick access to the iPad controls. *This is an example only. Controls vary by iOS version and are customizable.*

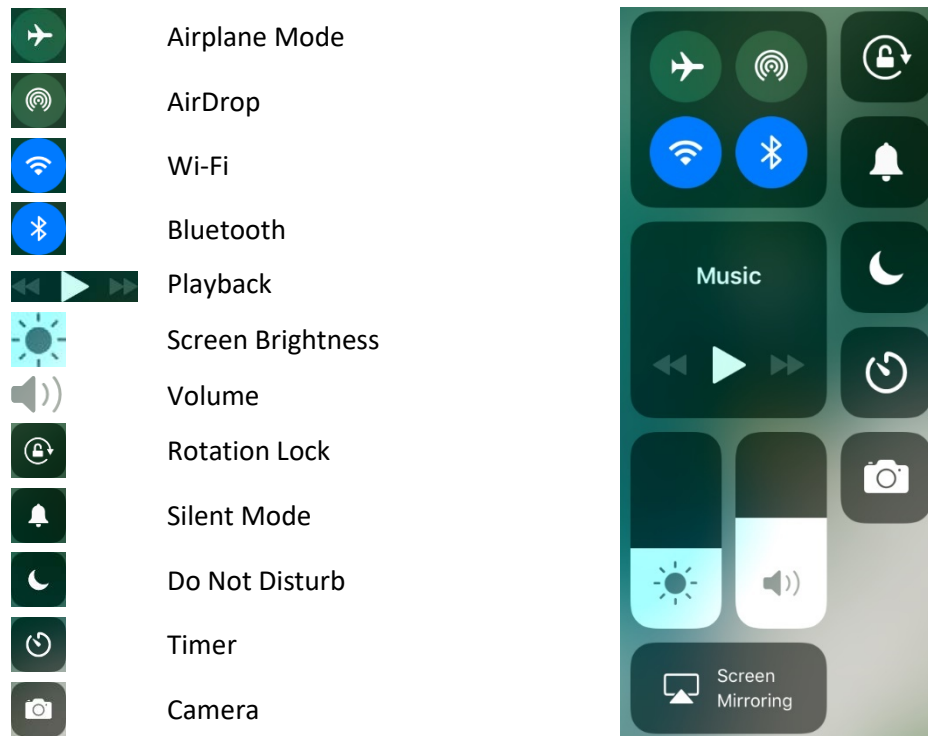To open Control Center:

To open Control Center, swipe up from the bottom edge of any screen.

To close Control Center:

To close Control Center, use any of the following methods:

- Tap the **Close** icon
- Swipe down
- Tap anywhere on the screen above Control Center
- Press the **Home** button

To prevent access to Control Center:

To prevent the user from accessing Control Center, do the following:

1. Choose **Settings > Control Center**.
2. Turn off **Access on Lock Screen** or **Access Within Apps** or both.

Syncing with iTunes

Syncing with iTunes copies information from your computer to the device and from the device to your computer. You can sync by connecting the device to your computer, or you can set up iTunes to sync wirelessly using Wi-Fi.

Setting up Wireless Syncing

Connect the iPad to your computer. In iTunes on your computer, select your iPad, click **Summary > Sync with this iPad over Wi-Fi**. If Wi-Fi syncing is turned on, the iPad syncs when (a) it is connected to a power source, (b) both the iPad and your computer are turned on and connected to the same wireless network, or (c) iTunes is open on your computer.

Syncing with iTunes on your Computer

Connect the device to your computer, select it in iTunes, and then set options in the different panes. If the device does not appear in iTunes, make sure you are using the latest version of iTunes, verify that the cable is correctly connected, and then try restarting your computer.

In the Summary pane, you can set iTunes to sync the device automatically when it is attached to your computer. To temporarily prevent syncing when you attach the device, hold down Command and Option (Mac®) or Shift and Control (PC) until you see the device appear in the iTunes window.

To sync using the Info, Music, or Photos panes:

Pane	Description
Info pane	Click Advanced to select options that let you replace the information on the iPad with the information from your computer during the next sync.
Music pane	Sync music using your playlists.
Photos pane	Sync photos and videos from a supported app or folder on your computer.

Importing Photos

1. Connect your device to the computer and launch iTunes.
2. Once iTunes is up, go to the **Photo** tab.
3. Put a check mark before **Sync Photos from** and choose where you want to import the pictures from.
4. Select the **Sync** button in the bottom right.
5. Once the sync is complete, disconnect the device from the PC.

Encrypting Backup Information

To encrypt the information stored on your computer when iTunes makes a backup, select **Encrypt iPad backup** in the Summary pane. Encrypted backups are indicated by a lock icon , and a password is required to restore the backup.

If you do not select this option, other passwords (such as those for mail accounts) are not included in the backup and you will need to reenter them if you use the backup to restore iPad.

Note: If you use iCloud to store your contacts, calendars, and bookmarks, do *not* sync them to the iPad using iTunes. If you turn on iCloud Photo Library, you cannot use iTunes to sync photos and videos to the iPad.

Backing up and Restoring Information

Important! This procedure does not back up TouchChat vocabulary files. For information on backing up vocabulary files, see “[Backing up a Customized Vocabulary File](#)” on page 31.

Backing up the Device

1. Open iTunes on the computer.
2. Connect your device to the computer using a USB cable.
3. In iTunes, the device will appear on the left side.
4. Right-click on the device and choose **Back Up**.
5. iTunes will start to back up the device.

Restoring the Device

1. Open iTunes on the computer.
2. Connect your device to the computer using a USB cable.
3. In iTunes, the device will appear on the left side.
4. Right-click on the device and choose **Restore from Backup**.
5. iTunes will then prompt you to select the backup. Choose the one that is the most recent sync, to ensure having the current versions.
6. iTunes will start restoring the device from the backup.

Updating the Device Software

When an iOS software update is available, you can perform the update wirelessly or, if you cannot update wirelessly, by using iTunes.

Updating Your iPad Wirelessly

1. Connect to the Internet with Wi-Fi
2. Choose **Settings > General > Software Update**. The Software Update screen will be displayed.
3. Tap **Download and Install**.

Updating Your iPad using iTunes

With the latest version of iTunes installed on your computer,

1. Connect the device to the computer.
2. Open iTunes and select your device.
3. Click **Summary** and then click **Check for Update**.
4. Click **Download and Update**.

Organizing Apps and Changing Wallpaper

You can rearrange your apps and organize them in folders. You can also change the background image used as wallpaper.

Arranging Your Apps

1. Touch and hold any app on the Home screen until it jiggles.
2. Drag the app to the position you want.
 - Drag an app to a different position on the same screen.
 - Drag an app to the edge of the screen to move it to a different Home screen.
 - Drag an app to the Dock at the bottom of the screen.
3. Press the **Home** button to save your arrangement.

Organizing Apps in Folders

You can have multiple pages of apps in a folder.

1. While arranging apps, drag one app onto another to create a folder.
2. Tap the name of the folder to rename it.
3. Drag apps to add or remove them.
4. Press the **Home** button when you finish.

Note: To delete a folder, drag all the apps out of it. The empty folder will be deleted automatically.

Creating a New Home Screen

While arranging apps, drag an app to the right edge of the last Home screen. The dots above the Dock show which of your Home screens you are viewing.

When iPad is connected to your computer, you can customize the Home screen using iTunes. In iTunes, choose **iPad > Apps**.

Changing the Wallpaper

Wallpaper settings allow you to set an image or photo as wallpaper for the Lock screen or Home screen. You can choose from dynamic and still images.

To change the wallpaper, choose **Settings > Wallpaper > Choose a New Wallpaper**.

When you choose an image for new wallpaper, the Perspective Zoom button determines whether your selected wallpaper is zoomed. For wallpaper you already set, go to the Wallpaper setting, then tap the image of the Lock screen or Home screen to see the Perspective Zoom button.

Restoring Your Original Apps Layout and Wallpaper

Choose **Settings > General > Reset > Reset Home Screen Layout**. The folders you created are removed, and the original wallpaper is restored.

Setting the Date and Time

You can set your iPad to retrieve the time and date automatically or you can set it manually. You also have the option to show time in 24-hour or 12-hour time.

Set Automatic Date and Time Updates

Choose **Settings > General > Date & Time**, and then turn **Set Automatically** on. If you set the iPad to update the time automatically, the iPad retrieves the correct time over the network and updates it for the time zone you are in.

Set the Date and Time Manually

Choose **Settings > General > Date & Time**, and then turn **Set Automatically** off.

Set 24-Hour or 12-Hour Time

Choose **Settings > General > Date & Time**, and then turn 24-Hour Time on or off.

Using Accessibility Features

You can turn on accessibility features individually and/or create a shortcut to a specific group of features.

Turning on Accessibility Features

To turn on one or more accessibility features individually, choose **Settings > General > Accessibility**. Accessibility features are listed by category: Vision, Interaction, Hearing, and Learning. Tap a feature to turn it on, or tap a feature and choose the option you want.

Setting up an Accessibility Shortcut

As an alternative to selecting individual accessibility features, you can create a shortcut to a specific group of accessibility features.

1. Choose **Settings > General > Accessibility > Accessibility Shortcut**. The following features will be listed: VoiceOver, Invert Colors, Grayscale, Zoom, Switch Control, and AssistiveTouch.

Note: If you already turned on Guided Access, Guided Access will be listed first and display a checkmark. See the next section, [“Setting up and Using Guided Access”](#).

2. Tap each feature you want to use. A checkmark will be displayed after each selected feature.

Setting up and Using Guided Access

Guided Access® helps the user stay focused on a task by dedicating the iPad to a single app and allowing you to control which app features are available. Use Guided Access to: Temporarily restrict the iPad to a particular app; disable areas of the screen that do not apply to a task, or areas in which an accidental gesture might be distracting; limit how long the user can use an app; or disable iPad device buttons.

Turning Guided Access On

Choose **Settings > General > Accessibility > Guided Access**. Then turn **Guided Access** on.

Configuring Guided Access Settings

When you turn Guided Access on, additional options appear: Passcode Settings, Time Limits, and Accessibility Shortcut.

Setting a Passcode

Passcode Settings allows you to set a passcode to prevent the user from leaving a Guided Access session.

1. Tap **Passcode Settings**.
2. Tap **Set Guided Access Passcode**. The Set Passcode window opens.
3. Tap numbers to set your passcode.

Note: This setting also turns on Touch ID, providing an alternative way to end a Guided Access session.

Setting Sounds

Time Limits allows you to set an alert tone or ring tone or have remaining Guided Access time spoken.

1. Tap **Time Limits**.
2. Tap **Sound**. The Alert Tones and Ringtones lists are displayed.
3. Tap a tone to hear how it sounds.
4. Keep tapping tones until you hear the one you want.

Note: To turn off *both* alert tones and ringtones, tap **None (Default)**.

To have the remaining Guided Access time spoken before time ends:

1. Tap **Time Limits**.
2. Turn on **Speak**.

Allowing Accessibility Shortcuts

If you set up a shortcut to a group of accessibility features (see [“Setting up an Accessibility Shortcut”](#) on page 22), you can make those accessibility features available during a Guided Access session.

1. Choose **Settings > General > Accessibility > Guided Access**.
2. Turn on **Accessibility Shortcut**.

With Accessibility Shortcut turned on, when you press the **Home** button quickly three times while Guided Access is enabled, a menu will list the accessibility features you chose. Simply tap a shortcut to turn on the feature.

Setting up and Starting a Guided Access Session

After turning on Guided Access, do the following:

1. Open TouchChat or any other app you want the user to work with.
2. Press the **Home** button quickly three times. The Guided Access options will be displayed at the bottom of the screen.

3. Adjust settings for the session.

To ...	Do this:
Disable app controls and areas of the app screen	Draw a circle or rectangle around any part of the screen you want to disable. Drag the mask into position or use the handles to adjust its size.
Turn on the Sleep/Wake button	Tap Options below “Hardware Buttons” and turn on Sleep/Wake Button . When this button is turned on, the iPad will sleep after a period of time, and the device lock button can be pressed to lock the device immediately. This setting conserves battery power. <i>Note:</i> When Guided Access is turned on, Auto-Lock time is not used, which can keep the device active and drain the battery.
Turn on the Volume buttons	Tap Options below “Hardware Buttons” and turn on Volume Buttons .
Prevent the iPad from responding to motion	Tap Options below “Hardware Buttons” and turn off Motion . When motion is turned off, the iPad will not switch between portrait and landscape orientations when the device is rotated nor will it respond to any other motions.
Prevent typing	Tap Options below “Hardware Buttons” and turn off Keyboards .
Ignore screen touches	Turn off Touch .
Set a session time limit	Tap Options below “Time Limit” and tap Time Limit on the pop-up menu to activate the settings. Then scroll through the numbers and tap the hour and minute numbers you want.

4. Tap **Start** in the upper right corner of the screen.

Ending a Guided Access Session

1. Press the **Home** button quickly three times.
2. Enter your Guided Access passcode.

Note: Alternatively, use Touch ID (which was enabled when you set the passcode). Touch ID uses the sensor in the Home button instead of your passcode.


Using TouchChat

TouchChat is intended for individuals who have difficulty using their natural voice. TouchChat is designed for individuals with Autism, Down Syndrome, ALS, apraxia, stroke, or other conditions that affect a person's ability to use natural speech.

Opening TouchChat

To open TouchChat from the Home screen, tap the TouchChat app.

Note: The TouchChat app logo will vary depending on the specific TouchChat app you have.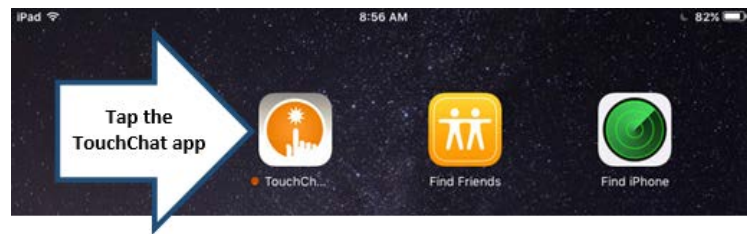

Closing TouchChat

There is a difference between minimizing the application and closing it. Tapping the Home button minimizes the application, but it continues to run in the background. To close TouchChat or any other application:


1. Tap the **Home** button on the device. This will minimize, not close, the application.
2. Double-tap the **Home** button. This will open the multi-tasking screen.
3. Drag the app up from the multi-tasking screen.
4. Once the application is closed, press the **Home** button to return to the Home screen.

Finding the TouchChat Version


1. Tap **Vocab**.
2. Tap **Choose New Vocab**.
3. In portrait mode, the software version will be displayed at the bottom of the screen.

Working with TouchChat Menus


To access the TouchChat menus, tap **Menu**. Menu options depend on where you are in the application and whether Edit Mode is turned on. For example: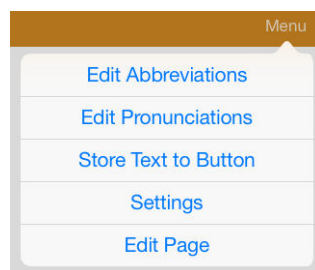

Menu options with Edit Mode turned off (gold menu bar)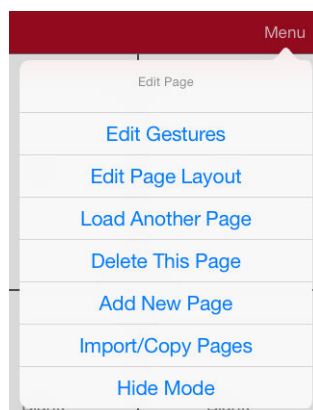

Menu options with Edit Mode turned on (red menu bar)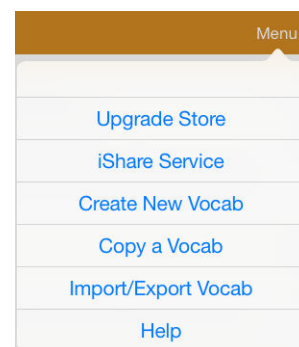

Menu options after choosing Vocab > Choose New Vocab > Menu (gold menu bar)

Working with Vocabulary Files

TouchChat provides a variety of pages that have different button layouts. Each set of pages, or vocabulary set, is called a vocabulary file. TouchChat-provided vocabulary files have locks in front of them indicating they cannot be deleted. This section provides brief descriptions of the available vocabulary files.

English Vocabulary Files

Note: When the system language is set to English, the English vocabulary files will be listed individually when you tap Choose New Vocab and the Spanish vocabulary files will be located in a folder labeled “Spanish”. When the language is set to Spanish, the Spanish vocabulary files will be listed individually when you tap Choose New Vocab and the English files will be located in a folder labeled “Inglés”.

4-Basic

4-Basic offers just that, a basic vocabulary option with 4 buttons per page.

Communication Journey: Aphasia

“Communication Journey: Aphasia” is a vocabulary file containing features and vocabulary designed to support people with aphasia. It was developed by Lois Turner (SLP), Anne MacCallum (SLP) and Sarah Gauthier (SLP Assistant) in consultation with a group of speech-language pathologists who work exclusively with people with aphasia and brain injuries. All the participants work at CAYA (Communication Assistance for Youth and Adults) and the GF Strong Rehabilitation Centre in Vancouver, BC, Canada. The framework of the file was derived from the Life Participation Approach to Aphasia from the Aphasia Institute in Toronto, Canada. The pages were created to facilitate participation in daily activities and to promote social links.

Aphasia, a disorder caused by damage to the parts of the brain where language is stored, may affect the ability to comprehend language, to express oneself, to read, or to spell and write. “Communication Journey: Aphasia” can easily be adapted for use by people with different types of aphasia and a range of cognitive levels and degrees of severity.

Symbols, photographs, visual scenes, videos, or words can be used on the message, topic, and script pages. Included are pages that facilitate supported communication techniques, self-advocacy, directing care, and repair of communication breakdowns. Types of information typically included in communication books—such as pain and emotion scales, maps, and templates for personal information and life stories—are also incorporated into the vocabulary file. Specialized features include variable “rate of speech” and “pause between words”. Grid size and vocabulary are easily customized to reflect and support the life experiences of the person with aphasia.

Although this vocabulary file contains starting-point vocabulary commonly required by adults with aphasia, it **must** be customized. Each person has a unique history, and the file must reflect that person’s life, experiences, and abilities.

MultiChat 15

MultiChat 15 features a 15-button layout and is available in three versions: one for school-aged individuals with emerging language skills, one for adolescents, and one for adults. There are multiple forms of communication available in this program: sentences, phrases, individual words, recordings for story-telling, and visual scenes. Also included are interactive play and reading pages, social pages and a News-2-You starter page for the weekly newspaper’s vocabulary. There are symbols on every button with the exception of several core words.

myCore

myCore is a combination of core and phrase-based vocabulary geared toward individuals who have literacy skills.

myQuickChat

myQuickChat is an introductory communication system geared toward child and adult AAC users with complex communication needs. myQuickChat is available for both kids and adults in 4, 8, and 12 locations, each with an identical 16-location Support Master Home Page. myQuickChat was created to offer a high frequency, phrase-based communication system in an easy to use yet very engaging format. The progressive system offers a variety of topics for everyday needs and conversation and provides AAC users immediate and more successful communication exchanges within a variety of settings.

Spelling

Spelling is a QWERTY keyboard page set with four word prediction buttons and a few pre-stored phrases.

VocabPC

VocabPC was designed by Gail Van Tatenhove, PA, MS, CCC-SLP, for adults and adolescents with developmental disabilities. Vocabulary in VocabPC is arranged as carrier phrases, interactive sentences, activity vocabulary, and naming words. VocabPC uses a 12 location page layout. Both portrait and landscape versions of VocabPC are included.

WordPower20 Simply

WordPower20 Simply incorporates carrier phrases with core words for quick and easy language generation. It can be used as a phrase-based system or as a word-based system. For example, you could use the carrier phrases “I want...”, “I like...”, and “I need...” to generate sentences. Or you could use the individual pronouns “I”, “it”, and “you”, or sentences starters such as “can” and “do.” When the individual pronouns are selected, a page of frequently used verbs and helping verbs also becomes available. You can quickly finish a sentence using a phrase such as “to eat...”, “to play...”, “to watch...”, etc., or you can select “to” and have a more expanded set of verbs available.

WordPower25 Touch & Scan

The WordPower25 Touch & Scan vocabulary is designed to be accessible via direct selection as well as scanning for quick and easy language generation. It incorporates carrier phrases with core words that move seamlessly between the main page and related categories. Individuals who need a sophisticated and efficient language system with a minimum number of cells per page will benefit from this robust vocabulary.

WordPower42

WordPower42 is a word-based vocabulary that allows for easy and intuitive communication. WordPower42 is a generative language system that will be familiar to those using WordPower on other systems, and it is easy to learn for new users.

WordPower42 Basic

WordPower42 Basic is a simplified version of WordPower42. This word-based system remains rich in core vocabulary, and it may be appropriate for beginning users or those who need a more basic vocabulary design. It is designed to make communication fast and easy.

WordPower48

WordPower48 is very similar to the 42-location, but in the extra column on the left, you now have a clear display, delete word, period, and plural. This configuration can be set up for scanning or to be used with a keyguard.

WordPower60

WordPower60 consists of a large number of high-frequency words available on the main page, resulting in fast communication with reduced keystrokes. Word completion and logical next words are used, and the “grammar” function provides morphological endings to verbs, nouns, and adjectives. Spelling with word prediction along with the category-based pages contained in all other versions of the vocabulary are also available. This vocabulary is configured to work with a keyguard.

WordPower60-Basic

WordPower60-Basic was designed with early language learners in mind, providing ample picture support and novel structures to facilitate expressive language development. Core vocabulary is embedded within category and activity pages, allowing easy generation of language from the main core page as well as within specific topic-based pages. Core words remain consistent in terms of placement and access, providing a natural bridge between core and fringe words. Early literacy is enhanced by providing pages designed to teach language and core through children’s books.

WordPower80

WordPower80 is for someone who has good vision, motor skills, and literacy skills. WordPower80 vocabulary pages consist of core vocabulary, spelling, and word prediction.

WordPower108

WordPower108 consists of a large number of high-frequency words on the main page, resulting in fast communication and quick access to core words. Word completion and logical next words are used, and the “grammar” function provides morphological endings to verbs, nouns, and adjectives.

WordPower108 with Keyboard

WordPower108 with Keyboard is a vocabulary consisting of high frequency core words, spelling, and word prediction. With the keyboard on the main page, the communicator moves seamlessly between words and spelling, thus minimizing keystrokes and the need for navigation. Some core words are embedded within the category pages in order to assist in bridging the gap between core and fringe vocabulary.

Spanish Vocabulary Files

Note: When the system language is set to Spanish, the Spanish vocabulary files will be listed individually when you tap Choose New Vocab and the English vocabulary files will be located in a folder labeled “English”. When the language is set to English, the English vocabulary files will be listed individually when you tap Choose New Vocab and the Spanish vocabulary files will be located in a folder named “español”.

MultiChat 15 Spanish

MultiChat 15 Spanish is the student version developed for devices running TouchChat. This Spanish version is very similar to the English MultiChat 15 Student file. However, due to the differences in the two languages, some changes have been made to accurately represent each language.

MultiChat 15 Spanish Bilingual

This vocabulary file is exactly the same as the MultiChat 15 Spanish file except that the labels on the buttons show both English and Spanish words or phrases. Please note that when the device speaks the words or phrases, they are only spoken in Spanish.

Sin sintaxis 4 x 4 English and Spanish

Sin sintaxis 4 x 4 English and Spanish consists of categories of words with a symbol on each button. This file is intended for children who do not have the ability to construct sentences yet.

Sintaxis 4 x 5 English and Spanish

Sintaxis 4 x 5 English and Spanish is intended for children who are beginning to create simple sentences. Each page contains 20 buttons, and each button contains a word or phrase with a picture. Words are organized into categories with some core words for creating beginning sentences.

Teclado

Teclado is a Spanish keyboard.

VocabPC Spanish

VocabPC Spanish is a direct translation of the English version. VocabPC was designed by Gail Van Tatenhove, PA, MS, CCC-SLP, for adults and adolescents with developmental disabilities. Vocabulary in VocabPC is arranged as carrier phrases, interactive sentences, activity vocabulary, and naming words.

WordPower48 Español

Designed to be intuitive and easy to use, WordPower 48 Español meets the communication needs of a wide group of individuals in Spanish speaking cultures. The vocabulary includes easy to access basic communication for beginning communicators, as well as sophisticated language and grammar for individuals with advanced cognitive and linguistic abilities.

WordPower48 Español Basico


WordPower48 Español Basico was designed with early language learners in mind, providing ample picture support and novel structures to facilitate expressive language development. Core vocabulary is embedded within the category and activity pages, allowing easy generation of language from the main core page as well as within topic-based pages. Core words remain consistent in terms of placement and access, providing a natural bridge between core and fringe words.

WordPower60 Español Básico

WordPower60 Español Básico is comparable in terms of content and design to the English WordPower60 Basic vocabulary. It was designed with early language learners in mind, providing ample picture support and novel structures to facilitate expressive language development. Core vocabulary is embedded within the category and activity pages, allowing easy generation of language from the main core page as well as within topic-based pages. Core words remain consistent in terms of placement and access, providing a natural bridge between core and fringe words. Early literacy is enhanced through the use of pages designed around several popular children's books. A low-tech communication board is included within the vocabulary file that can be printed as an alternative communication support.

Loading a Vocabulary File

1. Tap **Vocab**. Choose New Vocab will be displayed.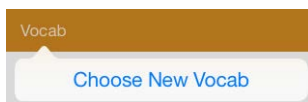

2. Tap **Choose New Vocab**. A page will display a list of the available vocabulary files. Scroll down to see them all.
3. Select the vocabulary file you want to load.
4. Once loaded, explore the pages and button organization by choosing the buttons. Buttons with arrows will navigate to new pages.

Note: If you plan to customize a vocabulary file, you must first make a copy of it. See [“Customizing a Vocabulary File”](#) and [“Copying a Vocabulary File”](#).

Customizing a Vocabulary File

Before making any customizations, we suggest that you set the system language to your language. For information on changing the system language, see [“Changing the iPad Language and Word Prediction Dictionaries”](#) on page 83.


Copying a Vocabulary File

To customize a vocabulary file, you must first make a copy of the original file. Once you make a copy, you can edit the copy. This keeps the original files in place in case they are needed at a later time.

1. Tap **Vocab** and tap **Choose New Vocab**.
2. Tap **Menu** and tap **Copy a Vocab**. The vocabulary files available on the device are listed by language.
3. Tap the vocabulary file to copy.
4. Give the file a new title and description.
5. Tap **Save**.
6. Tap the new file name to open the file.


Creating a New Blank Vocabulary File

1. Tap **Vocab** and tap **Choose New Vocab**.
2. Tap **Menu** and tap **Create New Vocab**.
3. Give the new vocabulary file a title and description.
4. Tap **Save**.
5. Scroll to the new file you created and tap it to open it.
6. Tap **Menu** and tap **Edit Page**. A page of blank buttons opens.
7. For each blank button, select **Create New Button** or **Copy Existing Button**.
8. When you finish, tap **Done**. You have created the first page of your new vocabulary.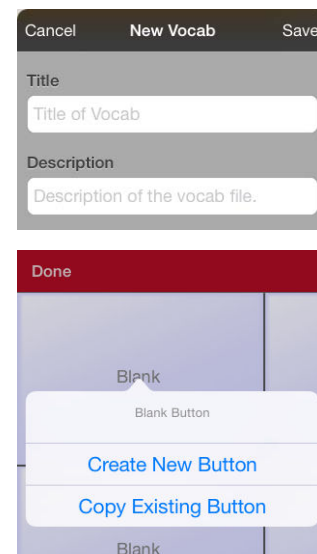


Deleting a Vocabulary File

1. From the home page of the vocabulary file, tap **Vocab**.
2. Tap **Choose New Vocab**.
3. Tap **Edit**.
4. The delete icon  will appear to the left of each padlock image. You will only be able to delete custom vocabulary files. The original vocab files are displayed with a locked padlock and cannot be deleted.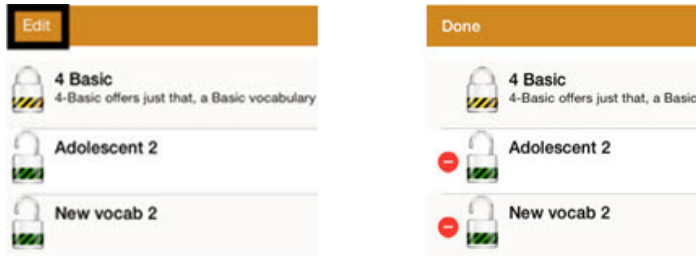

5. When you tap the delete icon , “Rename” and “Delete” buttons will appear.

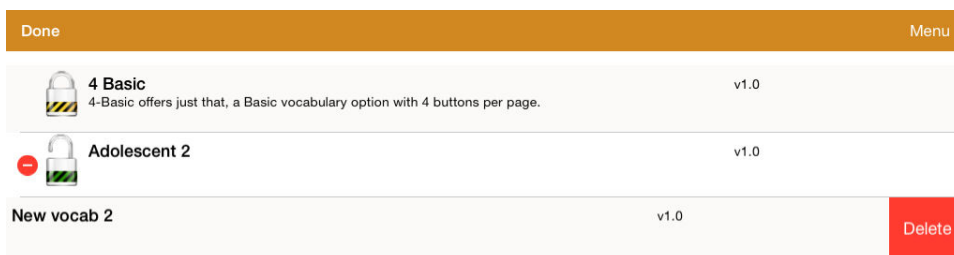6. Tap **Delete** and the vocabulary file will be deleted from TouchChat. Tap **Done**.

Backing up a Customized Vocabulary File

It is always a good idea to keep a copy of all custom files on a second computer for safekeeping. We suggest always using a wired connection to a PC to copy files consistently.

1. Connect the device to the computer.
2. Open iTunes on the computer.
3. On the device, tap **Vocab** and tap **Choose New Vocab**. Then tap **Menu**.
4. Tap **Import/Export Vocab**.
5. Tap **Export to iTunes**.
6. Choose the vocabulary file you want to back up.
7. Name the file and tap **Save**.
8. In iTunes on the computer, tap on the device and go to the Apps section.
9. Scroll down until you see TouchChat in the File Sharing area.
10. Highlight the vocabulary file that you are backing up.
11. Click **Save To** on the bottom right.
12. A Windows save box will appear. Select the folder in which you want to save the file.

Activating Edit Mode

You can only edit a copy of a vocabulary file, not the original. To make a copy, see [“Copying a Vocabulary File”](#) on page 30.

1. Go to the page you want to edit.
2. Tap **Menu**.
3. Tap **Edit Page**. The page is now in Edit Mode.

Using Chat Editor with TouchChat

Chat Editor is a supplemental program that runs on a Microsoft® Windows® desktop or laptop computer. It allows you to customize vocabulary files. Although customizing is also possible on the device itself, the Editor allows the family and/or professionals to customize the vocabulary when the device is not present.

Note: Having the custom vocabulary on a computer is always recommended as a backup to the device. See “[Copying a Vocabulary File from the Device to Chat Editor using iTunes](#)” on page 72.

Downloading and Installing Chat Editor

Important! With software release 2.0, we replaced TouchChat Windows Editor with Chat Editor. If you have used TouchChat Windows Editor in the past, it will no longer work with 2.0 files or newer.

The installation process installs Chat Editor on your computer and creates desktop shortcuts to the “Chat Editor” application and the “Chat Editor Import” folder.

Hint: To see a video that explains how to download and install Chat Editor, go to <http://touchchatapp.com/support/videos/touchchat-chat-editor-installation>

To download and install Chat Editor:

1. Go to <http://touchchatapp.com/support/software-updates>
2. Under “Chat Editor for Windows”, select the Download version link.
3. Save the downloaded Chat Editor_x.x.x_Setup.exe file to your desktop.
4. Double-click the Chat Editor_x.x.x_Setup.exe file to start the installation process.
5. Select the language you want to use for the installation and click **OK**.
6. Click **Next**, accept the license agreement, and click **Next**.
7. Click **Install**. When the installation process is complete, click **Finish**. You should see desktop shortcuts to the “Chat Editor” application and the “Chat Editor Import” folder.

Configuring Chat Editor

1. Double-click the “Chat Editor” desktop shortcut to open Chat Editor.
2. Select **TouchChat** from the Choose Product window and click **OK**.
3. Select your language from the Choose Language window and click **OK**.
4. Select one or more symbol sets from the Choose Symbol Sets window.

Important! This symbol set must match what is on the device you are supporting.

5. Click **OK**.
6. Type a descriptive name for the configuration in the Enter Name window and click **OK**.
7. Chat Editor will open.

Note: You may need to set up multiple configurations if you support more than one client. For example: Tommy with only SymbolStix® images and Sarah with both SymbolStix and PCS™ symbols.

Enabling Speech in Chat Editor

Note: When you install Chat Editor on a computer that had Chat Editor installed previously, speech will be enabled.

When you install Chat Editor on a computer that did not have Chat Editor installed previously, the following message will be displayed:

Speech is currently disabled.

Please connect a TouchChat device to this computer to enable speech.

Choose **OK** to close the message.

To enable speech in Chat Editor, simply connect a USB cable between your computer and your device. Test the Editor to verify that speech is now working and then disconnect the cable. Speech will now be enabled whenever you use the Editor; you will not need to re-connect the cable.

Opening Chat Editor

To open the Editor for normal use after you have set up a configuration, choose the Chat Editor shortcut from your computer's desktop. The Editor will open on your computer.

Using Chat Editor to Create Instructional Materials

Capture sequences of buttons from screens in Chat Editor to paste into instructional materials you create such as Word documents, PowerPoint presentations, flash cards, etc.

1. In Chat Editor, choose **Capture** in the top section of the Editor window. The Button Capture window will open.
2. With **Capture Selections** selected, start selecting the buttons you want to appear in your document. Each button you select will be captured in the Button Capture window.

3. When you finish capturing a sequence of buttons, choose **Copy to Clipboard**. The sequence will be cleared from the Button Capture window.

4. In your document, insert the cursor and select **Paste**. The sequence will be pasted into the document.
5. Capture the next sequence.
6. When you finish capturing sequences, select **Exit** to close the Button Capture window.

Notes on Capturing Button Sequences

If you need to try several sequences to find the one you want, turn off Capture Selections. Once you decide what you want to capture, turn Capture Selections back on.

If you make a mistake, select **Clear** and start capturing the sequence again. Keep in mind that Clear removes **all** captured content from the Button Capture window.

Working with Buttons


What a button speaks, how it appears, what icon is shown, what action the button performs, and the location of a button can be modified. All button modifications are performed in Edit Mode.

Editing a Button

1. Go to the page you want to edit and tap **Menu**.
2. Tap **Edit Page**. The page is now in Edit Mode.
3. Either tap or touch and hold the button you want to modify.

Note: If you touch and hold the button, keep your finger steady. If you move your finger, the button will start to move. See [“Moving a Button”](#) on page 42.

4. The Edit Button menu opens.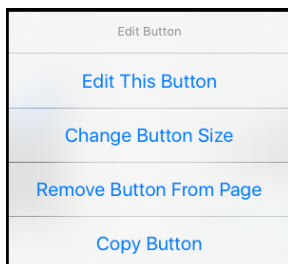

5. Tap **Edit This Button**. A window opens that will allow you to modify the button.
6. Tap the menu option you want to use.

Adding or Changing a Button Action

To add a button action:


1. Tap **Menu**, tap **Edit Page**, tap the button you want to modify, and tap **Edit This Button**.
2. Scroll down to the Button Actions section.
3. Tap **Add**.
4. Scroll through the list of actions and tap the action you want. It will be added below the existing action for the button.
5. Tap **Save** and **Done**.

To change a button action:

1. Tap **Menu**, tap **Edit Page**, tap the button you want to modify, and tap **Edit This Button**.
2. Scroll down to the Button Actions section.
3. Tap the existing action. The list of available actions will be displayed, with the existing action selected.
4. Tap the new action you want. The new action will replace the old action under Button Actions.
5. Tap **Save** and **Done**.

Removing or Changing the Order of an Action

1. While in the Button Actions area, tap **Edit**. The delete icon  will appear in front of each action linked to the button.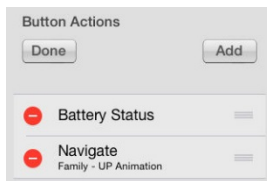

2. To remove an action, tap the delete icon  and then tap **Delete**. To rearrange the order of the actions, touch and hold the gray bars  to the right of an action and drag the action into the correct position.
3. Tap **Done**.
4. Tap **Save** and **Done**.

Modifying Text

Within the Edit window:

1. Tap in the Button Label area to change the label that will be displayed on the button.
2. Tap in the Button Message area to change the message that will be spoken.

Inserting Special Characters

Sometimes you may need to insert a special character in a word that you type, such as a letter with an accent; for example, **ó**.


On the keyboard, press and hold the applicable character key. A small menu of special characters available for that character will open above the key. Select the special character you want.

For example, to insert the special character **ó**, you would hold down the “**o**” key on the keyboard and select **ó** from the small menu.

Adding a Symbol from the SymbolStix Library

1. While in the Edit Button window, tap **Select Image**.
2. Tap the magnifying glass icon at the top right.
3. Type the name of the icon.
4. Tap **Search** on the keyboard.
5. Select the image you want.

Note: While editing a symbol, you can also choose to browse and find your icon by choosing **Select Image**, then tap a category.

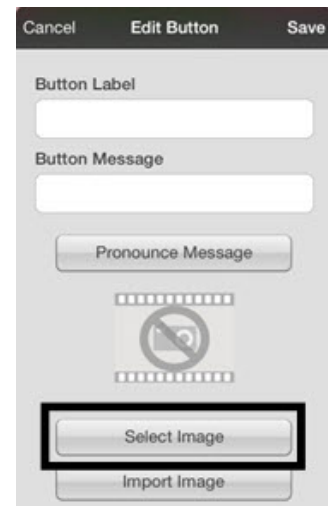Adding a Picture from the Image Library

1. Tap **Menu** and tap **Edit Page**.
2. Tap the button to which you want to add the picture.
3. Tap **Edit This Button**.
4. Tap **Import Image**. A small menu opens above Import Image.
5. Tap **Image Library**.
6. Tap **Camera Roll** or another available library.
7. Find the picture want and tap it.
8. Tap **Use** and give the button an appropriate label.
9. Tap **Save** and tap **Done**.

Using the Camera to Take a Picture for Your Button

1. While in the Edit Button window, tap **Import Image**.
2. Tap **Camera**.
3. Take your picture by pressing the camera icon.
4. Tap **Use Photo**.
5. Tap **Save** and **Done**.

Creating a Camera Button

You can create a button that allows you to take a picture that will be imported to the button automatically every time you tap the button. The picture will also be added to your library of imported images. Each time you press the button, a new image from the camera will replace the existing image on the button. If the old image on the button was imported and not used elsewhere, it will be deleted. An image will appear on the button until you press the button again.

To set up the button action:

1. Go to the page with the button location you want.
2. Tap **Menu > Edit Page**.
3. Press and hold the button you want to use.
4. Tap **Edit This Button**.
5. Add a label and message.
6. Scroll down to “Button Actions” and tap **Add** to open the list of actions.
7. Scroll down the list and tap **Take a Photo**.
8. The Take a Photo action will be added to the button’s actions.
9. Tap **Save** and then tap **Done**.

To take a photo for your button:

1. Tap the new button to activate the camera.
2. Take a picture of the object or person.
3. Tap **Use Photo**.
4. The photo will appear as the image on the button.

To delete photos:



To delete photos, you must connect your device to your computer and remove the photos using Explorer from the computer.

Resizing a Button

1. Tap **Menu** and tap **Edit Page**.
2. Tap the button you want to resize.
3. Tap **Change Button Size**.
4. Select the number of rows and columns you want.
5. Tap **Save**.

Changing a Button Style

1. While still in the Edit Button window, scroll down to the area you want to edit.
2. Choose to edit the font, size, color, and border and make those changes.
3. Tap **Save**.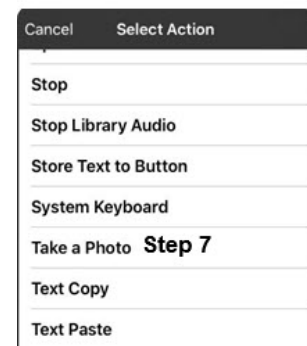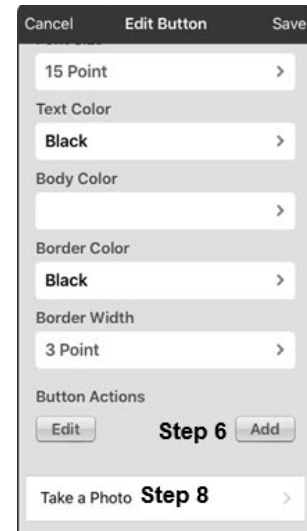

Creating a Button to Link to a Page

When a button action is set to Navigate or Visit you can determine what page that button will take you to. Navigate will take you to a New Page and leave you there requiring an additional button to be hit to return to your previous page or to the home page. Visit will take you there for one selection and immediately return to the previous page. The following linking button actions are available.

Action	Description
Navigate	Links to a new page and requires another button press to move away from this page
Visit	Links to a page for a single hit and then automatically goes back to the previous page
Navigate Back	Returns to the previous page
Navigate to Home	Links to the Home page of the vocabulary file


To create a linking button:

1. Turn Edit Mode on.
2. Tap the button to be linked to the next page.
3. Tap **Edit This Button**.
4. Enter the label and message. (Entering a symbol is optional.)
5. Under the Button Actions, tap **Add**.
6. Select a navigation action for the button from the drop-down list (Navigate, Visit, Navigate Back, Navigate to Home). Navigate and Visit offer a listing of pages to link to. If you choose Navigate Back or Navigate to Home, skip to step 8.
7. Choose the page to navigate to or visit from the listing. If the page does not already exist, choose + and create a new page to link the button to.
8. Tap an animation (how the page will appear as it navigates), or tap **No Animation**.
9. Tap **Save**.

Note: There may have been a previous action on this button. To remove that action, tap **Edit**, tap the delete icon , tap **Delete**, and tap **Done**.

Storing Text on a Button

1. Go to the page that includes the button on which you want to store text.
2. Tap **Menu** and then tap **Edit Page**.
3. Tap the button on which you want to store text.
4. Tap **Edit This Button**.
5. Name the button, and optionally create a message and modify the style.
6. Scroll down to the Button Actions section and tap **Add**.
7. Scroll down and tap **Store Text to Button**. The action will be displayed in the Button Actions section.
8. Tap **Save** and then tap **Done**.

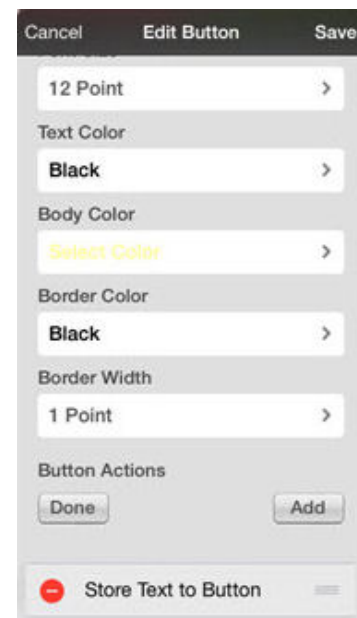Note: Once a sentence or phrase has been constructed, it can now be saved on a button by navigating to the location of the Store Text button that you created. Tap the button, tap **Continue**, tap the button in which you want to store the text, and then tap **Okay**. The text will appear in the button you selected, and each time you tap that button it will speak that saved text.

Creating a Volume Control Button

Excessive volume or sound pressure level when using earphones or headphones can cause hearing loss over extended periods of time.

You can add volume controls to buttons within vocabulary pages.

1. In TouchChat, tap **Menu > Edit Page**.
2. Go to the page on which you want to create the button.
3. Press and hold on the button you want to use. A list of button options appears.
4. Choose **Edit This Button**.
5. Add a button label and icon.
6. Under Button Actions, choose **Add**. A list of actions appears.
7. Choose either **Volume Up** or **Volume Down**.
8. Choose **Save**.

Creating a Battery Status Button

You can create a button that displays the battery status of both the device and the amplifier in the Speech Display Bar (SDB).

1. In TouchChat, tap **Menu > Edit Page**.
2. Go to the page on which you want to create the button.
3. Press and hold on the button you want to use. A list of button options appears.


4. Choose **Edit This Button**.
5. Add a button label and icon.
6. Under Button Actions, choose **Add**. A list of actions appears.
7. Choose **Battery Status**.
8. Choose **Save**.

Note: To view the status of the amplifier battery, you must have ChatWrap Connect installed. If ChatWrap Connect is not installed, refer to page 2 of *Installing and Setting up TouchChat on an iOS Device*, which was included in your TouchChat Express package.


Copying and Pasting a Button

Buttons can be copied and pasted to new locations. If you modify a copy of a button, you will edit only the copy.

1. Tap **Menu** and tap **Edit Page**.
2. Tap the button you would like to copy. The Edit Button menu opens.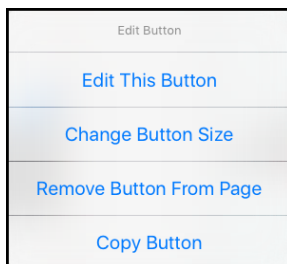

3. Tap **Copy Button**.
4. Tap **Done**.
5. Go to the page where you want to add the copied button.
6. Tap **Menu** and tap **Edit Page**.
7. Tap the location you wish to copy the button to. The Edit Button menu opens with additional options.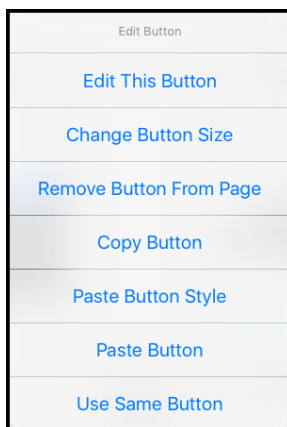

8. Tap **Paste Button**. The button you copied will appear in that location.
9. Tap **Done**.

Copying and Pasting the Same Button

If you modify a button that is used in multiple locations, the button will also be modified automatically everywhere else it appears. You may want to use this technique on buttons that contain tools used on each page.

1. Tap **Menu** and tap **Edit Page**.
2. Tap the button you would like to copy. The Edit Button menu opens.
3. Tap **Copy Button**.
4. Tap **Done**.
5. Go to the page where you want to add the copied button.
6. Tap **Menu** and tap **Edit Page**.
7. Tap the location you wish to copy the button to. The Edit Button menu opens with additional options.
8. Tap **Use Same Button**. The button you copied will appear in that location.
9. Tap **Done**.

Copying and Pasting a Button Style


“Button Style” refers to the background color and the font size and options that are associated with a button. There are times when it might be faster to copy and paste a button’s style from an existing button rather than editing each of these items on a new button.

1. Tap **Menu** and tap **Edit Page**.
2. Tap the button whose style you want to copy. The Edit Button menu opens.
3. Tap **Copy Button**.
4. Tap the button you want to paste the style to. The Edit Button menu opens with additional options.
5. Tap **Paste Button Style**. The button will display the pasted style.
6. Tap **Done**.

Moving a Button

1. Tap **Menu** and tap **Edit Page**.
2. Touch the button you want to move and drag it to another location.

Note: When you start to drag the button, the original location will be outlined in red. As you move into the new location, the red outline will move to that location.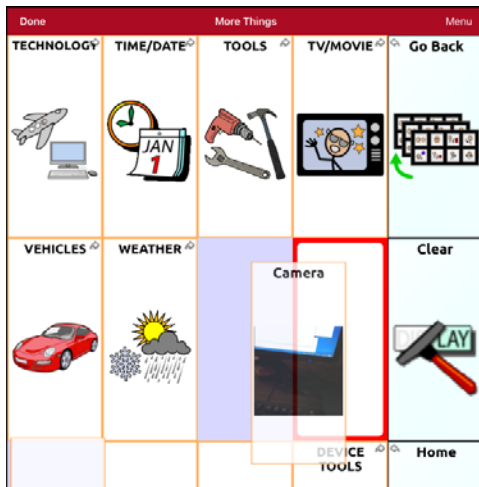


3. Upon release, the button will move to the new location. If there is already a button in the new location, it will move to the old location. The buttons will exchange locations.

Hiding a Button

There are times that presenting all buttons at once might be intimidating, or in order to complete a task a button might need to be hidden.

1. Tap **Menu** and tap **Edit Page**.
2. Tap the button you want to hide and tap **Edit This Button**.
3. Turn **Visible** off.
4. Tap **Save** and **Done**.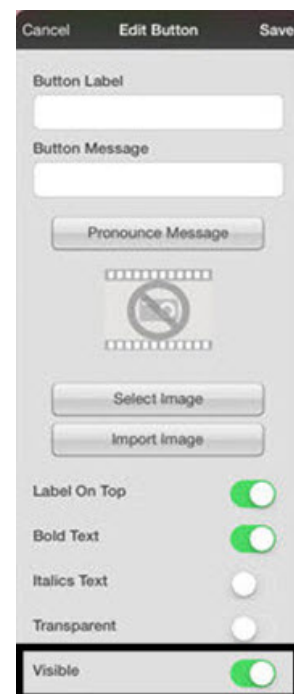

Hiding or Showing Multiple Buttons at Once

1. Tap **Menu** and tap **Edit Page**.
2. Tap **Menu** and tap **Hide Mode**.
3. Tap the buttons you want to hide or show.
4. Tap **Menu** and tap **Normal Mode**.

Changing Button Timing

Tap **Menu** and tap **Settings**. Scroll down to the “Buttons” section.

Activate on Release


This option offers to have buttons activated when released rather than upon touch. This allows the device user to slide his or her finger across the screen before finally releasing on the button that he or she wants to select. Turn the toggle switch to the **On** position.

Dwell Time


The dwell time setting controls the delay before a button press is recognized. When the user touches a button, a slight dimming and a white square box identifies the chosen button. As long as the user continues to keep his or her finger inside the button area the dwell timer counts down. Once the time is reached, the button press is accepted. To set a dwell time, tap **Dwell Time** and select a value from 0.1 Seconds to 5.0 Seconds.

Release Time

Release time is intended to help prevent accidental double button activations. When the time is set, all buttons are disabled for the specified period of time after each button press event. To set a release time, tap **Release Time** and select a value from 0.1 Seconds to 5.0 Seconds.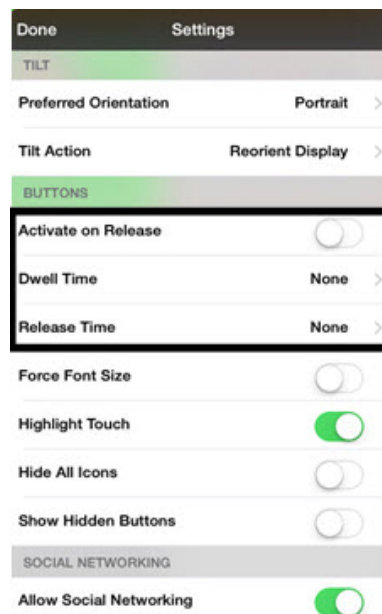


Hiding and Showing Button Navigation Arrows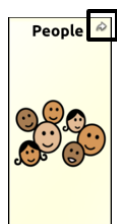

Buttons that navigate to new pages display a small arrow in the top right corner.

To hide or show all navigation arrows:

1. Choose **Menu > Settings**.
2. Under Navigation, tap **Button Navigation Icons**.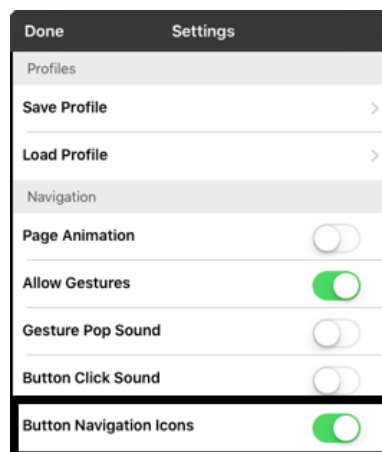

Working with Pages


TouchChat allows you to rename, duplicate, load, and create pages, choose a different keyboard, copy a page from a different vocabulary, and add gestures to pages.

Renaming a Page

1. Tap **Menu** and tap **Edit Page**.
2. Tap **Menu** and tap **Edit Page Layout**.
3. In the area under Page Name, tap, then backspace to erase the name, and then type the new name.
4. Tap **Save**.

Duplicating a Page


1. Tap **Menu** and tap **Edit Page**.
2. Tap **Menu** and tap **Import/Copy Pages**.
3. Tap **Duplicate This Page**. A duplicate of the page, followed by “(Copy)”, will appear in the list of pages in this vocabulary.
4. Tap **Done**.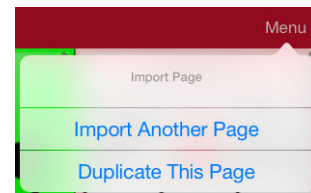

Loading Another Page


1. Tap **Menu** and tap **Edit Page**.
2. Tap **Menu** and tap **Load Another Page**.
3. Scroll up to find the page you want to load.
4. Tap **Done**.

Note: This page is now available to copy buttons or make additions or changes to your vocabulary set.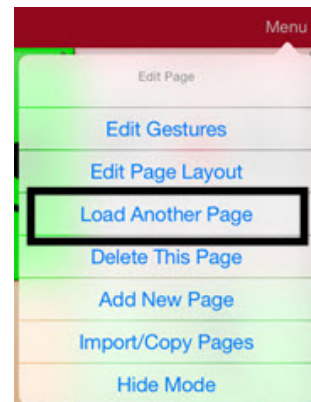


Creating a New Page

1. Tap **Menu** and tap **Edit Page**.
2. Tap **Menu** and tap **Add New Page**.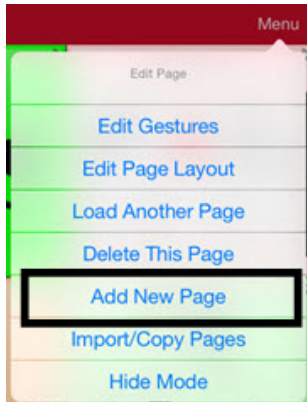

3. Give the page a name.
4. Select how many rows and columns you would like.
5. If you want this new page to be the Home page, tap **Make Home Page** to turn the option on.
6. Choose a background color.
7. Select an image (*optional*).
8. Tap **Save**. A new page will appear with blank buttons.
9. Start creating buttons on this page.
10. Tap **Done** when you are finished.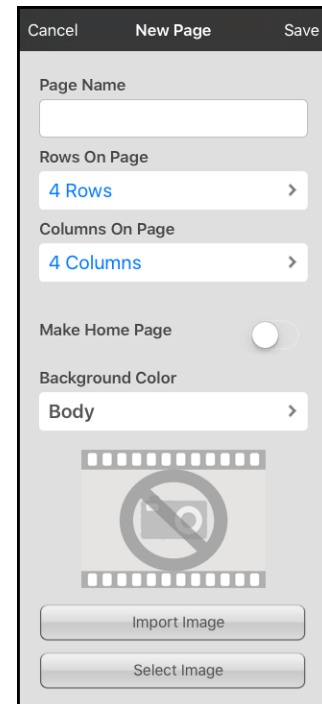

Creating a New Page from a Navigate Action

1. Display the page from which you want to navigate.
2. Tap **Menu** and tap **Edit Page**.
3. Tap the location for a button that will navigate to the page you want to create.
4. Tap **Edit This Button**. The Edit Button dialog opens.
5. Enter a button label and select or import an image.
6. Scroll down to the Button Actions section and tap the existing action to open the list of available actions.
7. Select the **Navigate** action to open the list of existing pages.
8. Tap the **+** in the upper right corner to create the new page.
9. Enter a page name and adjust the number of rows and columns.
10. Tap **Save**, select **No Animation**, and tap **Save** again.
11. To verify that the button navigates to your new page, tap **Done** to exit Edit Mode and then tap the button to navigate to the new page.
12. Tap **Menu**, tap **Edit Page**, and add buttons to the page.


Choosing a New Keyboard

1. Tap **Menu** and tap **Edit Page**.
2. Tap **Menu** and tap **Import/Copy Pages**.
3. Tap **Import Another Page**.
4. Tap the vocabulary file that contains the keyboard you want and tap the appropriate page to import.
Note: WordPower files have alternative pages (for example, QWERTY vs. ABC) within the same file. Simply edit the linking button to navigate to the new page **or** follow steps 6, 7, and 8.
5. Tap **Import**.
6. Edit the original keyboard button and have it navigate to the new keyboard page.
7. Edit the linking button to navigate to the new page.

Copying a Page from a Different Vocabulary File

There are times that borrowing previously created pages could be a time saver. TouchChat allows you to import multiple pages at one time.

1. Have your vocabulary file open.
2. Tap **Menu** and tap **Edit Page**.
3. Tap **Menu** and tap **Import/Copy Pages**.
4. Tap **Import Another Page**.
5. Tap the vocabulary file that contains the page you want and tap the appropriate pages to import. A checkmark will appear following each page you tap.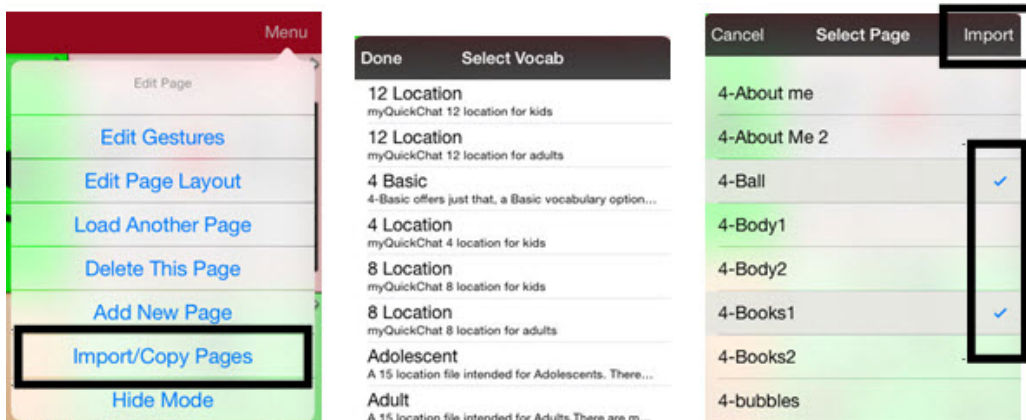


6. Tap **Import** and tap **Done**.
7. While still in Edit Mode, tap the button that you want to navigate to the imported pages.
8. Tap **Edit This Button**.
9. Give the button a label and, if needed, an icon and make any style changes you want.
10. Scroll down to Button Actions, tap **Add**, and then tap **Navigate**.
11. Tap the page you imported.
12. Tap an animation or **No Animation**.
13. Tap **Save** and tap **Done**.

Adding a Gesture to a Page

1. Tap **Menu** and tap **Edit Page**.
2. Tap **Menu** and tap **Edit Gestures**.
3. Tap **+**.
4. Add a label and message.
5. Tap the gesture type and tap the type you want. A check mark will appear following the type you selected
6. Tap a gesture action or tap **Add** to add an action.
7. To add gestures to an entire vocabulary file at one time, tap **Apply to All Pages**.

Note: To delete a gesture action, tap **Edit** and tap the action you want to delete. The delete icon  will be displayed in front of the action. Tap the delete icon  to delete the action.

8. Tap **Save** and tap **Done**.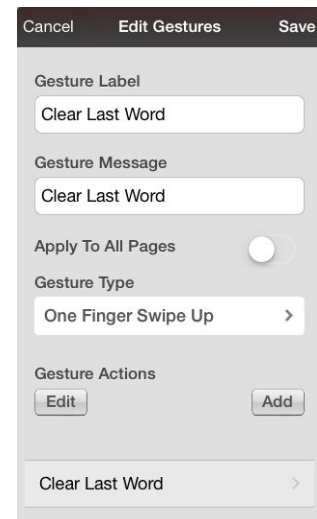

Creating a Template Page

You can create a template page in Chat Editor. This feature is handy when building vocabulary files with the same configurations of buttons to be displayed on each page. Create a page with the buttons that will be consistent on each page. Set this page as a template to be used when creating new pages.

1. Turn Edit Mode on.
2. Create a page to use as the template.
3. Click **Pages**.
4. Choose the page you would like to duplicate.
5. Choose **Save as Template**.

Using a Template to Create a Page

You can create a page from a template in Chat Editor.

1. Turn Edit Mode on.
2. Select **Pages > Templates > New Page from Template**.
3. Select a template category.
4. Select a specific template. When you select the template, a preview will be displayed.
5. Select **Choose** and type a descriptive name for the new page.
6. Select **Save**.

Editing an Abbreviation Expansion

1. Tap **Menu** and tap **Edit Abbreviations**.
2. Tap **+**.
3. In the space under Abbreviation, type the abbreviated form.
4. In the space under Expands to, type the full word, phrase, or sentence that it will expand to.
5. Tap **Save** and tap **Done**.

Note: When typing the abbreviation, type a period after the abbreviation to activate the action.

Editing Pronunciations

1. Tap **Menu** and tap **Edit Pronunciations**.
2. Tap **+**.
3. In the space under Word, type the correct spelling.
4. In the space under Pronounce As, type word phonetically or the way it should be pronounced. Tap **Pronounce Word** to determine if it is pronounced correctly. If not, change the spelling until the word is pronounced correctly.
5. Tap **Save** and tap **Done**.

Note: If the word already exists in the Pronunciation list, scroll up to find the word and then tap the word to edit to the correct pronunciation.

Adding Acapela Vocal Smileys

Acapela-Group has created the capability of using Voice Sounds and Exclamations.

Adding a Sound

Sounds are produced by the speaker's voice for laughing, breathing, sneezing, coughing, or whatever sounds our voices can produce to mimic sounds we make in our daily lives. Sounds are always between two pound signs; for example: **#LAUGH02#** in capital letters and sometimes followed by numbers if there are more than one of the same kind. The children's voices have more sounds than adult voices.

Adding an Exclamation

Exclamations include a variety of letters and symbols. Type exactly what you see.

Finding a Vocal Smiley

To find the list of Acapela Vocal Smiley commands:

1. Go to <http://www.acapela-group.com/doc/Vocal%20smileys/excla.html>
2. Click the link to the appropriate language.

Important! The language you select must match the language of the voice you are using.


3. Select the specific Vocal Smiley you want.

Example: Add a Smiley Face and Laugh Sound to a Button

A button can be set up with the command in the message. However, if you use the “speech message” action, the command appears in the Speech Display Bar (SDB), which isn't ideal. A better way may be to set up a button with an emoticon symbol in the message.

For example, you could use a smiley face symbol. Note that you may need to go to **Settings > General > Keyboards** and add the Emoji keyboard so that you can type emoticon symbols. Once the button is set up with the emoticon and speech message, go to Pronunciations and add a pronunciation for the smiley face emoticon. Enter a command such as **#LAUGH02#** for the pronunciation.


The Pronunciation Editor can be found in **Menu > Edit Pronunciations > +**. Add the same Emoticon used on the button to the Word box and add the command—for example, **#LAUGH02#**—to the Pronounce As box.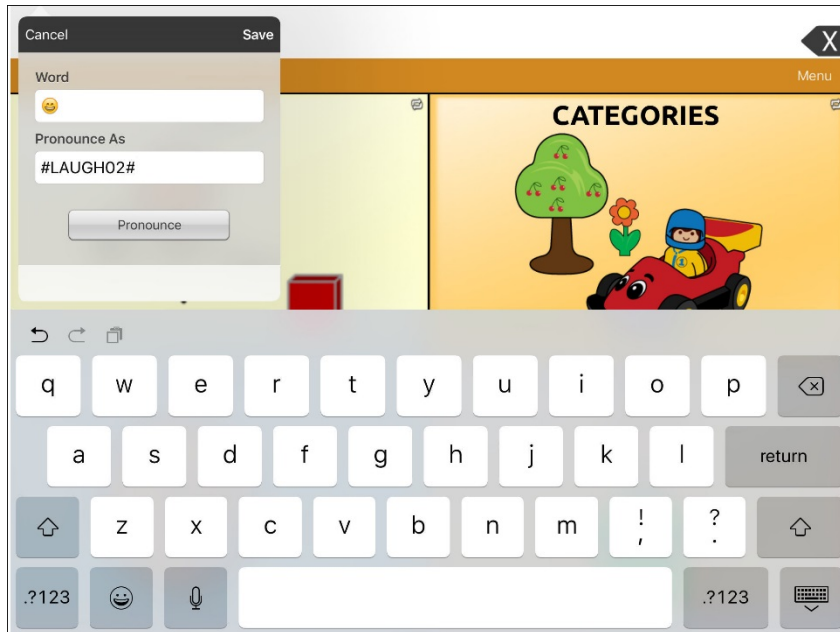


Now when you touch the button, the emoticon will appear in the SDB and the laugh sound will be heard.

Setting up Your VocaliD™ Voice

1. Enable a Wi-Fi connection.
2. Choose **Menu > Settings**.
3. Scroll to the Speech section and tap **Install VocaliD Voice**. A VocaliD login screen is displayed.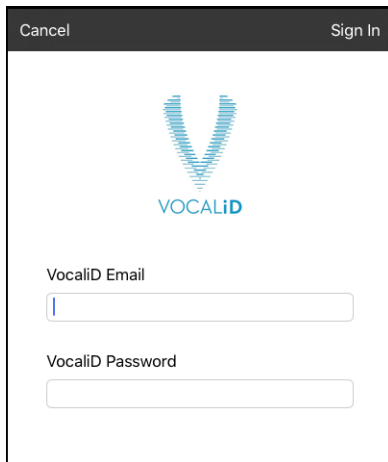

4. Enter your VocaliD email address and VocaliD password.
5. Tap **Sign In** in the upper right corner to display a Download Voice prompt.
6. Tap **Download Voice**. "Downloading Voice" will be displayed.
7. When the download is complete, "New Voice Installed" will be displayed.
8. Tap **Okay**.
9. Tap **Done**. The downloaded VocaliD voice is now your primary voice.
10. Try out the voice. You can change the voice rate and pitch in Settings.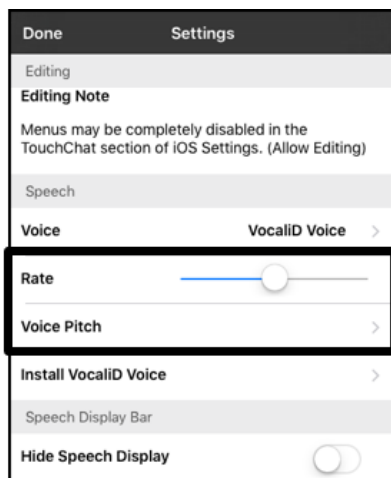

Voice Rate and Pitch settings

To change the rate: Move the slide bar.

To change the pitch: Tap **Voice Pitch** to open a list of pitch levels. Select the pitch you want: Lowest, Lower, Low, Normal, High, Higher, or Highest.

11. Test the voice again and adjust the rate and pitch as needed.

To switch to a different voice:

Any time you want to switch to a different voice, do the following:

1. Choose **Menu > Settings**.
2. Scroll to the Speech section. The Voice setting shows **VocaliD Voice**.
3. Tap **Voice** to open a list of available voices.
4. Tap or download the voice you want and tap **Done**.

Creating a Visual Scene

To create a visual scene, select a background image, set up the page, and add buttons.

Selecting a Background Image


Start by having the picture you want to use for the background image on the device. You can take a picture using your device if it has a built-in camera. Otherwise, transfer the picture to the device from a computer.

1. Tap **Menu** and tap **Edit Page**.
2. Tap the location to add a linking button (the button you wish to link to your visual scene) and tap **Edit This Button**.
3. Add a label, scroll down to the Button Actions section, and replace the existing action with **Navigate**.
4. Tap the **+** in the upper right corner to create your new visual scene page.
5. Enter a page name.
6. Adjust the number of rows and columns on the page. Choosing as many rows and columns as possible will make it easier to create buttons that fit with the images on the page.
7. Scroll down to **Import Image**.
8. Tap **Import Image** and take a picture or find the image you want. A thumbnail view of the image will appear above Import Image.
9. Tap **Save**, select **No Animation**, and tap **Save** again.
10. To verify that the button navigates to your new page, tap **Done** to exit Edit Mode.
11. Tap the button to navigate to the new page.
12. Tap **Menu** and tap **Edit Page**, and proceed to the next topic, [“Adding Buttons”](#).

Adding Buttons

Now with the background picture in place, using the blank buttons, buttons can be created that speak or move to other pages. In our example, create a button that speaks “Horse” whenever tapping on the horse in the picture (assuming we used a picture with a horse in it).

1. Tap the upper most left corner location that covers the ear.
2. Tap **Create New Button**.
3. Type **Horse** into the button message box.
4. Set the Transparent option to the **On** position. That way the scene behind it comes through.
5. Tap **Save**.
6. Tap the newly created button and choose **Change Button Size**. From here, you can adjust the button height and width so that it fits the entire image.
7. Tap **Save**.
8. Create the remaining buttons and re-size them to go along with the scene. All the buttons can have their own size and shape.
9. When you finish, tap **Done**.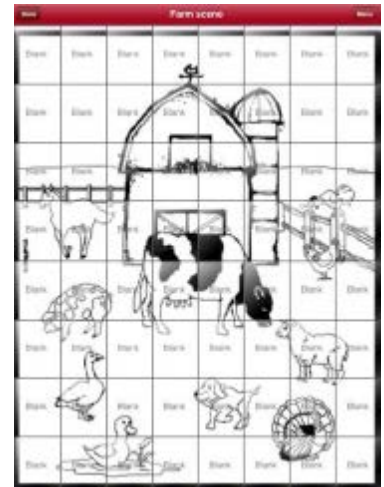

Enlarging Text in the Speech Display Bar

There are a number of ways to enlarge text in the Speech Display Bar (SDB).

Changing Text Size or Number of Lines in the SDB

1. In TouchChat, tap **Menu** and tap **Settings**.
2. Scroll to the Speech Display bar section.
3. Under Font Size, choose a larger font. Or, under Number of Lines, select the number of lines of text you want to appear in the Speech Display Bar.


Expanding the SDB by Changing Screen Orientation

You can set TouchChat to expand the Speech Display Bar (SDB) when the device is rotated between portrait and landscape.

Expand and Speak: When you rotate the device, text in the Speech Display Bar (SDB) will expand to fill the screen and the device will speak it.

Expand Only: When you rotate the device, text in the Speech Display Bar (SDB) will expand to fill the screen, but the device will not speak it.

1. In TouchChat, tap **Menu** and tap **Settings**.
2. Scroll down to the Tilt section.
3. Tap **Tilt Action** and tap either **Expand and Speak** or **Expand Only**.
4. Tap **Done**.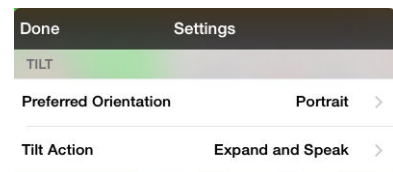

Creating a Tap Action

Create a tap action to enlarge the text in the SDB.

1. In TouchChat, tap **Menu** and tap **Settings**.
2. Scroll down to the Speech Display Bar section.
3. Under Tap Action, tap **Speak and Enlarge Text**.

Setting up Switch Scanning in TouchChat

As an alternative to directly selecting each button on the screen, the application offers single switch and two-switch scanning. To access the options for customizing a scanning technique, choose **Menu > Settings > Scanning**.

Important! For the scanning settings to be available, your amplifier must be turned on, charged, and communicating with the device. The amplifier light should be solid blue.

Setting	Description
Scanning Enabled	Turns Scanning on or off. You must turn scanning on to access the other scanning settings.
Auto-Scan	When Auto-Scan is turned on, scanning activates automatically. With Auto Scan off, a switch activation is required to move the scan through each button, one at a time.
Button Hold Scan	When Button Hold Scan is turned on, holding down the switch will initiate and continue a scan. Releasing the switch makes an activation.
Auto Restart	When Auto-Scan and Auto Restart are both turned on, a scan cycle restarts automatically after an activation.
Use Screen As Switch	Allows you to use the touchscreen as a switch.
Auditory Scan	Turns on audio feedback during scanning.
Use Headphone For Prompt	When Use Headphone For Prompt is turned on, you will hear prompts through a headphone
Prompt Voice	Allows you to choose the prompt voice.
Prompt Voice Rate	Provides a slider that allows you to set the Prompt Voice rate. Move the slider to the right for a higher voice rate or to the left to lower the rate.
Prompt Voice Pitch	Sets the pitch of the prompt voice. Available settings are Lowest, Lower, Low, Normal, High, Higher, and Highest.
Pause Between Words (Auditory Scan)	When Auditory Scan is turned on, Pause Between Words sets the amount of time between words in auditory prompts. Available settings are No Pause, 1 Second, 2 Seconds, and 3 Seconds.
Scan Speech Display Bar	When Scan Speech Display Bar is turned on, a scan will include the Speech Display Bar. When turned off, a scan will skip the Speech Display Bar.
Skip Empty Cells	When Skip Empty Cells is turned on, a scan will skip empty button areas. When turned off, a scan will include empty button areas.
Scanning Type	Sets the scanning pattern to Sequential or Row/Column.
Manual Repeat Time	Sets the amount of time a scan takes to move from one button to another while you are holding down the switch. Releasing the switch within the time you set will make an activation. Available settings range from 0.2 Seconds to 10 Seconds. <i>Note:</i> Manual Repeat Time is only available when Auto-Scan is turned off and Button Hold Scan is turned on.
Scan Cancel Time	Determines how long a scan will sit idle before it “cancels” the scan and starts over from the beginning. Scan Cancel Time is only useful when you have two switches connected to your device. Available settings range from 5 Seconds to 60 Seconds. If you don’t want to set an idle time, select Off. <i>Note:</i> Scan Cancel Time is only available when Auto-Scan is turned off.
Number of Switches	Sets the number of switches to be used: 1 Switch or 2 Switches.

Configuring Switches

To use an external switch, plug the switch into either switch jack A or switch jack B on the side of the device. You can choose to have scanning activate automatically, or you can choose to either hold the switch or step to each location. With Auto Scan off, a switch activation is required to move the scan through each button, one at a time.

1. To enable scanning, choose **Menu > Settings > Scanning > Scanning Enabled**.
2. Choose **Number of Switches** and select **1 Switch** or **2 Switches**.
3. To set scanning to activate without holding a switch, choose **Auto-Scan**.

Setting up Touchscreen Scanning

There are times when a user is not capable of activating specific areas on the screen. For example, maybe the user does not have the pointing skills to hit one button. Using the touchscreen to scan allows the user to tap anywhere on the screen to initiate a scan of the buttons. In this scenario, the device screen acts as a single switch.

1. Choose **Menu > Settings > Scanning > Enable Scanning**.
2. Choose **Use Screen As Switch**.
3. Configure other settings as appropriate.

Selecting the Scan Pattern

To select the scanning mode:

1. Choose **Menu > Settings > Scanning > Scanning Type**.
2. Choose Sequential or Row/Column

Using Sequential Scan Mode

A sequential scan steps through the buttons from left to right beginning in the first row. When the button you want is highlighted, activate your switch.

Using Row/Column Scan Mode

A row/column scan highlights each row of buttons beginning with the top row and move down the screen. When the row that contains the button you want is highlighted, activate your switch. The scan will then step through the buttons from left to right across the row. When the button you want is highlighted, activate your switch.

Configuring Speech Display Bar (SDB) Scans

To include the SDB in scans:

Selecting the SDB speaks the current text. Turn scanning of the Speech Display Bar on or off by choosing **Menu > Settings > Scanning > Scan Speech Display Bar**.

Configuring Empty Cells Scans

Turn scanning of empty button areas on or off by choosing **Menu > Settings > Scanning > Skip Empty Cells**. When selected, scanning will skip over an area without a button or a button that only has a speech message action but no text.

Setting Scan Timing

Scan speed is the amount of time it takes for your device to move from one button, row, or column to the next in a scan. The default scan speed is one second. At this setting, it takes one second from the time one key is highlighted until the next key is highlighted. To change the scan speed, choose **Menu > Settings > Scanning > Scan Speed**. Choose the appropriate time.

Configuring Re-scans

Select the number of times you want your device to scan the page automatically. Once you reach the set number of re-scans, the scan will stop until a switch is activated again. To set the number of re-scans, choose **Menu > Settings > Scanning > Number of Rescans**. The Number of Rescans window opens. Choose the appropriate number of rescans.

Turning on Auto Restart

This option allows you to restart a scan cycle automatically after an activation. To turn on auto restart, choose **Menu > Settings > Scanning > Auto Restart**.

Setting Auditory Scan

This option enables audio feedback during scanning. To set auditory prompts, choose **Menu > Settings > Scanning > Auditory Scan**. If you are scanning, simply activate your switch. To retrieve a message, activate the switch again after you hear the prompt.

Audio Output Device

Choose **Menu > Settings > Scanning > Use Headphone for Prompt**. You can hear prompts through the device's speaker. You can also plug stereo headphones into the headphone jack on the side of the device. This allows you, but not others, to hear the prompts. If you are using headphones, turn On User Headphone For Prompt.

Choosing a Voice

When choosing a voice to use for the auditory prompts, it can be helpful to choose a different voice than the one the system uses for speaking messages. To change the voice, choose **Menu > Settings > Scanning > Prompt Voice**. You can also adjust **Prompt Voice Rate** and **Prompt Voice Pitch** as needed.

Making an In-App Purchase

Saltillo provides additional voices, other language packages, unique vocabulary files, support packages, and curriculum support in the Upgrade Store. Apple provides an option for these materials to be purchased from within the TouchChat application.

To explore the Upgrade Store:

1. Enable a wireless connection.
2. Tap **Vocab** and tap **Choose New Vocab**.
3. Tap **Menu** and tap **Upgrade Store**.
4. Tap **Enter Upgrade Store**.

To make a purchase:

In the Upgrade Store, make a selection and enter your iTunes purchase information.

NCMEC First Responder Event

This free in-app download consists of TouchChat pages to support the project by The National Center for Missing and Exploited Children, entitled “How to Host a Sensory Friendly First Responder Event for Children with Autism”. None of the activities has a link to a Home page, so the Back button in TouchChat must be used to move from one activity to another. The idea is that the facilitator will set the page for a specific activity, and then the child will stay on that page. The facilitator will use the Back button to navigate back to the Home page. By design, some of the pages don’t speak. They are intended to allow the child to show, rather than tell, the facilitator how he or she is feeling.


Playing a Song from TouchChat

Audio files such as music contained in the device user’s iPod® media library can be played.

1. Edit a button and tap the action **Play Library Audio**.
2. Choose the song from your music Library.
3. Give your button a label and add a picture.

Note: You will also want to include a Stop button on your page in case you need to stop the song before it is finished playing. To create a Stop button:


4. Choose the location, edit the button, add a label and icon, and tap the action **Stop Library Audio**.
5. If Speech Message remains in the Button Actions section, tap **Edit** and click the delete icon  in front of Speech Message.
6. Tap **Save** and tap **Done**.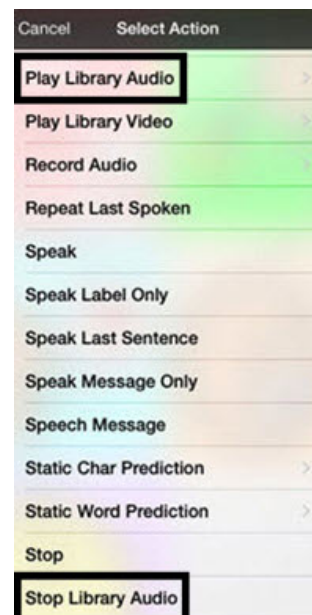

Using SocialChat

Text generated with TouchChat pages can be shared with social media, such as Facebook, Tumblr, and Twitter accounts.

1. Make sure a wireless network connection is established and accounts for the preferred social network are set up.
2. Go into the TouchChat app; generate the text you wish to post to the social network.
3. Press and hold on the Speech Display Bar (SDB) until the Share and Copy options appear.
4. Tap **Share** and tap the social network you wish to send the text to.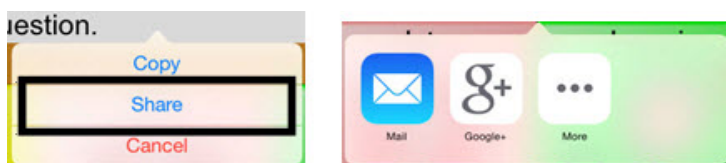

Set up a Button to Play a YouTube™ Video


Note: You must have an active Wi-Fi connection.

1. On your device, select the YouTube video you want.
2. Tap the **Share** icon and then tap the **Copy Link** icon. A “Copied” message should appear.

Note: If you have an older iOS version, tap the Share link and choose **Copy**.


3. In TouchChat, display the page that has the button you want to use to play the YouTube video.
4. Choose **Menu > Edit Page**.
5. Tap the button you want to use to use and choose **Edit This Button**.
6. Give the button a label and scroll down to the Button Actions section.
7. Select the existing action to open the Select Actions list.
8. Select the action **Play Video**. The Video Type window will display two options.
9. Select **YouTube Video**. The YouTube window will display a Video Link field.
10. Press and hold on the Video Link field. A small Paste prompt will appear.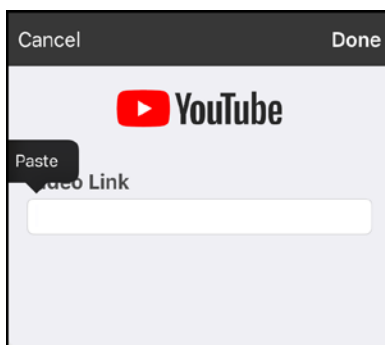

11. Tap the **Paste** prompt. The link you copied in step 2 will be pasted into the field.
12. Select **Done**. Play Video will be displayed under Button Actions.
13. Select **Save**. The label you entered will be displayed on the button.
14. Select **Done**. When you tap the button, the YouTube video should play.
 - If you do not have a Wi-Fi connection when you attempt to play the YouTube video, the display area will be black and the following message will appear:
No Connection to YouTube
An internet connection is required to play a YouTube video.
 - If there is an internet connection but YouTube cannot be reached, the following message will appear:
No Connection to YouTube
YouTube appears to be blocked by your internet service provider. Please contact your network administrator.

Blocking and Restoring Access to the Editing Menus

There may be times when you need to block access to the Editing menus. To do this, the **Use Menu Passcode** setting can be turned on or off. When this setting is turned on, the Menu and Vocab items can only be accessed by entering a password.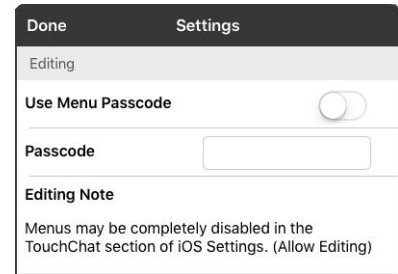

To password-protect the Editing menus:

1. In TouchChat, tap **Menu** and tap **Settings**. The Settings menu will open.
2. Under “Editing”, turn **Use Menu Passcode** on.
3. Tap in the Passcode field to open a keyboard.
4. Enter your password and tap **Done** on the keyboard.
5. Tap **Done** on the Settings menu.

Now when the user taps either **Menu** or **Vocab**, a password prompt will be displayed.

To reset your password:

If you forget your password, enter the numeric sequence **72584556** to overwrite the existing password. Then you can create a new password.

To remove password protection from the Editing menus:

1. In TouchChat, tap **Menu** or **Vocab**. The Password prompt will be displayed.
2. Enter your password. The Settings menu will open.
3. Turn **Use Menu Passcode** off.
4. To remove the password, tap in the Passcode field to open the keyboard. Use the Keyboard backspace key  to remove the password.
5. Tap **Done** on the keyboard.
6. Tap **Done** on the Settings menu.

Alternative: Allow Editing in iOS Settings

You can hide the editing menus completely by turning off **Allow Editing** in the iOS settings.

- When Allow Editing is turned off, **Menu** and **Vocab** are completely hidden.
- When Allow Editing is turned on, Menu and Vocab are accessible. You can then use the TouchChat settings to enable or disable access.


To change the Allow Editing setting, tap the **Home** button on the device and choose **Settings > TouchChat > Allow Editing**.

Working with Profiles

Profiles provide a snapshot of the current settings of the system to be saved for easy access at a later time. You can save multiple profiles to be used when needed. Creating multiple profiles allows you to switch between groups of settings.

Create and Save a Profile

1. Configure the settings you want associated with the profile. This includes the vocabulary file, settings, language, etc.
2. Go to **Menu > Settings > Save Profile**. The Profile Name window will open.

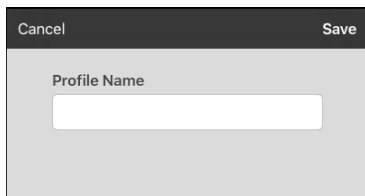3. Tap in the Profile Name field to open a keyboard.
4. Enter a name for the profile (for example, “Mary’s Profile”).
5. Choose **Save**. A profile will be created for Mary with all the current settings.

Note: To create a second profile (for example, “John’s Profile”), repeat these steps, making new settings changes to associate with this profile.

Load a Profile

To load a profile that you previously created:

1. Go to **Menu > Settings > Load Profile**. Profiles that you have created will be listed.


2. Tap the profile name you want. The associated settings will be loaded.

Rename or Delete a Profile


1. Go to **Menu > Settings > Load Profile**. Profiles that you have created will be listed.

2. Choose **Edit**. The delete icon  will appear to the left of each profile name.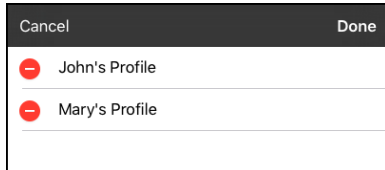

3. Tap the delete icon  for the profile you want to rename. Rename and Delete buttons will appear.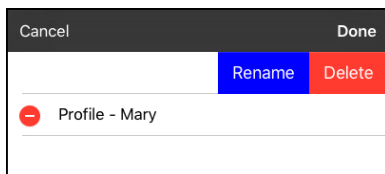


4. Rename or delete the profile.

To rename the profile: Choose **Rename**, enter the new name in the Profile Name field, and choose **Save**.

To delete the profile: Choose **Delete**. The profile will be deleted immediately.

Modify a Profile

1. Make the changes you want associated with the profile.
2. Go to **Menu > Settings > Save Profile**. The Profile Name window will open.
3. Tap in the Profile Name field to open a keyboard.
4. Enter the profile's original name and choose **Save**. A message will inform you that the file name already exists.

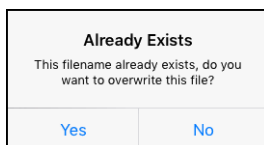5. Choose **Yes**. The changes will be saved.

Changing TouchChat Settings

TouchChat settings are located by going—within the TouchChat app—to **Menu > Settings**. From the TouchChat Settings screen, scroll to see all available settings.

Profiles

Setting	Description
Save Profile	Allows you to save the current settings as a profile.
Load Profile	Allows you to load, rename, or delete a profile.

Navigation

Setting	Description
Page Animation	When turned on, this setting allows you to navigate a page by scrolling up, down, right, or left.
Allow Gestures	When turned on, this setting allows you to add a gesture to a page such as swiping left or right to engage an action.
Gesture Pop Sound	When turned on, this setting provides audible feedback when a gesture is performed.
Button Click Sound	When turned on, this setting provides audible feedback when a button is selected.
Button Navigation Icons	When turned on, this setting displays the small navigation arrows in the top right corner of all buttons. When the setting is turned off, the arrows are hidden.

Page Size

Setting	Description
Status Bar	When turned on, this setting displays the device status bar above the Speech Display Bar.
Keyguard Inset	When turned on, this setting adjusts the display for a keyguard.

Editing

Setting	Description
Menu Passcode	When applicable, enter the password you want to use in this field.
Use Menu Passcode	When turned on, this setting requires a password to access the Editing menus.

Speech

Setting	Description
Voice	Since there are many voice options, voices are downloadable. This will save a lot of space on the app by only downloading the voices needed. To download a voice, tap the green down arrow and select Yes . <i>Note:</i> Some voices are only available at a cost. These voices must be purchased from the Upgrade Store before they are available in the Voice listing.
Rate	Use the slide bar to modify the rate of speech.
Voice Pitch	You can make a voice sound different by modifying the pitch. Select the pitch you want: Lowest, Lower, Low, Normal, High, Higher, or Highest.
Pause Between Words	You can add a speech delay between individual words. <i>Note:</i> This option is not available for a VocaliD voice.
Install VocaliD Voice	Install and set up a VocaliD voice.

Speech Display Bar

Setting	Description
Hide Speech Display	When turned on, this setting hides the entire SDB.
Allow Copy/Paste	When turned on, this setting gives the user the option to copy/paste from the SDB.
Auto Capitalization	When turned on, this setting capitalizes the first word of a sentence automatically.
Show Icons	Choose this setting if you want to display icons with text in the SDB.
Hide Delete Button	When turned on, this setting hides the SDB's Delete button.
Font	Select one of the 7 available fonts for use for text in the SDB.
Font Size	Determines the size of the font that appears in the SDB.
Font Color	Choose one of the 12 available font colors to apply to text in the SDB.
Background Color	Choose one of the 12 available background colors to apply to the SDB.
Number of Lines	Choose how many lines of text you want to appear in the SDB.
Single Line Cursor	Choose this setting if you want a cursor to appear in the SDB.
Tap Action	Choose Speak , Speak and Enlarge Text , or No Action when the SDB is tapped.

Expanded Speech Area

Setting	Description
Font	Select one of the 7 available fonts for use for text in the expanded speech area.
Font Size	Determines the size of the font that appears in the expanded speech area.
Font Color	Choose one of the 12 available font colors to apply to text in the expanded speech area.
Background Color	Choose one of the 12 available background colors to apply to the expanded speech area.

Speak

Setting	Description
Speak Characters	When this setting is turned on, each character will be spoken as entered.
Speak Words	When this setting is turned on, each word will be spoken as entered.
Speak Sentences	When this setting is turned on, each sentence will be spoken as entered.
Auto Clear	When this setting is turned on, the SDB clears automatically after speaking.
Prevent Interruption	When turned on, this setting will prevent interruptions of speaking a sentence if you accidentally hit another button while it is speaking.

Tilt

Setting	Description
Preferred Orientation	Select either Portrait or Landscape.
Tilt Action	Select an option to expand text in the Speech Display Bar and speak it, expand the text only, reorient the screen, or allow no action.

Data Logging

Setting	Description
Enable Logging	Turn on data logging, allowing your device to collect language usage data which you can upload to the Realize Language website for web-based analysis or save to a file to analyze manually.
Privacy Password	Set or change your data logging privacy password.
Clear Log	Clear usage data for the current person.
Realize Language Setup	Upload data collected for a person to the Realize Language website for analysis.

Buttons

Setting	Description
Activate on Release	When this setting is turned on, a button will not activate until released, rather than on touch.
Dwell Time	Controls the delay before a button press is recognized in case a user needs to slide across multiple buttons before pressing the intended button.
Release Time	All buttons are disabled for specified period after each button press event to prevent accidental double button presses.
Force Font Size	When this setting is turned off, the button label font size is automatically reduced as needed to allow the entire label to fit into the button area.
Highlight Touch	When this setting is turned on, the button that is touched is highlighted briefly when activated.
Hide All Icons	When turned on, this setting will hide all of the icons from the buttons and only the text will appear. Note that hiding the button icons will also force the SDB setting "Show Icons" off.
Show Hidden Buttons	When turned on, buttons that have been hidden will be displayed.
Dynamic Grammar Labels	When turned on, Dynamic Grammar Labels displays on button labels the changes that will be made to words when grammar rules are applied. See Appendix A: Advanced Grammar Features .

Social Networking

Setting	Description
Allow Social Networking	When turned on, this setting allows you to share text from the SDB to social networks.

Amp Connect

Setting	Description
Select Amp	Select your amplifier name to make a Bluetooth connection to the TouchChat software for getting proper battery status readings.
Set up Amp	Set left and right speaker volume, bass, and treble.

Scanning

See [Setting up Switch Scanning in TouchChat](#) on page 54.

Updating the TouchChat Application Software

1. Connect the device to the computer.
2. Launch iTunes on the computer.
3. Go to the Apps section of iTunes.
4. Scroll to the Updates section and locate the option “Check for Updates” in the bottom right corner of the screen.

Note: The location of the Updates option depends on the version of iTunes you are using.

5. Enter the Apple ID that you used to purchase TouchChat previously.
6. The update will start downloading.

You can also update TouchChat directly from the device. The App Store icon should show when updates are available to download on the device. When an update is available, the app store icon will display a number on the right hand corner of the icon.

1. Tap on the App Store icon
2. Tap on **Updates** on the bottom right of the screen.
3. The apps that require updating will show up.
4. Tap either **Update** on the application or **Update All** in the upper right hand corner.
5. Enter the Apple ID that you used to purchase TouchChat previously.
6. The update will start downloading.

Analyzing Language Development

You can use the data logging capabilities of your device to collect information on a person's language development. Then you can upload that information to the Realize Language™ website.

This process allows you to monitor, measure and maximize the person's use of the device.

Using the Realize Language Website

Realize Language is a subscription-based online service that organizes and analyzes information and presents the results in easy-to-understand graphic formats that provides valuable insights into each person's language development. Using this service from your computer, you can

- Track the person's progress and communication development over time
- Compare different aspects of communication automatically
- Create a detailed summary of performance
- Share information with others
- Quickly create reports that anyone can understand

To learn more, go to the Realize Language website: <https://realizelanguage.com/info/>

To view or download the *Realize Language Starter's Guide*, go to <https://realizelanguage.com/info/support>

For Customer Support:

If you have a problem with the Realize Language website, email support@realizelanguage.com.



If you have a problem with your device, call Saltillo technical support at 1-800-382-8622 or email service@saltillo.com.

Turning on Data Logging from Your Device

When data logging is turned on, your device collects language usage data which you can upload to the Realize Language website for web-based analysis or save to a file to analyze manually.

1. In TouchChat, tap **Menu** and tap **Settings**. The TouchChat Settings screen will be displayed.
2. Under Data Logging, turn on **Enable Logging**.

When data logging is turned on, the data logging icon appears in the Speech Display Bar.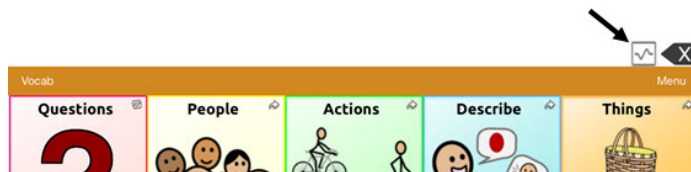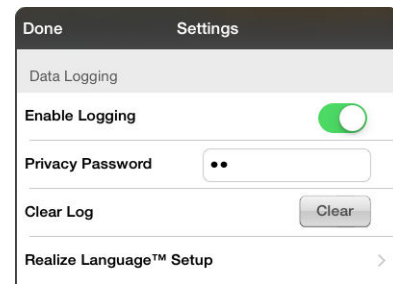


Important! If you plan to use the Realize Language website to analyze data, set up a person before you start collecting data.

Creating a Privacy Password

Data logging provides the option to create a privacy password to protect the data you collect against unauthorized access.


1. Under Data Logging, tap the password box located to the right of **Privacy Password**. A cursor will appear in the password box and keyboard will be displayed.
2. Type the privacy password you want to use and tap **Done**. The Confirm New Privacy Password prompt will be displayed.
3. Type your password again and tap **Enter**. “New Password Set” will be displayed.
4. Tap **OK** to close the message.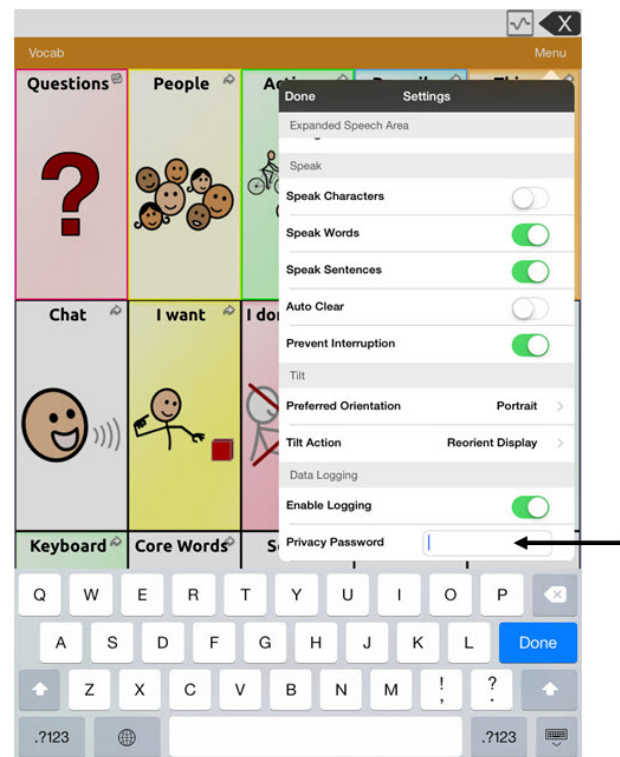

Uploading Data for Analysis

Upload data collected for a person to the Realize Language website for analysis.


1. Tap **Realize Language™ Setup** to open the Realize Language Setup window.
2. *Note:* The first time you upload data, enter your privacy password.
3. Enter your Realize Language account email address and Realize Language account password.
4. Tap the **Current Person** field and select the person for whom you want to upload data.
5. Tap **Upload Now**. “File successfully updated” will appear.
6. Tap **Save**. By tapping **Save**, your Realize Language account email address and Realize Language account password will be retained in the Realize Language Setup window. For future uploads, all you need to do is select the person from the Current Person list.
7. When the data has uploaded successfully, return to the Settings menu and use **Clear Log** to clear the person’s data. Then you can start collecting new data for that person.

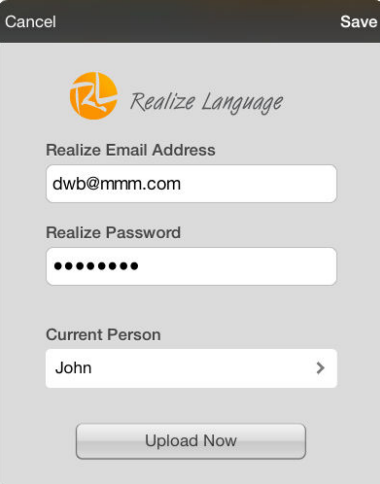A screenshot of the 'Realize Language Setup' window. At the top are 'Cancel' and 'Save' buttons. Below is the 'Realize Language' logo. The form contains three fields: 'Realize Email Address' with the text 'dwb@mmm.com', 'Realize Password' with masked dots, and 'Current Person' with a dropdown menu showing 'John'. An 'Upload Now' button is at the bottom.

Clearing Usage Data

To clear usage data for the current person:

1. Under Data Logging, tap the **Clear** button.
2. The Privacy Password prompt and keyboard will be displayed. Type your privacy password and tap **Enter**.
3. “Do you really want to Clear Log?” will be displayed. Choose **Yes** to clear the data log.
4. “Clear Log Completed” will be displayed. Tap **Okay** to close the message.

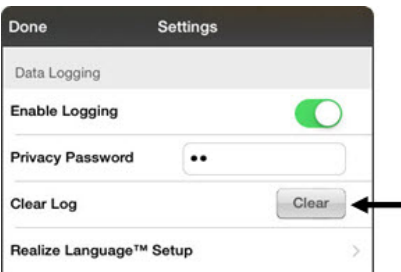
A screenshot of the 'Settings' screen, specifically the 'Data Logging' section. It shows a toggle for 'Enable Logging' which is turned on. Below it is a 'Privacy Password' field with masked dots. A 'Clear Log' button is visible, with an arrow pointing to it from the right. At the bottom is a 'Realize Language™ Setup' link with a right arrow.

Changing or Removing Your Privacy Password

By default, each time you save data to a file, you enter a password. Entering a password helps ensure privacy—only you will be able to view the data. You can change this privacy password as often as you want, or you can eliminate the need for a privacy password.

To change your privacy password:

1. Under Data Logging, tap the password box located to the right of **Privacy Password**. A Privacy Password prompt and keyboard will be displayed.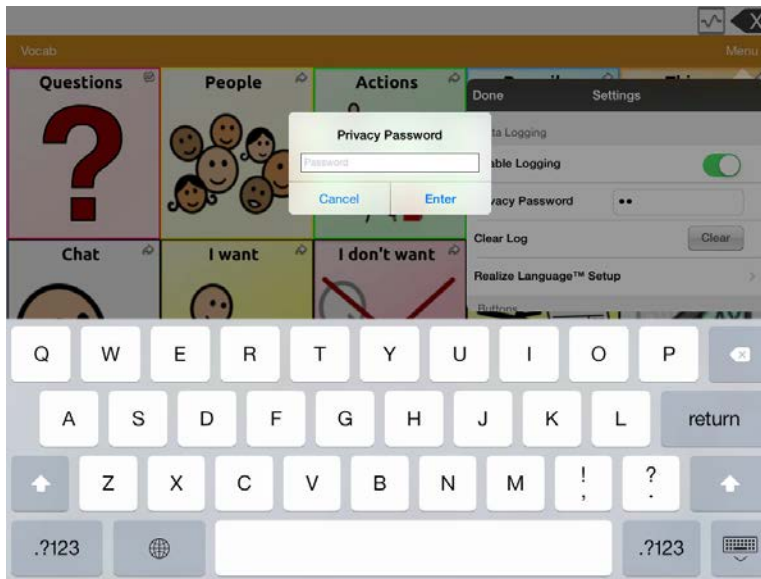

2. Type your old privacy password and tap **Enter**. A cursor will appear in the password box.
3. Use the backspace key  to clear the Privacy Password field.
4. Type your new password and tap **Done**.
5. The Confirm New Privacy Password prompt will be displayed. Type your new password again and tap **Enter**.
6. “New Password Set” will be displayed. Tap **OK** to close the message.

To remove your privacy password:

If you prefer not to enter a password each time you save data to a file, use this procedure. Keep in mind, however, that you will also lose the privacy protection a password provides.

1. Under Data Logging, tap the password box located to the right of **Privacy Password**. A Privacy Password prompt and keyboard will be displayed.
2. Type your old privacy password and tap **Enter**. A cursor will appear in the password box.
3. Use the backspace key  to clear the Privacy Password field and tap **Done**.
4. “Privacy Password Cleared” will be displayed. Tap **OK** to close the message.

Transferring and Copying Vocabulary Files

Note: Once you have loaded TouchChat software version 2.0 on your device, you can only transfer files from your device to another device running version 2.0 or higher. For example, you cannot transfer files from software version 2.0 to a device running version 1.13. However, you can transfer files from a device running a software version lower than 2.0 to a device running version 2.0. For example, you can transfer files from a device running TouchChat version 1.13 to a device running TouchChat version 2.0.

Copying Files To and From iShare

Note: You must have a wireless connection to use this feature.

1. In TouchChat, tap **Vocab** and tap **Choose New Vocab**.
2. Tap **Menu** and tap **iShare Service**.
3. Create a log-in.
4. To download: Tap the appropriate file and download it.
5. To upload: Tap + and then the appropriate file to upload.

Transferring a Vocabulary File from Chat Editor to the Device using iShare

1. Open Chat Editor on your computer.
2. Choose **Library > Transfer > Vocab to File**. The Transfer box will appear.
3. Highlight the file you want to transfer and click on **Transfer** in the top left. A Windows Save As dialog box will appear.
4. Save your vocabulary file to your computer (remember where you save it).
Important! Make sure that “.ce” remains at the end of the file name.
5. After the file has been saved, open Internet Explorer on the computer and go to www.ishareservices.com.
6. Log into your iShare account. If you do not have an account, go ahead and register for one.
7. From the iShare account screen, click **Add File**.
8. On the Add File screen, enter the information that is needed: name, revision, description, and keywords.
9. Select which category you want your vocabulary file associated with.
10. Select which type of file your vocabulary will be. This will make your file private, public, or a group file.
11. Click the **Browse** button, find the vocabulary file you transferred from Chat Editor, click the file, and then click **Open**.
12. Click **Submit**, just below the file area. This will bring you back to the browse file area, in which you should see your file listed.
13. On your device, tap **Vocab** and tap **Choose New Vocab**.
14. Choose **Menu**.
15. Choose **iShare Service**.
16. Log in using the iShare account you set up earlier.
17. Choose the **My Files** folder.
18. Find the vocabulary file in the category where you placed it.
19. After choosing the file, in the file details screen, click the **Download** button on the bottom left.
20. The “Download complete OK” screen should appear when the download is finished.
21. Click the arrow on the top left of the screen until you see a **Done** square and click it. You should be back at the Vocab File screen, where your vocabulary file is listed.
22. Choose your vocabulary file.

Copying a Vocabulary File from the Device to Chat Editor using iShare

1. Turn the TouchChat device on.
2. Tap **Vocab** on the TouchChat device and tap **Choose New Vocab**.
3. Choose **Menu**.
4. Choose **iShare Service**.
5. Choose either **My Files** or **Public Files**.
6. Browse using the All Categories, choose **+**, and select the file you want to upload.
7. When using **+**, you will need to type a short name, description, and keywords for the file. Then choose **Upload**.
8. Choose **OK** on the Upload Complete box that appears on the TouchChat device screen.
9. Open Internet Explorer on the computer.
10. Log into your iShare account at www.ishareservices.com.
11. On your computer, search the categories on the iShare site to find your uploaded file.
12. Click the green arrow that corresponds to your file to download it to your computer.
13. Click **Save** on the file Download box that appears. (Note where you place the file.)
14. Copy the file to the **Chat Editor Import** folder.
15. Open Chat Editor and a progress bar will monitor the download, followed by an Import File progress box.
16. When the import is finished, you can open the downloaded vocabulary file from **Library > My Resources** in Chat Editor.
17. Select the file.

Transferring a Vocabulary File from Chat Editor to the Device using iTunes

1. Open Chat Editor on your computer.
2. Choose **Library > Transfer > Vocab to File**. The Transfer box will appear.
3. Highlight the file you want to transfer and click **Transfer** in the top left.
4. Save your vocabulary file to your computer (remember where you save it).
5. Open iTunes on the computer.
6. Attach the device to the computer.
7. Tap on the device (left column) and go to the Apps section.
8. Scroll down until you see TouchChat in the File Sharing area.
9. Tap **TouchChat**. "Add File" should appear in the lower right-hand section.
10. Tap **Add File**. A Windows Save screen will appear.
11. Locate the file you want to add to your device and tap **Open**.
12. The file should appear in the TouchChat Documents section.
13. Go to the device and go to the Vocab File section.
14. Tap **Menu** in the top right and tap **Import/Export Vocab**.
15. Choose **Import from iTunes**.
16. Select the file you want to import, then tap **Save** in the top right portion of the screen. Your imported file should appear in the Vocab File list.
17. Select the file.

Copying a Vocabulary File from the Device to Chat Editor using iTunes

1. Connect the device to the computer.
2. Open iTunes on the computer.
3. On the device, go to the Vocab File section and tap **Menu** in the top right portion of the screen.
4. Tap **Import/Export Vocab**.
5. Tap **Export to iTunes**.
6. Choose the vocabulary file you want to export.
7. Name the file that you are exporting and tap **Save**.
8. In iTunes on the computer, tap on the device and go to the Apps section.
9. Scroll down until you see TouchChat in the File Sharing area.
10. Highlight the vocabulary file that you are exporting.
11. Click **Save To** on the bottom right.
12. A Windows save box will appear. Select the folder in which you want the file saved.

To place the file into Chat Editor, use the Chat Editor Import folder located on your computer's desktop. The folder was placed there when Chat Editor was installed. If the folder is not there, you can locate the import folder depending on the version of Windows you are using at:

Windows Version	Path to Import Folder
Windows XP	My computer > C drive (local drive) > Documents and Settings > Users (*log in ID*) > Application Data > Chat Editor > Import
Windows Vista or Windows 7, 8, or 10	C: Drive > Users > *User ID* > AppData > Roaming > Chat Editor > Import

Note: The Application Data folder (for Windows XP) or the App Data Folder (for Windows Vista/7) might be hidden. To make these folders visible, go to **Control Panel** and choose **Folder Options**. Then choose the **View** tab, and under Advanced Settings > Hidden Files and Folders, select **Show Hidden Files and Folders**. Then click **OK**. The folders should now be visible. The vocabulary file should start importing automatically after a few minutes.

Sharing a Customized Vocabulary File

To share a customized vocabulary file, both devices need to be running iOS 7 or later. Devices that can share include the following: iPhone 5 or later; iPad 4th generation or later; iPad Mini; iPod touch® 5th Generation or later; iPad Air; iPad Pro®; iPad Flat Out Fun. Also, Wi-Fi and Bluetooth will need to be turned on, and both devices need to remain awake. The distance that the file will transfer depends on the environment.

Sending the File

1. Tap **Vocab** and tap **Choose New Vocab**.
2. Tap **Menu** and tap **Import/Export Vocab**.
3. Tap **Export and Share**.
4. Tap the vocabulary file you wish to export.
5. Make sure the name is correct and tap **Done**.
 - TouchChat will prepare the file to be transferred.
 - Choose the tool you want to use to share the file. For example, AirDrop.
 - A menu will appear at the bottom of the screen with the people that you can share the file with.
6. Tap the user to send the file to. The receiving user will receive a message prompting to accept or decline the transfer.
7. After the user accepts the transfer, the file will start the send.

Receiving the File

Note: Multiple apps can share a vocabulary file: Dropbox®, Google Drive™, Gmail™, etc. This example uses AirDrop.

Turn on AirDrop (if not already turned on)

1. Swipe the bottom of the screen to bring up Control Center.
2. Tap **AirDrop**.
3. Choose an option.

Contacts Only: Makes your device visible only to your contacts. The users must have each other's Apple ID email address (the email that you use for your Apple ID) in their contacts. **Important!** The IDs must be under the category "iCloud". For example, you might have the other user's email under "home" or "work". You must edit your contacts and change the category to "iCloud". If both users do this, they will be able to see each other when sharing with Airdrop.

Everyone: Makes your device visible to everyone who has AirDrop turned on

Receive the file

1. After the file has been transferred, tap **Yes** to import the vocabulary file.
2. Name the file accordingly, and then tap **Save**.
3. Once the file has been imported, tap **Vocab** and tap **Choose New Vocab**.
4. Find the file that was transferred and open it.

Setting up and Using Switch Scanning


As an alternative to directly choosing each button on the screen, single-switch and two-switch scanning are available.

Enabling Scanning

Press the device's **Home** button. Then choose **Settings > General > Accessibility > Switch Control**. Turn Switch Control **on**.

Defining the Switches

You can either use an external switch that is plugged into device switch jack A or B or you have the option to use the screen as a switch.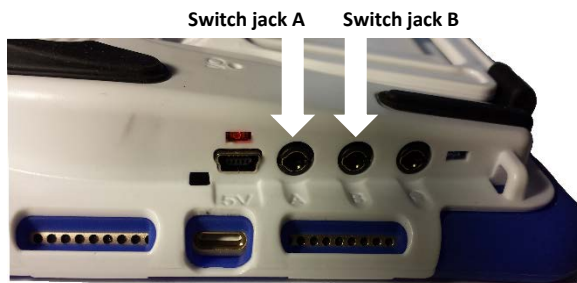

TouchChat Express 8 used as example

To use an external switch:

1. Plug the switch into either switch jack A or B. If you are using two switches, plug a switch into each.
2. Choose **Settings > General > Accessibility > Switch Control**.
3. Choose **Switches > Add New Switch > External**.
4. Press the external switch.
5. Name the switch.
6. Choose the switch action to perform.
 - If you are using just one switch, choose **Select Item**.
 - If you are using a second switch, choose **Add New Switch**. For the switch action, you could choose **Move To Next Item**.
 - If you need another option, you can use the screen as a switch.

After setting up your switch:

Press the device's **Home** button. Then choose **Settings > General > Accessibility > Switch Control**. Turn Switch Control **off** and then back **on**.

Setting the Scanning Style

You can choose to have the system automatically scan or to step to each location. With step-scanning, a switch must be pressed to move the scan to each button, one at a time. With auto-scanning, a switch must be pressed to start the scan and the system will continue to scan until the switch is pressed again.

To set the scanning style, choose **Settings > General > Accessibility > Switch Control > Scanning Style**. Three scanning styles are available: Auto Scanning, Manual Scanning, and Single Switch Step Scanning.

Note: Long Press is not currently available within the TouchChat app.

Auto Scanning

Auto Scanning moves from item to item automatically after a specified duration. When Auto Scanning is selected, these timing settings are available.

Setting	Description
Auto Scanning Time	Determines how fast scanning performs with Auto Scanning turned on
Pause on First Item	Determines how long Auto Scanning will wait after input
Loops	Determines how many times each item is visited prior to stopping the scan
Move Repeat	Determines how long before “Move to Next/Previous Item” is repeated while a switch remains pressed

Manual Scanning

Manual Scanning allows the user to step from item to item by activating one switch and then select an item by activating a second switch. When Manual Scanning is selected, these timing settings are available.

Setting	Description
Auto Hide	Determines how long it takes for the scanning interface to disappear due to inactivity
Move Repeat	Determines how long before “Move to Next/Previous Item” is repeated while a switch remains pressed

Single Switch Step Scanning

Single Switch Step Scanning allows the user to step from item to item by activating the switch. When the user reaches a desired item, he or she stops activating the switch for a specified dwell time. The item is then selected automatically. When Single Switch Step Scanning is selected, these timing settings are available.

Setting	Description
Dwell Time	Determines the duration before the user can select an item.
Auto Hide	Determines how long it takes for the scanning interface to disappear due to inactivity
Move Repeat	Determines how long before “Move to Next/Previous Item” is repeated while a switch remains pressed

Additional Timing Options

These timing options are available under “Switch Stabilization”.

Setting	Description
Hold Duration	This option sets the duration the switch must be held before a press is recognized. This duration must be less than the Auto Tap time so that a double input press can be used.
Ignore Repeat	This option sets the duration in which multiple presses of input are treated as a single input. This duration must be less than the Auto Tap time so that a double input press can be used.

Configuring Additional Scan Settings

Tap Behavior

By default, when the user selects an item, a pop-up menu appears, providing options for interacting with that item (Tap, Scroll, etc.). Two alternative behaviors are available. To access these alternatives, choose **Settings > General > Accessibility > Switch Control > Tap Behavior**.

Setting	Description
Auto Tap	Turning Auto Tap on hides the pop-up menu, unless the user activated the switch within a specified duration.
Always Tap	Turning Always Tap on displays the pop-up menu at the end of the scan. This allows the user to select an item with a single switch activation and scan to the pop-up menu to access it. The pop-up menu appears in the middle of the screen.

Always Tap Keyboard Keys

When Always Tap Keyboard Keys is turned on, keyboard keys will always be selected with a single switch activation instead of displaying a pop-up menu immediately when selected.

Scan Audio

Scroll to the “Audio” section and turn **Sound Effects** and **Speech** on or off.

Menu Scanning

To include all menus in scans, leave **Included Menu Items** set at **All**.

To exclude one or more menus from scanning, tap **Included Menu Items** to open a list of menus. Then tap the name of each menu you want to exclude to remove the checkmark.

Scanning Cursor Size and Color

The “cursor” is the highlight that appears around the scanned button area. To change the cursor size and color, scroll to the “Visual” section.

To double the thickness of the highlight, turn **Large Cursor** on. To change the color of the cursor, tap **Cursor Color** to open a list of color options. Blue, Red, Green, Yellow, and Orange are listed, and a checkmark appears after the color currently selected. Tap the color you prefer.

Pause on First Item

Pause on First Item allows you to find the next item you want to select before a scan resumes. When this setting is turned on, a scan will pause on the first item for the number of seconds you specify and will pause again after the item has been selected and the scan resumes.

Setting Loops

When Auto Scanning is turned on, the Loops setting determines the number of times a scan will go through all items and start over automatically. When the set number of loops finishes, the scan will stop. You can start the scan again by activating the switch.

Sound Effects

When Sound Effects is turned on, a clicking sound is activated as a scan moves from one item to another.

Speech

When Speech is turned on, an item is spoken as well as highlighted when a scan selects it.

Menu Items

When Menu Items is turned on, you can select what items show up in the pop-up menu during a scan.

Group Items

When Group Items is turned on, a scan display items on the screen in groups instead of individually. This allows you to scan between groups, select a group, and then select an item within the group.

Cursor Color

Use the Cursor Color setting to change the color of the scan border to make it easier to see as it moves.

Long Press

Note: Long Press is not currently available within the iOS software.

Dwell Time

When Step Scanning is selected, Dwell Time allows you to set the number of second the user must wait for an item to be selected on the screen automatically.

Auto Hide

When Step Scanning is selected, Auto Hide will make the scanning cursor disappear if the user does not activate the switch within a predetermined number of seconds. Activating the switch will redisplay the cursor.

Using Keyboards

Your iPad allows you to use one or more on-screen keyboards or an Apple Wireless Keyboard.

Using On-screen Keyboards

You can change a keyboard's text entry options and layout; split and move a keyboard; or add keyboards and switch between them.

Setting Text Entry Options

Choose **Settings > General > Keyboard**. Then turn the options under "All Keyboards" on or off.

Changing the Keyboard Layout

1. Choose **Settings > General > Keyboard > Keyboards**.
2. Select a keyboard.
3. Choose a keyboard layout.

Using a Split Keyboard

1. Choose **Settings > General > Keyboard** and turn **Split Keyboard** on or off.
2. With an on-screen keyboard displayed, touch and hold the Keyboard key  and slide your finger to **Split** and release. The keyboard will split in half and move to the middle of the screen.
3. You can merge and move the keyboard as follows:
 - To return to the default keyboard and leave it in the middle of the screen, touch and hold  and slide your finger to **Merge** and release.
 - To return to the default keyboard, docked at the bottom of the screen, touch and hold  and slide your finger to **Dock and Merge** and release.
 - To move the default keyboard to the middle of the screen, touch and hold  and slide your finger to **Undock** and release.
 - To return to the default keyboard to the bottom of the screen, touch and hold  and slide your finger to **Dock** and release.

Adding a Keyboard for Another Language

Choose **Settings > General > Keyboard > Keyboards > Add New Keyboard**. Then select a keyboard.

Switching or Hiding Keyboards

If you have not added any keyboards, tap  to switch to the emoji keyboard.

If you have added keyboards, tap  to switch to the one you used previously. Continue tapping to access other available keyboards, or touch and hold , and then slide to choose a different keyboard.

To hide the onscreen keyboard, tap the Keyboard key .

Using the Apple Wireless Keyboard

You can use the Apple Wireless Keyboard to enter text. The keyboard connects via Bluetooth, so you must first pair it with iPad.

Note: The Apple Wireless Keyboard may not support keyboard features that are on your device. For example, it does not anticipate your next word or automatically correct misspelled words.

To pair an Apple Wireless Keyboard with the iPad:

1. Turn on the keyboard.
2. Choose **Settings > Bluetooth** and turn on Bluetooth.
3. Tap the keyboard when it is displayed in the Devices list.

Once the keyboard is paired, the keyboard reconnects to the iPad whenever it is in range. When the Apple Wireless Keyboard is connected, the onscreen keyboard does not appear.

To unpair a wireless keyboard:

1. Choose **Settings > Bluetooth**.
2. Tap next to the keyboard name.
3. Tap **Forget this Device**.

Using an External Keyboard with TouchChat

If you need to use an external keyboard with TouchChat, TouchChat needs to give a command to replace the TouchChat keyboard with a system keyboard. If an external keyboard is not attached, the iOS keyboard will appear on the screen. The command to use the system keyboard needs to be stored to a TouchChat button for turning on and off.

1. Go to the page where the button will be created.
2. Tap **Menu** and tap **Edit Page**.
3. Tap the button where the system keyboard button will be located.
4. Select **Edit This Button**.
5. Add a button label and, if needed, an image.
6. Modify the button style, if necessary.
7. In the Button Actions section, tap **Add**.
8. Tap **System Keyboard**.
9. Tap **Save**.

Entering and Modifying Text

Use the onscreen keyboard or an Apple Wireless Keyboard to enter text.

Typing Text

Tap a text field to open the onscreen keyboard and then tap letters. If you touch the wrong key, slide your finger to the correct key. A letter is entered when you release your finger from the key.

To ...	Do this:
End a sentence with a period and space quickly	Double-tap the space bar.
Enter uppercase text	Tap Shift to type uppercase or touch the Shift key and slide to a letter. Double-tap Shift to engage caps lock. To enter numbers, punctuation, or symbols, tap the Number key 123 or the Symbol key #+= .
Enter accented letters or other special characters	Touch and hold a key and then slide to choose one of the options.

Note: If you see a word underlined in red, tap it to see suggested corrections. If the word you want is not displayed, type the correction.

Working with Predictive Text

As you enter text, the keyboard will predict your next word. Tap a word to choose it, or accept a highlighted prediction by entering a space or punctuation mark. When you tap a suggested word, a space appears after the word. If you enter a comma, period, or other punctuation mark, the space is deleted.

To ...	Do this:
Reject a suggestion	Tap your original word, which is shown as the predictive text option with quotation marks.
Hide predictive text	Pull down the suggested words. If you want to see the suggestions again, drag the bar up.
Turn off predictive text	Touch and hold  or  , and then slide to Predictive .

Correcting Misspelled Words

Even if you turn off predictive text, the keyboard will continue to suggest corrections for misspelled words.

To ...	Do this:
Accept a correction	Enter a space or punctuation, or tap Return .
Reject a correction	Tap the “x.” If you reject the same suggestion a few times, it will no longer be suggested.

Modifying Text

To modify text, do the following:

1. Touch and hold the text to display the magnifying glass.
2. Drag to position the insertion point.
3. Tap the insertion point to display the selection options.
 - Double-tap a word to select it.
 - Drag the grab points to select more or less text.
 - In read-only documents, such as webpages, touch and hold to select a word.
4. Cut, copy, or paste over selected text.

Note: With some apps, you can also get bold, italic, or underlined text (tap B/I/U); get the definition of a word; or have the iPad suggest an alternative. You may need to tap to see all the options.

5. To justify text, select the text, and then tap the left or right arrow.
6. To undo your last change, shake the iPad and then tap **Undo**.

Using Shortcuts

A shortcut lets you enter a word or phrase by typing just a few characters. For example, type **omw** to enter “On my way!” You can also add your own shortcuts.

To create a shortcut, choose **Settings > General > Keyboard > Shortcuts**.

Hint: If you have a word or phrase you use that you do not want corrected, create a shortcut, but leave the Shortcut field blank.

Use iCloud to keep your personal dictionary up to date on your other devices. Choose **Settings > iCloud** and turn on iCloud Drive or Documents & Data.

Dictating Text

You can use the iPad to dictate text as well as type.

To turn on dictation:

1. Make sure the iPad is connected to the Internet.
2. Choose **Settings > General > Keyboard > Enable Dictation**.

To dictate text:

1. Tap the iPad keyboard.
2. Speak the words you want to dictate.
3. When you finish, tap **Done**.
4. Tap again to continue dictating.

To insert text, tap to place the insertion point first. You can also replace selected text by dictating.

To add punctuation or format text:

Say the punctuation or format. For example, “Dear Mary comma the check is in the mail exclamation mark” becomes “Dear Mary, the check is in the mail!” Punctuation and formatting commands include:

- quote ... end quote
- new paragraph
- new line
- cap—to capitalize the next word
- caps on ... caps off—to capitalize the first character of each word
- all caps—to make the next word all uppercase
- all caps on ... all caps off—to make the enclosed words all uppercase
- no caps on ... no caps off—to make the enclosed words all lowercase
- no space on ... no space off—to run a series of words together
- smiley—to insert :-)
- frowny—to insert :-(
- winky—to insert ;-)

Searching Apps

Many apps include a search field that allows you to enter search criteria to find something within the app.

Using Spotlight Search

Spotlight Search not only searches your iPad, but also shows suggestions from the App Store and the Internet. You can also use Spotlight Search to find and open apps.

1. Drag down the middle of any Home screen to reveal the search field.
2. Type your search criteria. Results are displayed as you type.
3. To hide the keyboard and see more results on the screen, tap **Search**.
4. Tap an item in the list to open it.

Choosing Apps and Content to Search

By default, all apps and content listed under “Search Results” are selected for Spotlight Search. To deselect apps and content that you do not want included in searches, do the following:

1. Choose **Settings > General > Spotlight Search**.
2. Tap any apps or content that you do not want to include in searches.

Limiting Spotlight Search to Your iPad

1. Choose **Settings > General > Spotlight Search**.
2. Turn off **Spotlight Suggestions**.

Turning off Location Services for Spotlight Suggestions

1. Choose **Settings > Privacy > Location Services > System Services**.
2. Turn off **Safari & Spotlight Suggestions**.

Changing the iPad Language and Word Prediction Dictionaries

TouchChat provides American English, British English, and Spanish word prediction dictionaries. The device's Language settings determine which dictionary is used. Set up your device for the language you currently use. To select your language:

1. Choose **Settings > General**.
2. Select the **Language & Region**.
3. Select **iPad Language**.
4. Select your language.

Setting up Restrictions

Use this feature to turn off access to Safari, YouTube, iTunes, installing apps, deleting apps, Location, In-App Purchase, etc. Also set restrictions on content (music, movies, and TV shows).

To turn off items you would like to restrict, from the device:

1. Tap **Settings**, then **General**, and then **Restrictions**.
2. Tap **Enable Restrictions**.
3. Create a four-digit passcode.
4. Enter the passcode twice to set restrictions.


Changing Screen Orientation

Important! TouchChat screen orientation settings do not affect iOS screen settings. However, when screen orientation is locked in iOS, it is locked in TouchChat as well, regardless of the TouchChat setting.

Changing Screen Orientation in TouchChat

Note: TouchChat screen orientation settings do not affect iOS screen settings.

1. Tap **Menu** and tap **Settings**.
2. Scroll to the Tilt section and tap **Tilt Action**.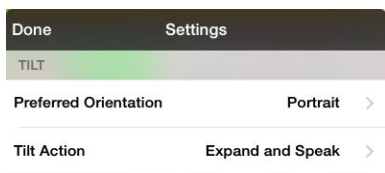

3. Tap one of the options:


Option	Description
Expand and Speak	When you rotate the device, text in the Speech Display Bar (SDB) will expand to fill the screen and the device will speak it.
Expand Only	When you rotate the device, text in the Speech Display Bar (SDB) will expand to fill the screen, but the device will not speak it.
Reorient Display	When you rotate the device, the screen orientation will switch between portrait and landscape.
No Action	When you rotate the device, the screen orientation will not change. Screen orientation will be locked.

4. Tap **Done**.

Locking Screen Orientation in TouchChat

Note: TouchChat screen orientation settings do not affect iOS screen settings.

1. Tap **Menu** and tap **Settings**.
2. Scroll to the Tilt section and tap **Tilt Action**.
3. Tap **No Action**.
4. Tap **Done**.

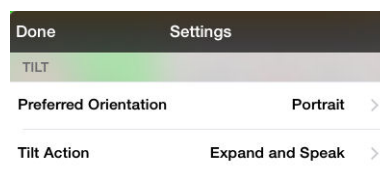Locking Screen Orientation in Control Center

1. Swipe from the bottom of the screen to open **Control Center**.
2. Tap the **Orientation Lock** icon .
3. The orientation lock icon at the bottom of the screen will be highlighted, and the orientation lock icon will also appear in the status bar at the top of the screen.

Attaching a Strap

If You Received Loops with Your Strap:

Your package included a strap and two loops for attaching the strap. Do not attach a strap directly to the plastic case as this may damage the device.

The loops are potential choking hazards. Use them with caution.

Decide whether you want to carry the unit in landscape or portrait orientation. Attach two loops to the appropriate side as shown.

Step 1: Feed the end of the loop through the hole in the case.

Step 2: Thread the metal ring through the loop.

Step 3: Pull the metal ring to tighten the loop.

Step 4: Attach the connectors on the ends of the strap to the metal rings on the loops.

If you received metal rings with your strap, see the next page.

If You Received Metal Rings with Your Strap:

Your package included a strap and two metal rings for attaching the strap. Do not attach a strap directly to the plastic case as this may damage the device. For devices that include a handle, simply attach the strap to the bottom holders if you remove the handle.

 The metal rings are potential choking hazards. Use them with caution.

Decide whether you want to carry the device in landscape or portrait orientation. Attach two metal rings to the appropriate side as shown.

Step 1: Open the ring as you would open a key ring. *Suggestion:* Use your fingernail, a screwdriver, or a dime.

Step 2: Push the open ring onto the holder in the case.

Step 3: Allow the ring to close on the holder.


Step 4: Clip the shoulder strap to the ring.

Positioning, Removing, or Reattaching the Stand

Your device comes with a stand that can be positioned for easy viewing. If you do not want to use the stand, you can remove it from the back of the device. You can reattach it easily at any time.

Positioning the Stand

When closed, the stand is held in place by a latch.

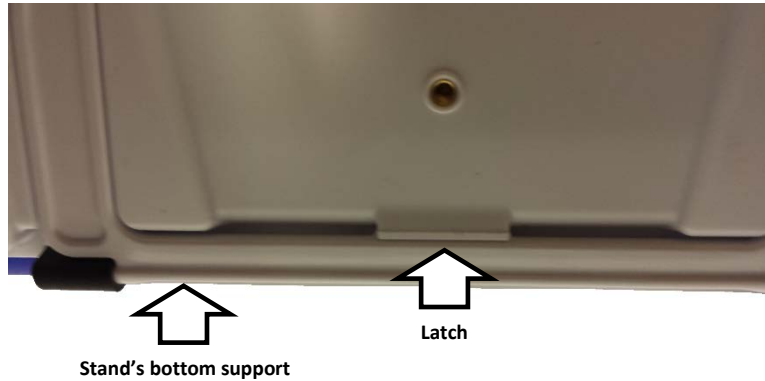To position the device at an angle for easy viewing of the screen, do the following:

1. Place the device face-down on a flat surface.
2. While holding your thumb on the edge of the device below the stand's bottom support, press on the **inside** of the support with the tip of your index finger (or the tips of several fingers) to pull the support **away** from the latch.
3. Lift the support away from the back of the device until the legs latch into place. Then position the device.

Removing the Stand

If you do not want to use the stand, you can remove it from the back of the device.

1. Position the stand so that it is not latched in place.
2. Insert the tip of a flat-head screwdriver into one of the hinges as shown.
3. Press in on the end of the leg until the leg disengages from the holder.
4. Pull the leg out of the holder.
5. Pull out the other leg.

Note: Be sure to store the stand in a safe location in case you want to reattach it later.

When the stand is removed, it becomes a potential hazard for choking or for poking the eye.

Reattaching the Stand

If you remove the stand, you can reattach it easily at any time.

1. Squeeze near the end of one leg of the stand with your thumb and index finger.
2. Push the leg into the holder until it snaps into place.
3. Repeat for the other leg.

Removing the Handle

If you do not plan to use the handle, you can remove it by removing the screws that hold it in place.

Remove the two screws

The handle and screws are potential choking hazards.

Attaching a Wheelchair Mounting Plate

Three threaded holes are provided on the back of the device for attaching an optional wheelchair mounting plate. If you purchased the wheelchair mounting plate accessory, follow these instructions to attach the plate to your device.

1. Place the device face-down on a flat surface. Be careful not to scratch the display.
2. Align the three tapered holes in the mounting plate with the three threaded holes on the back of the device.

Important! Be sure that the mounting plate is oriented so that the additional hole for the holding pin is in the upper left.


3. Insert and tighten the screws.


Replacing the Rubber Boot

To replace the rubber boot, follow these instructions.

1. Remove the existing plastic shell/rubber boot assembly from the device.
2. Remove the existing boot from the shell.
3. Fit one end of the device into the replacement rubber boot. Be sure to line up the buttons and ports on the device's sides with the corresponding receptacles and openings in the boot.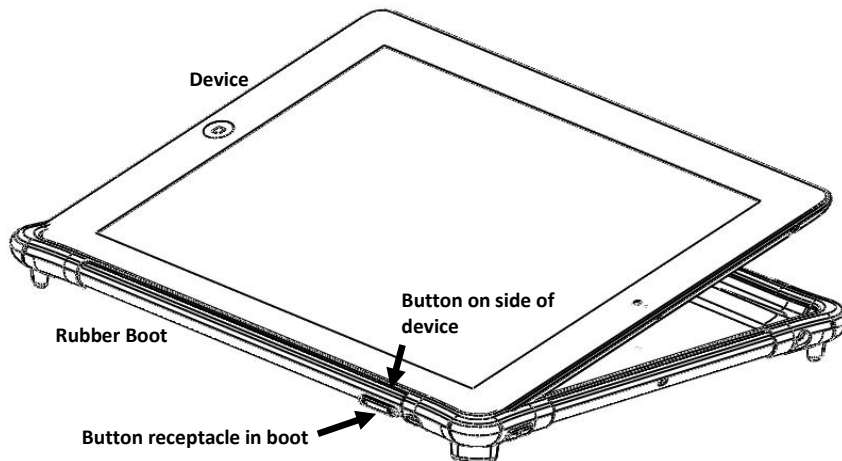

4. When you are sure that the device and boot are properly aligned, push the device down into the boot and pull the edges of the boot over the sides until the device is fully “wrapped” in the boot on all sides.
5. Fit one end of the device—wrapped in the boot—into the shell. Be sure to line up the buttons and ports on the device's sides with the corresponding openings in the shell.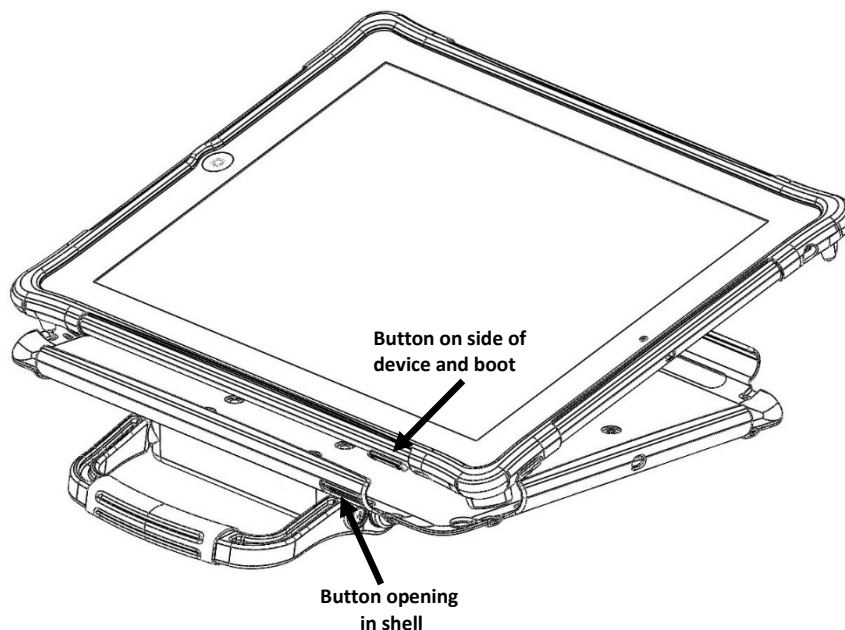

6. When you are sure that the device/boot assembly is properly aligned, carefully push each side into the shell until all sides snap into place.

Cleaning and Disinfecting

If your TouchChat Express will only be used by one person, routine cleaning and disinfecting should be sufficient. If, however, more than one person will be using the unit, take additional measures to clean and disinfect the unit after it has been used by one person before allowing it to be used by another.

Routine Cleaning and Disinfecting

Important! Do not use solvents or abrasives or spray water or cleaner on the unit.

Cleaning the touchscreen

The touchscreen is very sensitive. Fingerprint smears, dust, grime, saliva, etc. will affect its performance. The screen also reacts to raindrops and extremes in temperature (condensation).

To dust the touchscreen, use a soft, lint-free cloth. For heavier cleaning, use a slightly dampened, lint-free cloth and then dry the screen with another soft, lint-free cloth.

Cleaning the TouchChat Express components and keyguard

The unit can be disassembled and the TouchChat Express components and keyguard washed in hot, soapy water. Dry them thoroughly before reassembling the unit. The best practice for regular cleaning is to dry accessories by hand, lay them on a dry towel, and let them air dry for about 5 minutes before placing them back on the device.

Disinfecting

To disinfect everything but the display, use a solution of vinegar and water ($\frac{1}{4}$ cup vinegar to 1 cup water). Wipe the rubber boot (not the display) using a cloth dampened with this solution.

Wash the plastic shell and keyguard separately with the vinegar and water solution. Allow all equipment to air dry. **Do not use vinegar and water on the display!**

What about drooling/saliva?

Any saliva should be wiped up immediately, especially on the touchscreen or around any of the connectors. Use a slightly dampened, lint-free cloth to wipe the device.

What about spills, rain, or accidental immersion?

If you spill liquid or any runny substance on your TouchChat Express; if you are caught in a downpour; or if the unit is accidentally dropped in liquid, immediately dry the components as thoroughly as possible. Try using the device. If it does not work, call the Saltillo Service Department and tell them what happened. They may ask you to send in your device for servicing.

Disinfecting TouchChat Express and Accessories for Multiple Users

Note: This section applies only when more than one person will be using the same unit. In that situation, it is vital to effectively disinfect the unit and accessories that have been used by one person before allowing them to be used by another.

First Step: Put on protective gloves

Next Step: Clean the TouchChat Express components and accessories

1. Wipe down the TouchChat Express components and accessories with disinfectant wipes (Virucidal, Bactericidal, Pseudomonacidal, Tuberculocidal, Fungicidal)—Metrex CaviWipes™. Follow the disinfectant product manufacturer's instructions for *cleaning*.
2. Take a Q-tip with cleaner applied (for example, Windex® or another commercial cleaner) and clean inside all cracks and crevices. You may want to use a toothbrush or similar brush.
3. Blow the components off with an air hose, or wipe it dry.

Important! Make sure all soil is removed from the TouchChat Express components and accessories. This is vital before proceeding to disinfecting the equipment.

Next Step: Disinfect the TouchChat Express components and accessories

1. Wipe down the TouchChat Express components and accessories with a new disinfectant wipe (Virucidal, Bactericidal, Pseudomonacidal, Tuberculocidal, Fungicidal)—Metrex CaviWipes. Follow the disinfectant product manufacturer's instructions for *disinfecting*.
2. Make sure to clean inside all cracks and corners, and wipe more than once if necessary to keep the components wet for a minimum of three (3) minutes.
3. Allow the components and accessories to dry.

Final Step: Wipe the touchscreen

1. When the TouchChat Express components and accessories are dry, wipe down the viewing area (device touchscreen, keyguard, etc.) with glass cleaner, so the screen does not discolor.
2. Allow the equipment to dry.

Troubleshooting

Speakers/amplifier no longer produce sound. / Sound is limited to the device's speakers only.

First try this: Make sure the amplifier has been charged. If the amplifier is charged ...

Next try this: Make sure Airplane Mode is off. Press the device's **Home** button and choose **Settings**. If Airplane Mode is on, turn it off.

Then try this: Check the blue LED on the side of the amplifier. If it is flashing, the amplifier needs to be reconnected. Choose **Settings > Bluetooth** and select the Bluetooth identifier from the Bluetooth "Devices" list.

To contact Saltillo for operational or technical support

Phone 1-800-382-8622

Email service@saltillo.com

Warranty

Saltillo Corporation warrants the TouchChat Express to be free from defect in material and workmanship under normal use for the device warranty period of one year from date of purchase.

Please contact the Saltillo Service department (1-800-382-8622, option 1) prior to making alterations to the TouchChat Express for mounting, transportation, etc.

All warranty service should be arranged through Saltillo Corporation prior to sending a system back. Please call the company at 1-800-382-8622, option 1, or email at service@saltillo.com to obtain a Return Authorization number prior to sending the system to Saltillo.

If replacement is necessary, the replacement TouchChat Express may be a new or re-conditioned TouchChat Express of equal value.

Note: Saltillo Corporation considers bending of the unit, submerging it into liquid of any kind, or any breaks and/or cracks in the LCD screen to be misuse. However, Saltillo will cover one screen replacement or one iPad replacement per warranty year.

Shipping address:

Saltillo Corporation
Attn: (Insert RA # here)
1022 Heyl Road
Wooster, OH 44691


Appendix A: Advanced Grammar Features

The features explained in this appendix are intended for persons with extensive knowledge of grammar. Please note that this is an alternative way to set up grammar within a vocabulary file. WordPower files already have a grammar in place.

Creating a Grammar Button

To create a button that changes a word to a different grammatical form:

1. Go to the page with the button location you want.
2. Tap **Menu > Edit Page**.
3. Press and hold the button you want to use.
4. Tap **Edit This Button**.
5. Add a label. For example, enter **-s**.
6. Scroll down to the Button Actions section.
7. Tap **Add**.
8. Tap **Grammar Action**. The Grammar actions will be displayed.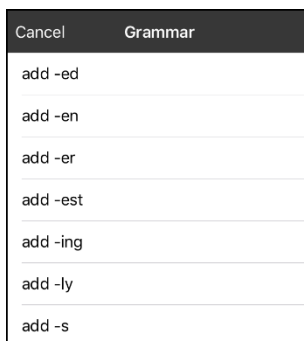

9. Tap the action you want. For example, tap **add -s**. The action will be listed in the Button Actions section.
10. Tap **Save**. In this example, a button labeled “-s” will be available to pluralize all nouns.

Identifying Parts of Speech

To apply a part of speech to a button:

1. Navigate to a page of “things” (nouns).
2. Choose **Menu > Edit Page**.
3. Press and hold a “thing” button (for example, **car**), choose **Edit This Button**, and scroll down to the **Actions** section.
4. Choose **Add** to open the list of actions.
5. Select **Apply Grammar Property**. The Part of Speech menu will list adjective, noun, and verb options.
6. Select the applicable part of speech (for example, **noun**).


7. Choose **Save**. In this example Apply Grammar Property (noun), will appear at the bottom of the list of actions defined for the button.

Important! The grammar property must appear first in the list of actions. You'll need to move it to the top of the list.

8. Tap **Edit**. A delete icon  will appear in front of each action and gray bars  will appear to the right. Touch and hold the gray bars  to the right of the Apply Grammar Property (noun) action and drag it to the top of the list of actions.
9. Choose **Save**.
10. Repeat steps 3 through 10 for the remaining "thing" buttons on the page.

Creating a Grammar Rule Button

1. Press and hold the button you want to set up as a pluralize button, choose **Edit This Button**, enter a label for the button (for example "-s"), and choose the **Actions** tab.
2. Choose **Add** to open the list of actions.
3. Select **Apply Grammar State**. The Grammar menu will display a list of options.

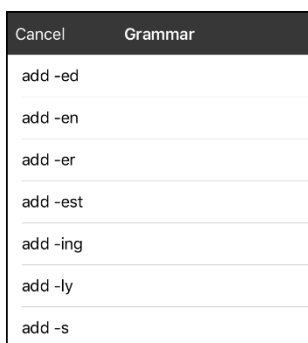These options perform as follows:

Option	Application
add -ed	verb simple past tense
add -en	verb past perfect participle
add -er	adjective comparative
add -est	adjective superlative
add -ing	verb present participle/gerund
Add -ly	adverb
add -s	noun plural

4. Select the applicable option (for example, **add -s**).
5. Choose **Save**. In this example, Apply Grammar State (add -s) will appear in the list of actions defined for the button.
6. Remove any other actions that were already on the list.
7. Choose **Save**. In this example, a button labeled "-s" is now available to pluralize all words identified as nouns.

Using Your Grammar Rule Button

Place the grammar rule button on a page of applicable items. For example, place a pluralize button on a page of things (nouns). If you tap your pluralize button and then choose a thing, you will need to tap the pluralize button again before pluralizing another thing.

1. In the Speech Display Bar (SDB), enter the words you want to speak **up to** the thing you want to pluralize. For example, enter “I want more”.
2. Tap the “-s” button you created in the previous section, [Creating a Grammar Rule Button](#).
3. Choose the thing button you want (for example, **car**). The SDB will display “I want more cars”.

Clearing a Grammar Rule

You may find it useful to create a Clear Grammar State button.

If you tap the pluralize button and decide you don’t want to pluralize a word on the current page, you can move to another page of things, select a noun from that page, and that noun will be pluralized.

If, however, you tap the pluralize button and decide you don’t want to pluralize any word, the software will pluralize the next noun you select unless you clear the pluralize state. In this case, after you tapped the pluralize button and then changed your mind, you would navigate to your Clear Grammar State button and tap it. From that point forward, nouns won’t be pluralized until you tap your pluralize button.

To create a button for clearing the current grammar state:

1. Go to the button you want to use and choose **Menu > Edit Page > Edit This Button**.
2. Give the button a label, scroll down to the Actions section, select **Add**, and select the action **Clear Grammar State**,
3. Remove any other actions that were already on the list.
4. Choose **Save**. This button will now clear any grammar state that has been activated.

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